


# 極尚椅

MAX Gorgeous

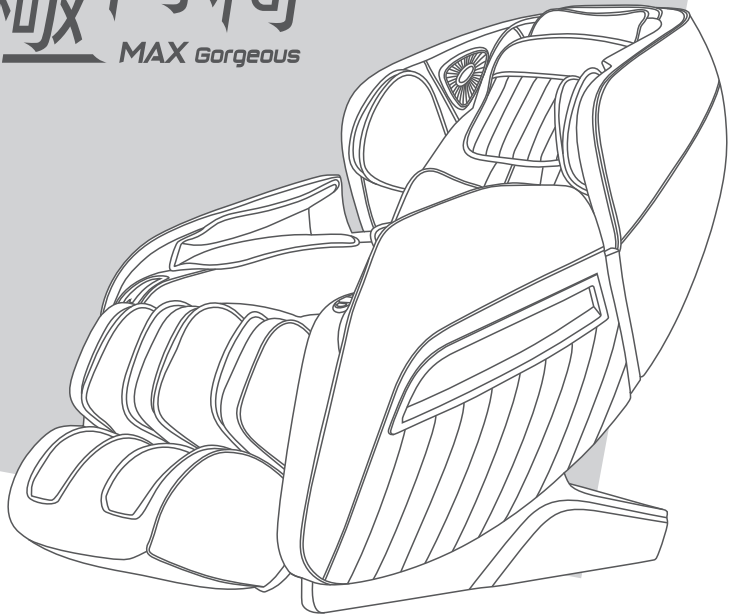

**User Manual**

**MAX-77A305**





## COMPANY PROFILE

After two decades of experience in the health industry, **maxcare** founder utilized his knowledge and expertise in massage therapy from an Asian-Pacific perspective to establish our brand - **maxcare**. Our massage therapy business was expanded to Hong Kong in 2012 and is conferred 2017 Hong Kong Top Brand Award. Based in Hong Kong, we are specializing in design & innovation, manufacturing, and retailing of massage and healthcare products.

Believing in the value of 'Technology improves our quality of life' and 'Caring for your family', **maxcare** has been innovating highly qualified products with international patents. Besides, **maxcare** products fulfill various international safety standards, namely EU CE Mark, US UL/ETL Mark, Germany GS Mark, TUV certification, and China CCC Mark.

Motivated by our value, **maxcare** founder has been leading us to be one of the most competitive brands in Hong Kong health industry. **maxcare** applies the contemporary fashion design – neat and modern – on the mechanical engineering of massage chairs. To deliver our value and offer incredible massage experience and healthcare solution to the public, **maxcare** devotes to the innovation, design, manufacturing the top-notch family massage products. By integrating the company philosophy – Spirit Of Healthy Life - into our products, **maxcare** brings health to everyone.

# 極尚椅

MAX Gorgeous

Thank you for purchasing the **maxcare**® products. Before using the product, please read this manual thoroughly, particularly the Safety Precautions section, for the correct use of the product.

This manual should be well kept for easy reference. It enables you to perform a safe and efficient operation.

## CONTENT

---

Safety Precautions .....	P.01 - P.02
Safety Warning .....	P.03 - P.04
Name and Function of Components .....	P.05 - P.07
How to Use .....	P.08 - P.17
Cleaning and Maintenance .....	P.18 - P.19
Technical Specifications .....	P.19



## Safety Precautions



### 1. Important safety warning

- Keep children and pets away from the movable parts of this product for the avoidance of accidents caused by the up-and down-ward movement of the calf rest and back cover.
- When in use, sliding or transporting, please ensure that there are no people or pets nearby (the back, bottom, front and sides of the product), and that no foreign objects or pets inside the product, to avoid accidents.
- Please use well-grounding power supply suited to this product.
- Please pull out the plug after use or before cleaning to avoid injury or damage to the product.
- Don't use accessories which are not recommended.
- Don't use this product outdoors.
- Please read this manual instruction carefully before operation.
- Any other usage that is not listed in this manual instruction is forbidden.
- Recommended usage time of this product is 20 minutes/ session.
- Please check if any part of this products is damaged before use. If so, please do not use it and contact us for repair.
- Please do not use it if the cover is broken or damaged.
- Please do not use it if the shell or surface is damaged.
- Please do not drop anything into this product.
- Please do not fall asleep while using this product.
- Please do not use it if drunk or feeling unwell.
- Please do not use it within one hour after having meal.
- Please do not adjust to the maximum intensity at the beginning of the massage to avoid the risk of being injured.
- Please do not keep sharp objects in pockets when use.
- Keep the plug clean from needles, rubbish or water.
- If the power cord is damaged, do not attempt to change on your own. Please contact us for repair for safety reasons.



### 2. Environment for usage

- Please do not use it in a moist environment such as bathrooms.
- Please do not use it when there is a drastic change in temperature.
- Please do not use it under heavy-dusty or caustic environment.
- Please do not use it where there is not enough space or not good ventilation.
- Please switch off the power and unplug the machine after use.
- Please do not use it when surrounding temperature is up to 40°C.
- Please do not use it when surrounding temperature is below 5°C. The lubricant can thicken quickly and increase the friction between parts.
- Turn off and pull out the plug during thunderstorms.
- Place the product on a flat surface.
- Do not use outdoors.
- Do not place it near fire or directly under the sun.
- When moved from a low-temperature storage to a warm environment, the moisture inside the machine may affect, or damage, its functionality. Please rest the product within room temperature for 1 hour before use.



### 3. People who have the below symptoms, please consult a doctor before use

- People who are suffering from osteoporosis.
- People who are suffering from heart disease or carrying electronic medical equipment such as heart pacemaker.
- People who have a fever.
- Pregnant women or women that are in menses period.
- People who got injured or skin diseases.
- Children under 14-year old or people mentally unbalanced without being supervised are not allowed to use this machine.
- People who are told by doctor to have rest or people who feel unwell.
- People whose bodies are wet are forbidden to use this product.



## Safety Precautions

- The appliance has heating function. People who are insensitive to heat must be careful when using the appliance.
- People who are receiving medical treatment, especially those who are physically unfit.
- People who have open wounds.
- People who aim for recovery.



## 4. Safety

- Please do not place heavy objects on this product.
- Do not use other therapy devices while using this product.
- This product is not to be used for medical treatment purpose.
- Only one person at a time.
- Recommended usage time of this product is 20 minutes/ session.
- It is possible to place a mat underneath the machine to avoid damage to the floor.
- Check if the voltage is suitable to the specification of this product.
- Do not use wet hands to pull the plug.
- Do not allow water to get into the product, for the avoidance of electric shock or damage to this product.
- Do not pull or drag the power cord when unplugging. Please be gentle.
- Do not damage the wires or change the circuit of this product.
- Do not use wet cloths to clean the electric parts such as the switch and plug.
- Be away from this product during power cut to prevent injury in case power resumes suddenly.
- Stop using this product if there appears any irregularity, and please contact us immediately.
- Stop using this product if it causes discomfort, and please consult a doctor.
- This appliance is not intended for people (including children) with reduced physical, sensory or mental capabilities, or a lack of experience and knowledge, unless they are under supervision or instruction concerning the use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- If the power cord is damaged, it must be replaced by us in order to avoid a hazard.
- Do not ride on the handle to avoid injury or malfunction.
- Keep the power cord away from hot or moving objects.
- Please cut off power and pull out plug before moving.
- If the product is found damaged, broken, suffering from power leakage, or with exposed parts, please stop using it.



## 5. Maintenance

- This product should be maintained by us, users are prohibited to disassemble or maintain by themselves.
- Please always turn off the main power after use.
- Do not use this product if the socket is loose.
- If this product is to be left idle for long, please tie up the power cords and keep it in dry and dustless environment.
- Do not keep this product at a high temperature or near fire. Do not keep under direct sunlight.
- Please clean this product with dry cloths. Do not use thinner, benzene or alcohol.
- The mechanical components of this product are specially designed and made, no special maintenance required.
- Do not point sharp objects at this product.
- Do not roll or pull this product above uneven ground, it should be lifted before moving.
- Please use it intermittently; do not keep it continuously running for long.




## 6. Solution of ordinary malfunctions


- It is normal if the motor makes noises during operation.
- If the remote control can't work normally, please check the plug and socket if they are connected, and make sure the switch is turned on.
- If the rated working time is over, the power will automatically go off. If this product works continuously for too long, the temperature control will automatically turn off the machine, if so, please allow a 30-minute break before resuming.

 **Safety Warning** **Warning**

When the calf rest is ascending during use, do not apply pressure on it. This is to prevent the massage chair from falling forward and causing damage or injury.

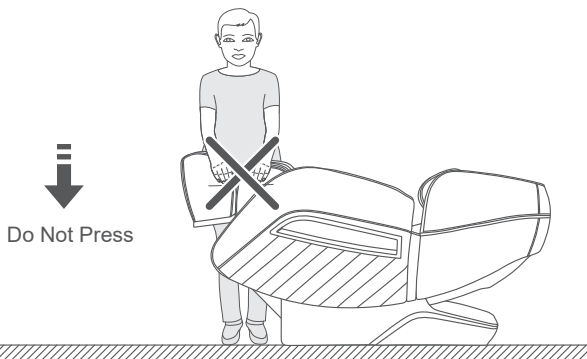
 **Warning**

Prior to sitting on the massage chair, please make sure the mechanical massage hands are not located under the seat. This is to avoid the discomfort caused to the hip area when sitting and the possible damage to the massage hands.

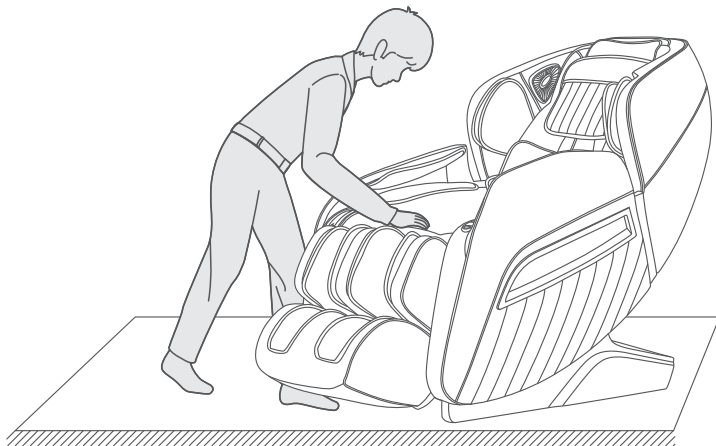


 Safety Warning Warning


In “standby” mode, or when the massage chair has been connected to power, do not allow children to drag/ pull the power cord. Damage to the power plug (or its parts) due to forceful pulling/ dragging will cause damage to the product or possible electric shock. It is advised to unplug the massage chair and tie up the power cord when not in use.

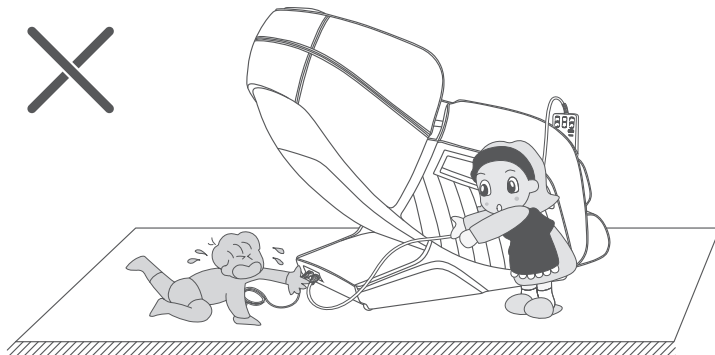 Warning


When switch to “Zero Gravity” mode, please ensure that there are no kids or pets under the calf rest and back area of the massage chair, or else there will be damage to the products as well as personal injury due to the downward movement of the chair.

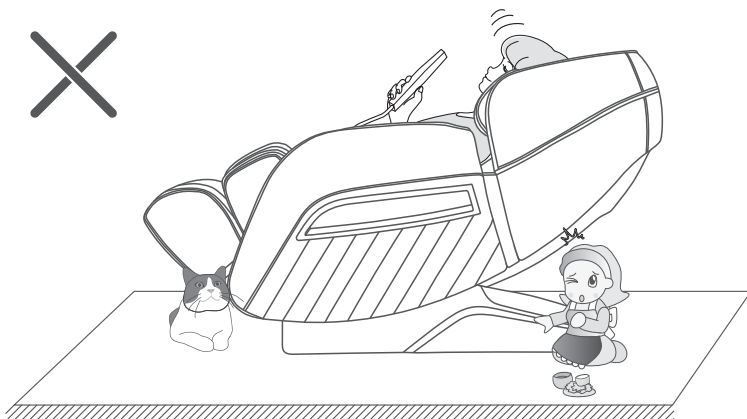

 Name and Function of Components

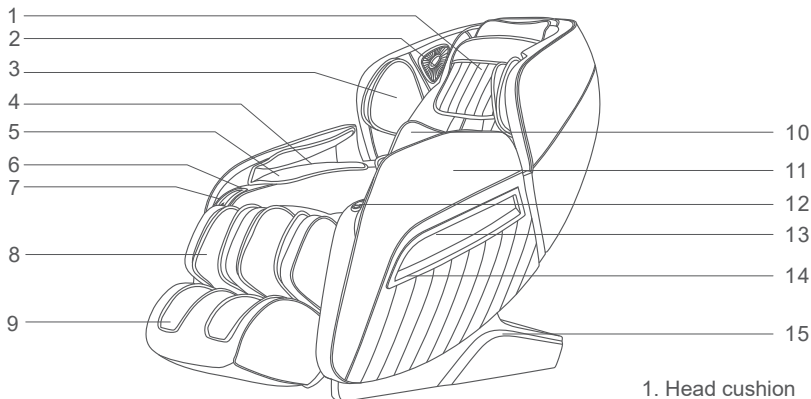

- 1. Head cushion
- 2. 3D digital audio
- 3. Upper arm airbag component
- 4. Arm airbag component
- 5. Salient massage hand with magnetic therapy
- 6. Remote controller or mobile phone placement slot
- 7. USB charging port
- 8. Legrest
- 9. Footrest
- 10. Waist air bag
- 11. Armrest
- 12. Armrest shortcut key
- 13. Embedded atmosphere lamp
- 14. Armrest decoration
- 15. Side cover

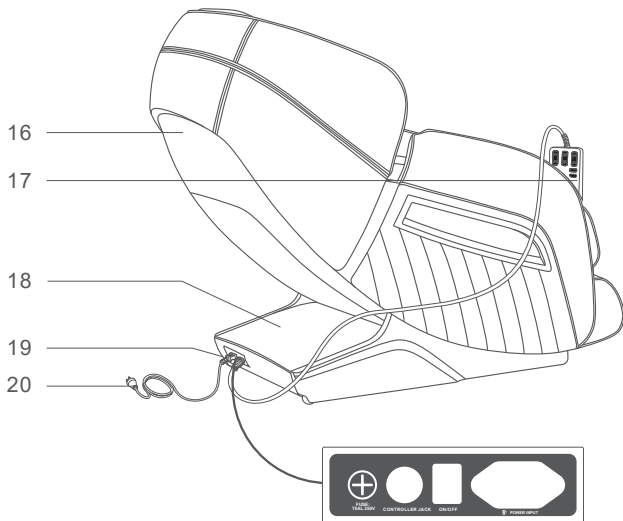






## Name and Function of Components



### Function:

- This product is designed with a set of four- wheels driven mechanical massage hands that can move up and down while stretching back and forth.
- There are six simulative massage techniques: Shiatsu, Kneading, Tapping, Knocking, kneading and Tapping, Kneading and Shiatsu.
- Set with the shoulder automatic detection and location.
- Set with auto massage function: 16 auto massage function included “Neck and Shoulder” , “Back and Waist” , “Full-body Stretch” , “Relax” , “Comfort” , “Mobile Zombie” , “Bottom Toning” , “Thai Massage” , “Detox” , “Quick Refresh” , “Humpback Fix” , “Circulation Boost” , “Total Relax” , “Sport Recovery” , “Spine Massage” and “Office Regimen” .
- Manual massage for the upper body with 3 options: overall, partial, and fixed. Each massager method with 5 levels of speed adjustable. In the state of partial and fixed massage, massage hand can adjust the position, adjust the massage function shiatsu, kneading & tapping, tapping, kneading, knocking. Also, the width between the two kneading balls is adjustable with three levels: wide, medium, and narrow.
- Air pressure massage function with 3 modes: full body air pressure, arms, back and leg air pressure, and three intensity optional.
- Calf kneading function: kneading in opposite direction and air pressure massage function can relax stiff calf muscle from longtime standing.
- Leg designed with roller massage function
- The footrest can be extended to suit different leg lengths.
- The back-heating function uses carbon fiber as an infrared heating source.
- Backrest frame down slides used technology to deviate from the wall by keeping zero space, reach the maximum space saving.
- SL-shaped ultra-long rail perfectly matches the human body shape, ensuring a close-fitting massage from neck to thigh.
- The music player system: Bluetooth digital speakers are built into left and right upper arms. The Bluetooth function of the smart device can be used to play music wireless.
- Both side of hand airbags are equipped salient massage bands with magnetic therapy function.
- The quick control panel with the power button, auto massage button, linkage adjustment button and zero gravity button on the left armrest.
- USB charging port on the right armrest.
- LED atmosphere lights are installed on the sides of the armrests.
- Equipped with an LCD screen remote control.
- Automatic reset function: the chair and the massage hands automatically resume to starting position when turned off.

 Name and Function of Components

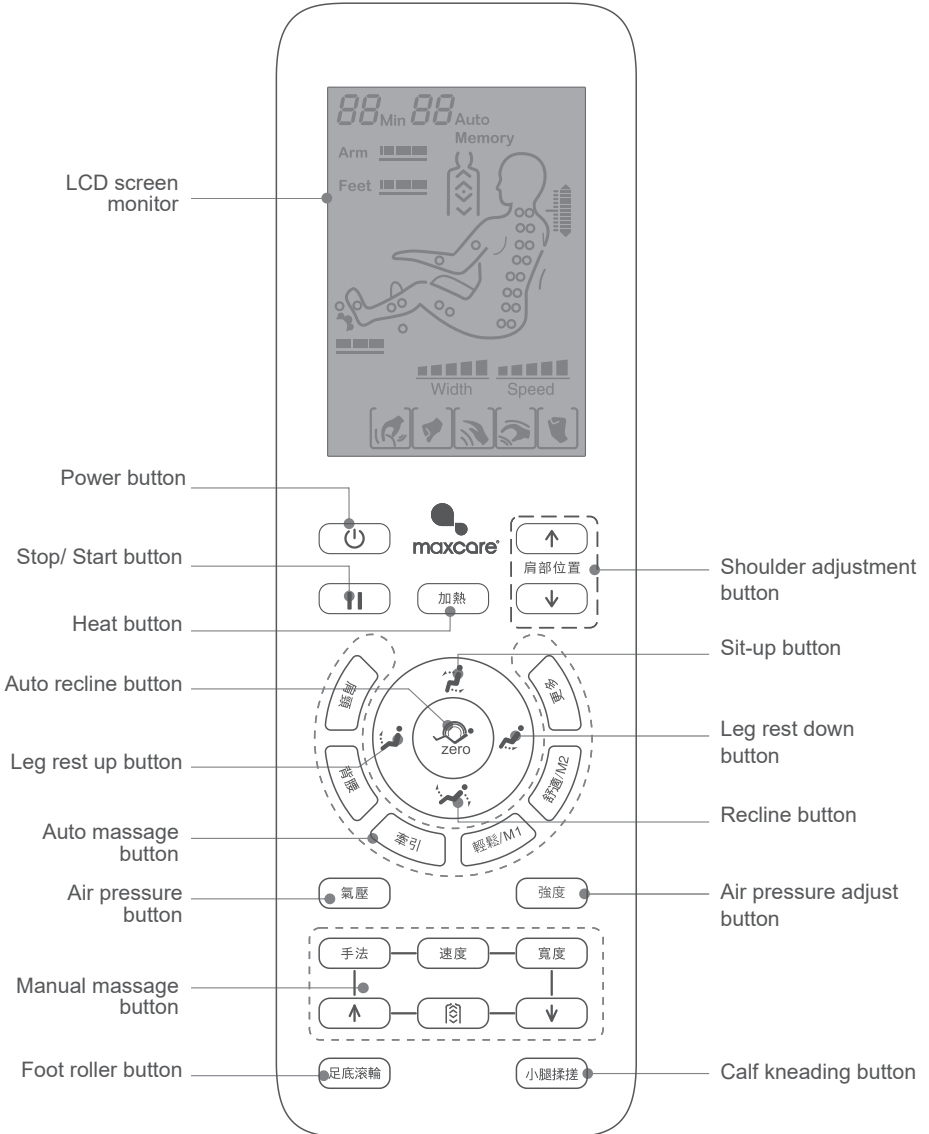

Note: The specific display refers to the actual object of this product!

## How to Use

### A. Connect to power



Diagram of plugging in








Diagram of the switch position

### B. Massage Function Operation

1. Press the power button and start massage.
2. Auto adjust the position.
3. Start the program of automatic shoulder positioning and massage acupressure point detection. Please wait for the completion of testing.
4. If there is a mismatch between the massage balls and the actual shoulder position, you can press the up and down button to adjust them to a suitable position. There are 10 levels for you to adjust the shoulder position. If no adjustment on shoulder position is made within 10 seconds, the massage program will start automatically.

Noted: In the startup progress, you can select the auto function. If you select the same auto function repeatedly, you can skip the startup process and directly enter the massage mode.



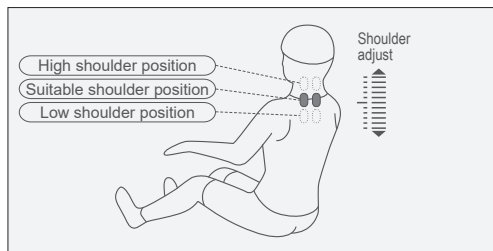



Button	Description	Display
 Shoulder adjust (Up)	Move the massage balls upward to a suitable position	
 Shoulder adjust (Down)	Move the massage balls downward to a suitable position	




During automatic shoulder positioning and massage acupressure point detection processing (The shoulder position adjustment icon will show on the monitor, and when hear the beep sound from the remote controller, press UP/DOWN button to adjust the shoulder position), you can choose 16 auto massage functions (Neck and Shoulder, Back and Waist, Full-body Stretch, Relax, Comfort, Mobile Zombie, Bottom Toning, Thai Massage, Detox, Quick Refresh, Humpback Fix, Circulation Boost, Total Relax, Sport Recovery, Spine Massage and Office Regimen). (You can skip the Step 4.)

 How to Use

Auto massage



Function	Button	Description	Display	
Auto Massage		Neck and Shoulder: Massage neck and shoulder area with the special designed program, to effectively relieve shoulder pain and promote blood circulation (to brain).	F1 Auto	
		Back and Waist: Focus on hip and thigh area for a shaping effect, help burning excessive body fats.	F2 Auto	
		Full-body Stretch: During the footrest and back are adjusted to the highest position, the massage ball will knead at the waist.	F3 Auto	
		Relax: Wake body muscle and other mechanism to start the day fresh after a night of sleep.	F4 Auto	
		Comfort: Through massaging the regimen acupuncture points to nurse and balance the body, improving sleep quality.	F5 Auto	
			Mobile Zombie: Massage certain acupuncture points to revive neck and shoulder strength. Heating function turn on automatically to promote blood circulation.	F6 Auto
			Bottom Toning: Focus on hip and thigh area for a shaping effect, help burning excessive body fats. Heating function turn on automatically to promote blood circulation.	F7 Auto
			Thai Massage: Traditional Thai massage consisting of whole-body stretching. Comprehensively erase body tiredness.	F8 Auto
Detox: Mainly tuina, kneading and tapping massage the waist and hip area.			A1 Auto	

 How to Use

Auto Massage		Quick Refresh: Mainly acupressure and tuina, medium speed of swing massage.	A2 Auto
		Humpback Fix: Mainly targeting at waist area to tackle waist pain and improve waist health	A3 Auto
		Circulation Boost: Improve blood circulation, reenergize body and boost the immune system.	A4 Auto
		Total Relax: Full body relax and ease tiredness.	A5 Auto
		Sport Recovery: Fast recovery from soreness after workout.	A6 Auto
		Spine Massage: Massage every section of the spinal cord to ensure a health spine. Help with disc herniation and humpback.	A7 Auto
		Office Regimen: Especially designed for office workers who tend to sit for a long time. Keep the muscle active by massaging and enhance body health.	A8 Auto


















**Memory mode**

Press “輕鬆/M1” or “舒適/M2” for 3 second will enter to memory mode M1/M2. It will revert to the previous memory massage program.






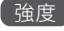


Function	Button	Description	Display
Memory massag		Memory mode 1	Memory
		Memory mode 2	Memory

 How to Use



Manual massage

Function	Button	Adjust Massage Ball	Display
Technique		Shiatsu: Speed and width	
		Kneading: Speed	
		Tapping: Speed and width	
		Kneading and tapping: Speed	
		Knocking: Speed and width	
		Kneading and Shiatsu : Speed	
Speed		5 Grades speed adjustable	 Speed
Width		3 Grades widths adjustable	 Width
Back stretch		Fixed position (fixed point) massage	
		Small area (partial area) massage back and forth	
		Overall massage back and forth	
		Massage point adjust (in fixed point, partial modes you can adjust the massage position)	Message hand move up
			Message hand move down

Air pressure function massage

Function	Button	Description	Display
Air pressure mode		Full body: Arm, foot and waist will start air pressure massage.	Arm  Feet 
		Waist and Arms : Mainly start air pressure massage	Arm 
		Legs: Mainly start air pressure massage on legs and feet.	Feet 
Intensity		With 3 grades adjustable.	Arm  Feet 




**How to Use**
**Foot roller massage**

Function	Button	Description	Display
Foot roller		Press the foot roller button to start or stop the roller massage with 3 intensity level to adjust.	



**Calf knead massage**

Function	Button	Description	Display
Calf knead		Press the calf knead button to start or stop the calf knead massage.	






**Heat massage**


Function	Button	Description	Display
Heat		Press the heat button to start or stop the heat therapy, heating within 3 minutes.	

**Pause/start massage button**

Function	Button	Description	Display
Pause/start		Press the pause/start button during massage progress, all massage function will be stopped immediately (Stop over 20 minutes will shut down the massage chair). Press the pause/start button again to continue the previous massage actions.	

**Lying angle adjustment**



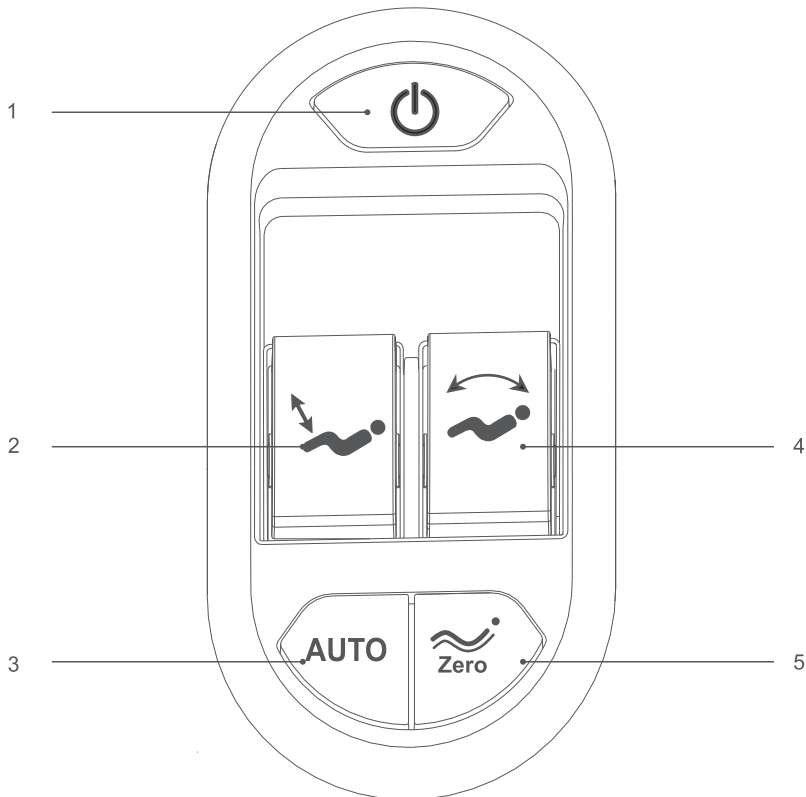

Function	Button	Description
Sit-up button		Press and hold the sit-up button, the back cover will slowly ascend while the lag rest will slowly descend. If the massage chair is completely sit-up, it will have "di...di..." sound and stop the motion. Release the button to stop the motion.
Recline button		Press and hold the sit-up button, the back cover will slowly descend while the lag rest will slowly ascend. If the massage chair is completely laid down, it will have "di...di..." sound and stop the motion. Release the button to stop the motion.
Leg rest up button		Press and hold the leg rest up button to raise the leg rest, it will have "di...di..." sound and stop the motion. Release the button to stop the motion.
Leg rest down button		Press and hold the leg rest up button to lower the leg rest, it will have "di...di..." sound and stop the motion. Release the button to stop the motion.
Zero-gravity button		Press the zero-gravity button to change the recline mode, 3 modes for you to switch.

 How to Use C. Quick control panel

1. Power button: To start the massage chair.
2. Leg rest adjustment button: Adjust the leg rest raise or lower.
3. Auto button: To start the auto massage function.
4. Sit-up/Recline button: Adjust the back rest and leg rest raise or lower at the same time.
5. Zero gravity button: Adjust the angle of zero-gravity.




 How to Use

Button	Description
	Power button: Press the power button to start or shut down all the massage functions.
	Sit-up button: Press and hold the sit-up button to synchronically lift the back and lower the lag rest. Release the button to stop the motion.
	Recline button: Press and hold the recline button to synchronically lower the back and lift the lag rest. Release the button to stop the motion.
	Zero-gravity button; Press the zero-gravity button to set the chair to the zero-gravity mode. There are 3 zero-gravity modes in total, press to switch between the 3 modes.
	Leg rest up button: Press and hold the leg rest up button to raise the leg rest. Release the button to stop the motion.
	Leg rest down button: Press and hold the leg rest up button to lower the leg rest. Release the button to stop the motion.
AUTO	Auto-mode button: Press to switch between auto-massage programs.

**Right armrest**

Remote controller  
or mobile phone  
holder

USB charging port

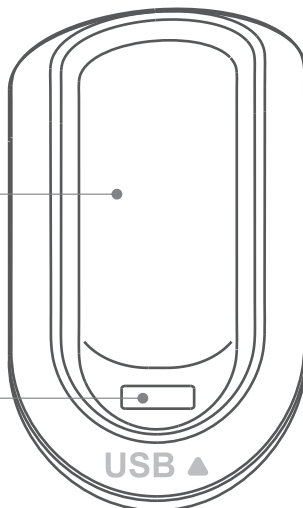

 **How to Use**

 **D. Operating the Bluetooth Speakers**

1. Select "Setting" on your smartphones/tablets -> Choose "Bluetooth" -> Turn on "Bluetooth" -> Searching new Bluetooth devices.
2. The Bluetooth name of the massage chair will be in the format of "MCM-XXXX". Select and connect.
3. Open music player on your personal devices and enjoy the music via the Bluetooth speakers on the massage chair.

 **E. Turn off the power to stop the massage**


- 1, Press the power button during massage progress, all massage function will be stopped immediately, the backrest and foot part will be reset. Or the massage time finished, all massage function will be stopped immediately, backrest and foot part will not reset.
- 2, Memory function: When the screen 'Memory" flashes, press M1/M2 to save the certain massage program.
- 3, Cut off the power of the whole machine.

Figure (Switching power off)




 **F. Leg Rest Installation**

1. After connecting the leg rest and the massage chair with the wires and the air hose, remove the cable ties and insert the connecting part into the seat (Figure 1).
2. Place the round tubes on the left and right sides of the leg rest onto the leg rest support (Figure 2).

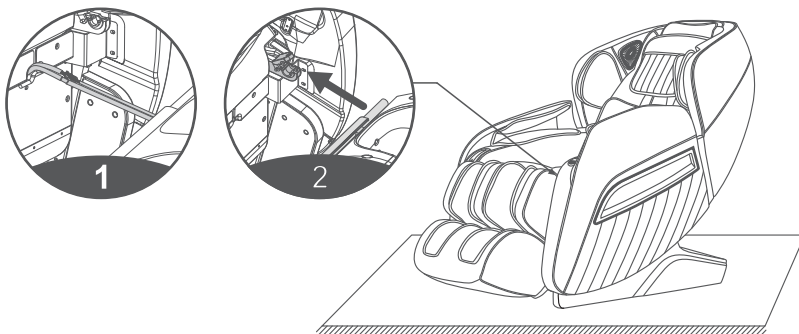


 How to Use

3. Next, turn the leg rest fixed cover to cover the round tubes on both sides of the leg rest (Figure 3).
4. Fasten the leg rest to the massage chair with two M4 screws (one on each side). Check whether there is any abnormality during turning up and down, or any tube, cable or wire is folded. If everything is OK, the leg rest is installed (Figure 4).

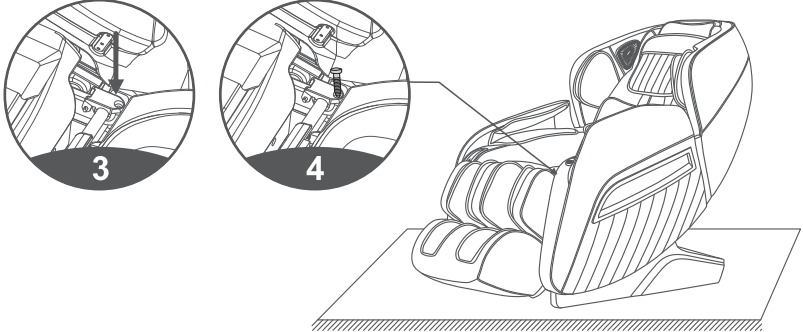

 **G. Headrest & back cushion installation and use**

Using the headrest can lessen the massage intensity borne by the neck and shoulders. You may choose to use the headrest according to your own needs(recommended). Back cushion is attached to the back through zipper (1), headrest is attached to the back cushion through zipper(2).

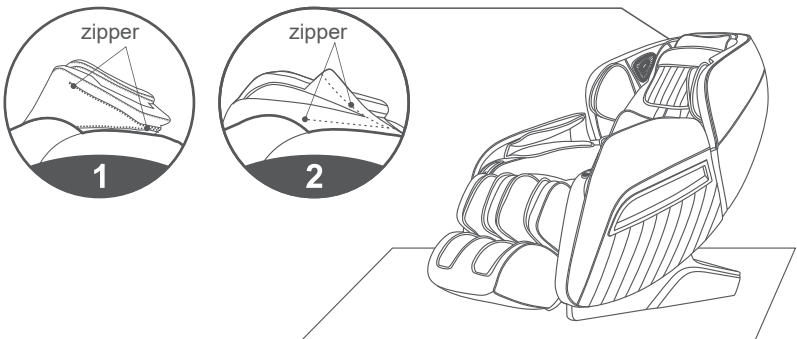




## How to Use



### H. Floor protection

The floor may be damaged due to the heavy pressure from the massage chair. It is advised to place a floor mat or other materials as a buffer to prevent the damage.

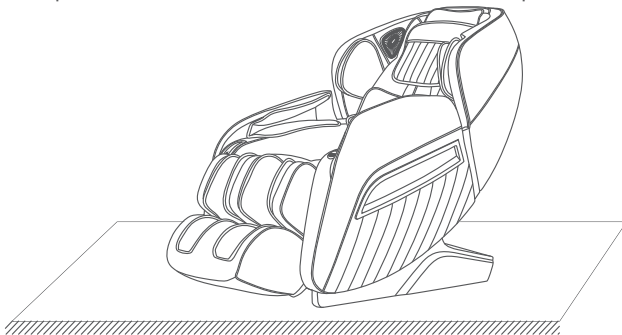


**CAUTION;** Do not move the chair with its wheels on a bumpy ground or in a narrow space. Two people are required to lift the chair. Hold the leg rest cover and the dent in the back cover while lifting the chair (please do not lift from the armrest).



### I. Method of movement

Please make sure unplugged the power. Hold a groove in the back plastic cover to tilt the backrest backward to a certain degree (with the gravity center resting on the wheels), push the chair forward or backward with your hands and finally resume the chair to the normal position in a slow and gentle manner. (Note: During movement you must turn off the power switch first and then unplug the power line and hand controller connecting line.)

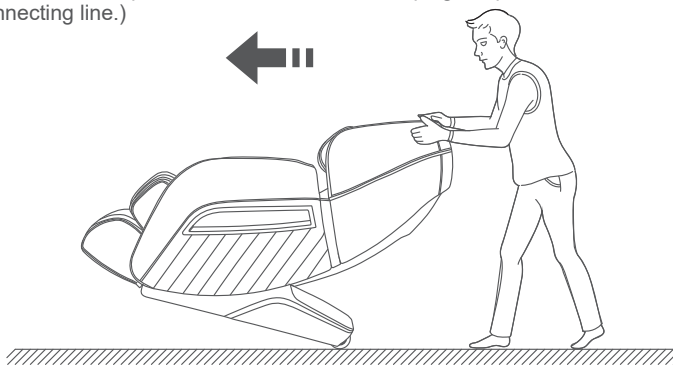

**CAUTION**

The massage chair with its wheel are suitable for moving in a short distance. Please use other tools or method to move the massage chair when you need to move in long distance.

## Cleaning and Maintenance

### Cleaning

- Before cleaning, please make sure the switch has been turned off and the power plug has been disconnected from the socket.
- Use soft cloths to clean the surface of product and control panel.



#### CAUTION

- Do not pull the plug from the socket with wet hands.
- Please do not rinse the product. Avoid water/ liquids from getting into the product.
- Do not use corrosive cleaning agents like benzene or thinner to clean the product, otherwise, the color of product will fade, and the internal parts may be corrupted or broken.
- Do not disassemble the product when it is humid. Simply let it dry.

### Maintenance

- Make sure that the plug has been disconnected from the socket before maintenance.
- Keep the product clean and away from high-temperature/ humid areas, or any environment with volatile gases.
- If this product is to be left idle for long, properly cover it with a dust cover (self-purchase).



#### CAUTION

- Please keep the product away from direct sunlight, or high temperature, to avoid fading or product damage.

### Kindly reminding

- If the product is used through the year, please check conditions as below:
  - \* Burnt smell.
  - \* The power cords do not work every time.
  - \* The power cords glow.
  - \* The power cords damage.
  - \* Other abnormalities.




#### CAUTION

- If above mentioned happens, please stop using the product.
- Switch off the power supply, pull out the plug, and contact us.
- Do not disassembly the product by yourself.

## Maintenance and Cleaning of PU Leather

PU leather is made by the coating and drying of polyurethane. Compared with genuine leather, PU leather is more resistant to tear, heat and sunlight. Depending on its nature and chemical substances, however, it should be kept away from many dusty, high/low-temperature, humid environment, under direct sunlight or places with corrosive substance. Otherwise, the leather could tear, quickening aging and hydrolysis.

 **Cleaning**

1. Please do not use organic reagent or grease solution, especially strong solvent, e.g. alcohol, turpentine and thinner, to clean the surface of the PU leather. Otherwise it will corrode the surface.
2. Please do not use water or washing powder to clean the surface of the PU leather. Otherwise there will be cracks.
3. When cleaning the surface of the PU leather, choose a cleaning agent that has a pH value of between 5~7, no dentifrice, and is harmless to the environment.
4. For dirty PU surface, apply egg whites to the surface using a piece of soft cloth. Avoid using a brush for cleaning.

 **Maintenance**

1. Keep a good ventilation indoor, overly dry or humid environment can accelerate the aging of leather
2. Please keep the product away from direct sunlight or air vent, otherwise the leather will be hardened. Direct sunlight will also cause fading.
3. People are generally sweatier in summer. The holes on the surface of the leather will absorb sweat. While heat and high humidity prompt the chemical reaction between certain substances in sweat and the leather, and hence, causes smell. Please clean the surface more often with a piece of lightly wet cloth.
4. Please do not use alkaline cleaning agents on the leather. It will adversely affect the elasticity of the leather and lead to cracks as a result.

 **Technical Specifications**

Product Name	: Max Gorgeous
Product Model	: MAX-77A305
Rated Voltage	: 220-240V~ 50/60Hz
Rated Power	: 150W
Rated time	: 20 minutes
Noise	: ≤ 55dB
Product Gross Weight	: 110kg
Product Net Weight	: 90kg
Product Size (Erected)	: 154cm x 75.5cm x 115cm
Product Size (Stretched out)	: 193cm x 75.5cm x 87cm
Safety structure	: Class I
Compliance	: GB4706.1-2005 GB4706.10-2008



商標持有：美斯凱實業有限公司  
公司地址：香港北角健康東街39號柯達大廈二期1601室

Trademark holder: Maxcare Industrial Limited  
Address: Rm 1601 Kodak House II, 39 Healthy Street East,  
North Point, Hong Kong

客戶服務熱線 Customer Service Hotline :  
香港 Hong Kong - (852) 2950-9383  
中國 China - (86) 400 882-1883

[www.maxcare.net](http://www.maxcare.net)  
[www.maxcareonlineshop.com](http://www.maxcareonlineshop.com)