


# 至型椅

PRIME CHAIR

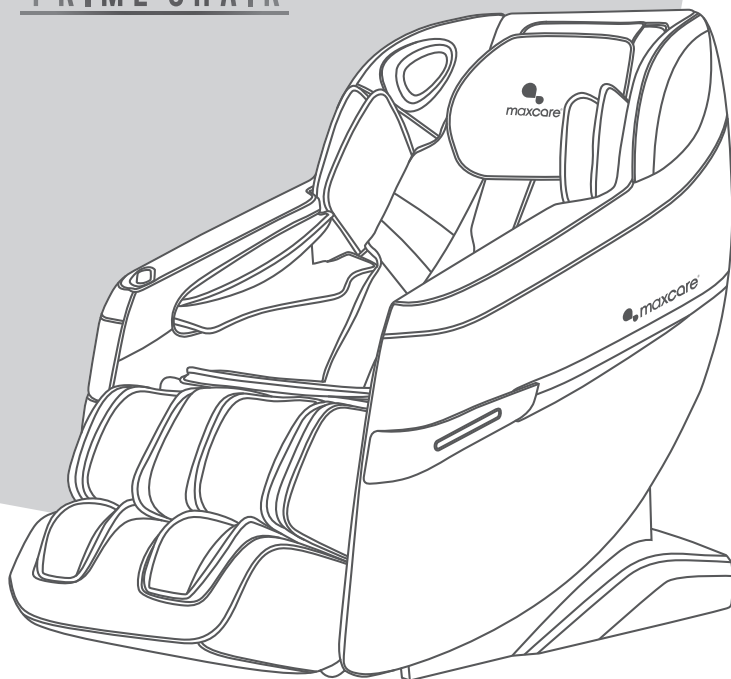

## User Manual

MAX-78A202S (Silver)

MAX-78A202B (Black)





## COMPANY PROFILE

After two decades of experience in the health industry, **maxcare** founder utilized his knowledge and expertise in massage therapy from an Asian-Pacific perspective to establish our brand - **maxcare**. Our massage therapy business was expanded to Hong Kong in 2012 and is conferred 2017 Hong Kong Top Brand Award. Based in Hong Kong, we are specializing in design & innovation, manufacturing, and retailing of massage and healthcare products.

Believing in the value of 'Technology improves our quality of life' and 'Caring for your family', **maxcare** has been innovating highly qualified products with international patents. Besides, **maxcare** products fulfill various international safety standards, namely EU CE Mark, US UL/ETL Mark, Germany GS Mark, TUV certification, and China CCC Mark.

Motivated by our value, **maxcare** founder has been leading us to be one of the most competitive brands in Hong Kong health industry. **maxcare** applies the contemporary fashion design – neat and modern – on the mechanical engineering of massage chairs. To deliver our value and offer incredible massage experience and healthcare solution to the public, **maxcare** devotes to the innovation, design, manufacturing the top-notch family massage products. By integrating the company philosophy –Spirit of Healthy Life - into our products, **maxcare** brings health to everyone.

# 至型椅

## PRIME CHAIR

Thank you for purchasing **maxcare**'s products. Before using the product, please read this manual thoroughly, particularly the Safety Precautions section, for the correct use of the product.

This manual should be well kept for easy reference. It enables you to perform a safe and efficient operation.

### CONTENT

---

Safety Precautions .....	P.01 - P.06
Name and Function of Components .....	P.07 - P.08
Control Interface .....	P.09
How to Use .....	P.10 - P.29
Cleaning and Maintenance .....	P.30
FAQs and Solutions .....	P.31
Technical Specifications .....	P.32



## Safety Precautions



### 1. Important safety warning

- Please read this manual instruction carefully before operation.
- Any other usage that is not listed in this manual instruction is forbidden.
- Keep children and pets away from the movable parts of this product to avoid accidents caused by the up-and down-ward movement of the calf rest and back cover.
- When in use, sliding or transporting, please ensure that there are no people or pets nearby (the back, bottom, front and sides of the product), and that no foreign objects or pets inside the product, to avoid accidents.
- Please use well-grounding power supply suited to this product.
- Please pull out the plug after use or before cleaning to avoid injury or damage to the product.
- Do not use unauthorized accessories.
- Do not use this product outdoors.
- People whose bodies are wet are forbidden to use this product.
- Recommended usage time of this product is 20 minutes/ session.
- Please check if any part of this products is damaged before use. If so, please do not use it and contact us for repair.
- Please do not use it if the cover is broken or damaged.
- Please do not use it if the shell or surface is damaged.
- Please do not drop anything into this product.
- Please do not fall asleep while using this product.
- Please do not use it if drunk or feeling unwell.
- Please do not use it within one hour after having meal.
- Please do not adjust to the maximum intensity at the beginning of the massage to avoid possible injury.
- Please do not keep sharp objects in pockets when use.
- Keep the plug clean from needles, rubbish or water.
- If the power cord is damaged, do not attempt to change on your own. Please contact us for repair for safety reasons.



### 2. Environment for usage

- Please do not use it in a moist environment such as bathrooms.
- Please do not use it when there is a drastic change in temperature.
- Please do not use it under heavy-dusty or caustic environment.
- Please do not use it where there is not enough space or not good ventilation.
- Please switch off the power and unplug the machine after use.
- Please do not use it when surrounding temperature is up to 40°C.
- Please do not use it when surrounding temperature is below 5°C. The lubricant can thicken quickly and increase the friction between parts.
- Turn off and pull out the plug during thunderstorms.
- Place the product on a flat surface.
- Do not use outdoors.
- Do not place it near fire or directly under the sun.
- When moved from a low-temperature storage to a warm environment, the moisture inside the machine may affect, or damage, its functionality. Please rest the product within room temperature for 1 hour before use.



### 3. People who have the below symptoms, please consult a doctor before use

- People who are suffering from osteoporosis.
- People who have cancers, acute illnesses, heart diseases, or severe high blood pressure, etc. are not suitable to use this product.
- People who have/ had back pain, or legs, waist, neck numbness/ injuries (disc herniation, spondylosis and cervical protrusion, etc. patients) are not suitable to use this product.
- People who have rheumatoid arthritis, rheumatism, gout are not suitable to use this product.
- People who have respiratory difficulties are not suitable to use this product.
- People who have blood circulation problems such as thrombosis, severe aneurysm, or acute venous tumor, or infectious skin diseases are not suitable to use this product.
- People who have perception disorders due to diabetes are not suitable to use this product.
- People who have spinal abnormalities or spine curvature disorders are not suitable to use this product.
- People whose health condition is poor are not suitable to use this product.
- People who are suffering from heart disease or carrying electronic medical equipment such as heart pacemaker.
- People who have a fever.



## Safety Precautions

- Pregnant women or women that are in menses period.
- People who got injured or skin diseases.
- Children under 14-year old or people mentally unbalanced without being supervised are not allowed to use this machine.
- People who are told by doctor to have rest or people who feel unwell.
- The appliance has heating function. People who are insensitive to heat must be careful when using the appliance.
- People who are receiving medical treatment, especially those who are physically unfit.
- People who have open wounds.
- People who aim for recovery.



## 4. Safety

- Please do not place heavy objects on this product.
- Do not use other therapy devices while using this product.
- This product is not to be used for medical treatment purpose.
- Only one person at a time.
- Recommended usage time of this product is 20 minutes/ session.
- It is possible to place a mat underneath the machine to avoid damage to the floor.
- For safety reason, maintain a minimum of 0.5m between the product and others when use.
- Check if the voltage is suitable to the specification of this product.
- Do not use wet hands to pull the plug.
- Do not allow water to get into the product, to avoid electric shock or damage to this product.
- Do not pull or drag the power cord when unplugging. Please be gentle.
- Do not damage the wires or change the circuit of this product.
- Do not use wet cloths to clean the electric parts such as the switch and plug.
- Be away from this product during power cut to prevent injury in case power resumes suddenly.
- Stop using this product if there appears any irregularity, and please contact us immediately.
- Stop using this product if it causes discomfort, and please consult a doctor.
- This appliance is not intended for people (including children) with reduced physical, sensory or mental capabilities, or a lack of experience and knowledge, unless they are under supervision or instruction concerning the use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- If the power cord is damaged, it must be replaced by us in order to avoid a hazard.
- Do not ride on the handle to avoid injury or malfunction.
- Keep the power cord away from hot or moving objects.
- Please cut off power and pull out plug before moving.
- If the product is found damaged, broken, suffering from power leakage, or with exposed parts, please stop using it.



## 5. Maintenance

- This product should be maintained by us, users are prohibited to disassemble or maintain by themselves.
- Please always turn off the main power after use.
- Do not use this product if the socket is loose.
- If this product is to be left idle for long, please tie up the power cords and keep it in dry and dustless environment.
- Do not keep this product at a high temperature or near fire. Do not keep under direct sunlight.
- Please clean this product with dry cloths. Do not use thinner, benzene or alcohol.
- The mechanical components of this product are specially designed and made, no special maintenance required.
- Do not point sharp objects at this product.
- Do not roll or pull this product above uneven ground, it should be lifted before moving.
- Please use it intermittently; do not keep it continuously running for long.



## 6. Solution of ordinary malfunctions

- It is normal if the motor makes noises during operation.
- If the remote control can't work normally, please check the plug and socket if they are connected, and make sure the switch is turned on.
- If the rated working time is over, the power will automatically go off. If this product works continuously for too long, the temperature control will automatically turn off the machine, if so, please allow a 30-minute break before resuming.



**CAUTION**

Please make sure all the power is cut off before moving.  
The power cord and all other cables should be disconnected.




## Safety Precautions



### Warning

When the calf rest is ascending during use, do not apply pressure on it. This is to prevent the massage chair from falling forward and causing damage or injury.

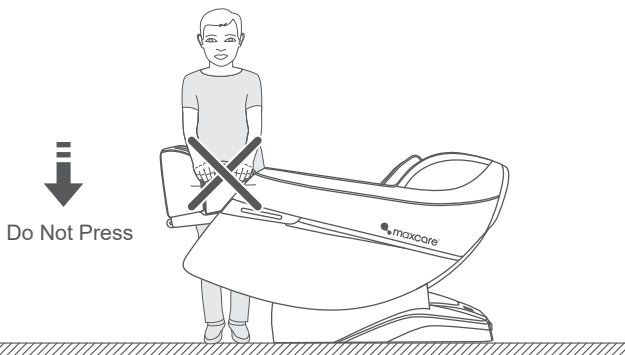

### Warning

Prior to sitting on the massage chair, please make sure the mechanical massage hands are not located under the seat. This is to avoid the discomfort caused to the hip area when sitting and the possible damage to the massage hands.






## Safety Precautions




### Warning

In “standby” mode, or when the massage chair has been connected to power, do not allow children to drag/ pull the power cord. Damage to the power plug (or its parts) due to forceful pulling/ dragging will cause damage to the product or possible electric shock. It is advised to unplug the massage chair and tie up the power cord when not in use.

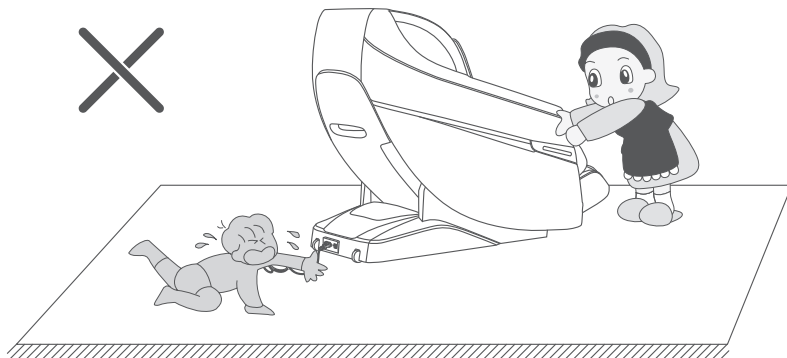

### Warning

When switch to “Zero Gravity” mode, please ensure that there are no kids or pets under the calf rest and back area of the massage chair, or else there will be damage to the products as well as personal injury due to the downward movement of the chair.

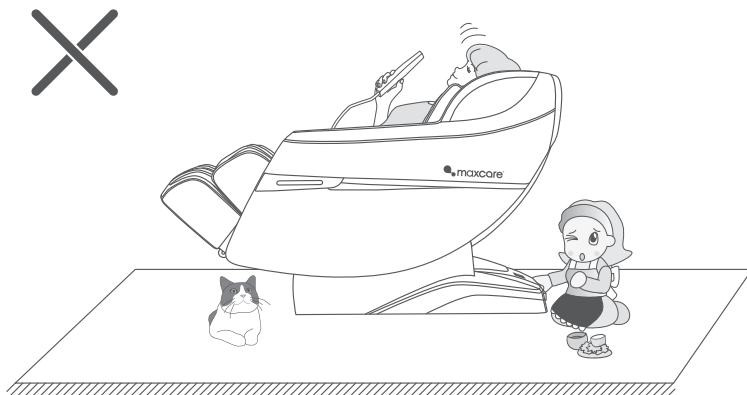




## Safety Precautions




### Warning

Take off your belongings before starting the massage chair. (Such as: Keys, hand watch, hard object, and so on. It might cause damage to the product or personal injury.

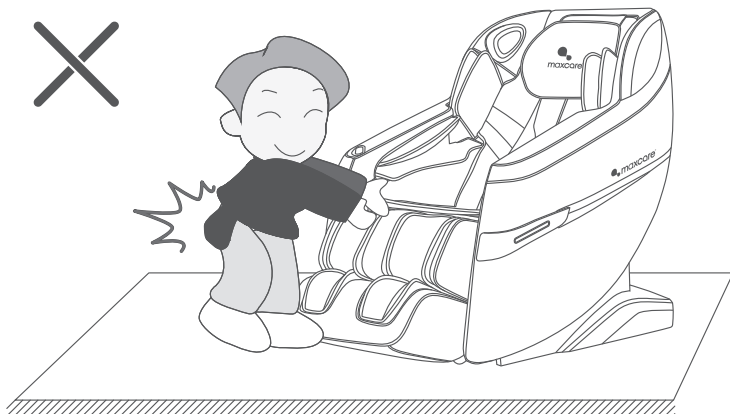

### Warning

Never unplug and switch off the main power when the massage chair is operating. It might cause damage to the product or personal injury.

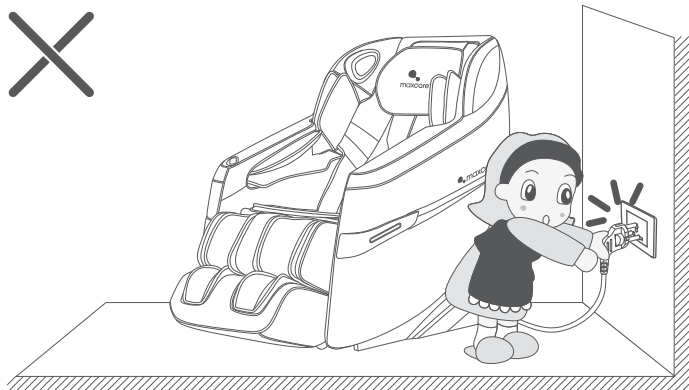




## Safety Precautions




### Warning

Try no to sleep when using the massage chair. It might cause personal injury.


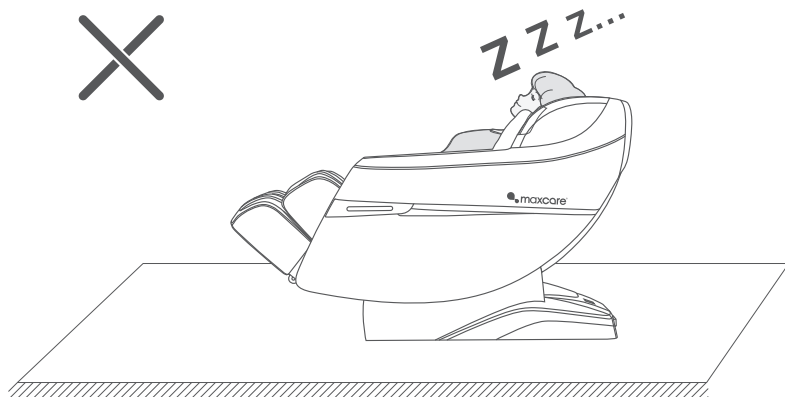


### Warning

Do not use it with two people at the same time, it will cause damage to the product or personal injury. kids are not allowed to play on the chair with toys, or climb on any parts of the chair, otherwise, it will cause damage to the product or personal injury.

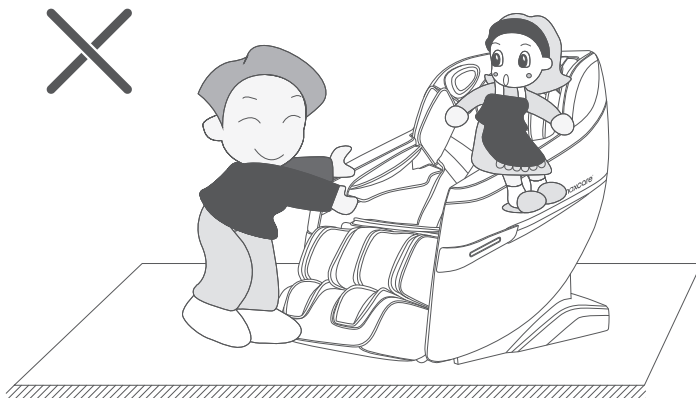

## Name and Function of Components

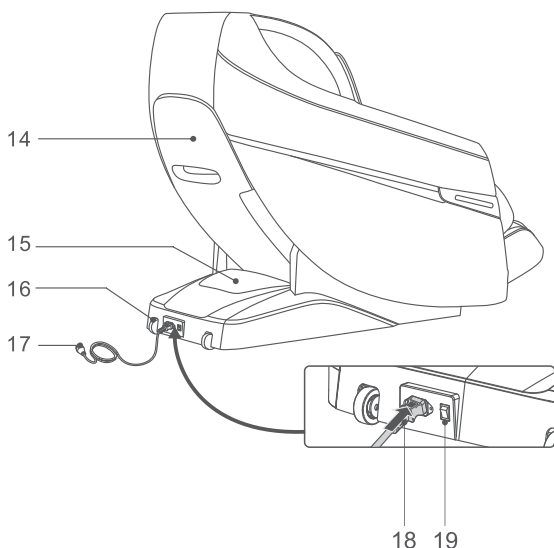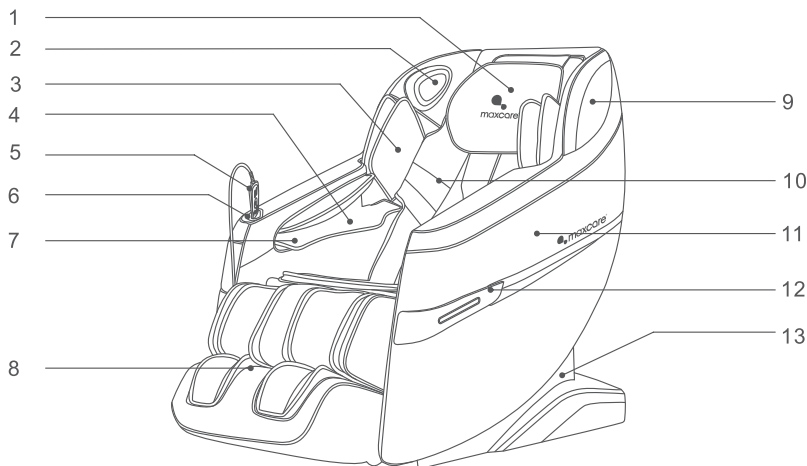

1. Headrest
2. Bluetooth Speaker
3. Upper Arm Airbags
4. Arm Airbags
5. Remote Handset Controller
6. Remote Handset Controller Holder
7. Rugged Massage Belt and Magnetotherapy
8. Leg Rest
9. Upper Armrest Decoration
10. Back Cushion
11. Armrest
12. Armrest Decoration
13. Side Cover
14. Back Cover
15. PCB Cover
16. Wheels
17. Power Cord and Plug
18. Power Jack
19. Power Switch




## Name and Function of Components



### Function:

- This product is equipped with a set of silent smart 3D massage heads which move up and down and front and back.
- Capable of automatically detecting and fine-tuning shoulder position; detecting body curve and shiatsu points to adjust accordingly the movement of the massage heads. This makes the massage experience more personal and scientific.
- 15 massage automatic settings: Pain Relief, Neck And Shoulder, Bottom Toning, Full-body Stretch, Fresh Morning, Humpback Fix, Office Regimen, Spine Massage, Noon Recharge, Relax, Quick Refresh, Comfort, Thai Massage, Sport Recovery and Circulation Boost.
- 3 memory programs: Memory 1, Memory 2, Memory 3.
- Manual Massage Function for 5 selected areas: Spot, Partial, Upper Back, Lower Back and Full Back.
- It is provided with manual massage function: including seven massage methods such as knead, tap, knead and tap, shiatsu, knock, and 3D (3D I, 3D II).
- Level adjustment function: 3 strength levels for 3D massage, 3 for air pressure, and 4 for duration.
- Upper body air massage covering palms (4 built-in airbags), shoulders ((4 built-in airbags) and arms (4 built-in airbags). 3 intensity levels available.
- Lower body air massage covering calves (12 built-in airbags). 3 intensity levels available.
- Forward-sliding design eliminates the large space requirement between the chair and the wall.
- SL-shaped ultra-long rail perfectly matches the human body shape, ensuring a close-fitting massage from neck to thigh.
- The airbags in palm area allows rugged massage and magnetotherapy.
- Back and calf heating function: source of infrared heating.
- Music player system: 3D digital speakers installed above each armrest. Connect to your smart device and play music via Bluetooth.
- Leg stretch massage program: By programming the movements of the leg rest and back of the massage chair, together with air pressure at calves and the massage heads at back, provide a comfortable leg stretch function.
- Leg rest height adjustment, synchronized seat and leg height adjustment, zero gravity angle adjustment.
- Roller massage function: roller massage and gua sha (scraping) for sole. 3 speed levels available.
- Flexible leg rest: The length of the leg rest is in accordance with the length of the legs. The built-in springs help with the adjustment.
- Automatic reset upon shutting down.
- TFT LCD display remote controller.

 Control Interface

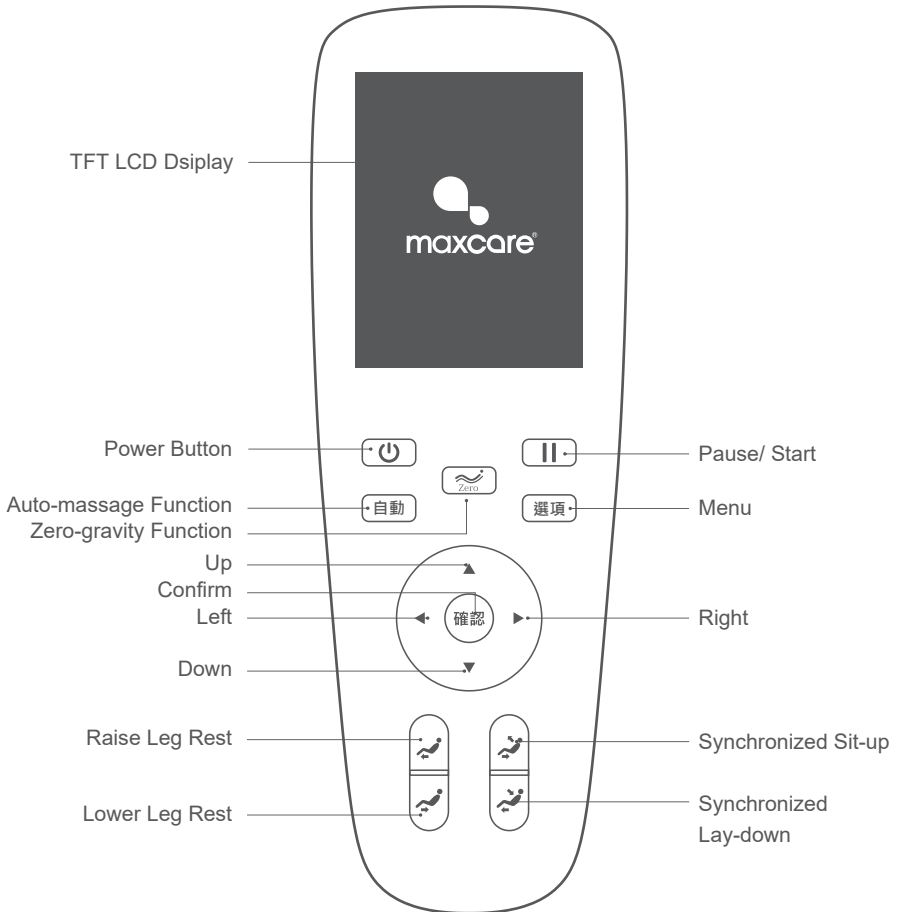


Note: The display is subject to the actual product.


## How to Use

### 1. Connect to power


#### 1 Connect power diagram

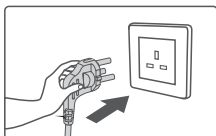

#### 2 Massage chair connect diagram

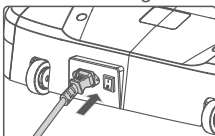

#### 3 On / off switch diagram

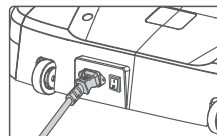

### 2. Begin Massage


1. Press the power button to activate the massage program. The back and leg rest will be set to the default position.
2. After the angle is set, the chair will automatically detect shoulder position and body shape. Wait for the detection process to be over; or, press 'OK' to skip the process and directly start massaging.



## How to Use


### 1. Shoulder Adjustment

When detection is over, there will be “tik tik” sound. Further finetune the position of the massage hands using the “up” and “down” buttons, if needed. There are 11 finetune adjustment levels for shoulder position. If there is no adjustment after 10 seconds, the massage will begin automatically. Or, press the “enter” button to directly enter the massage program. If the shoulder position is not detected, the massage hands will be set at the default height to begin massage.

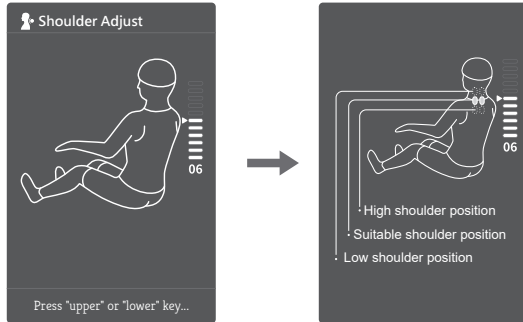

Button		Description
Up		Mechanical massage hands move upward.
Down		Mechanical massage hands move downward.


During massage, if shoulder adjustment is needed, go to shoulder adjustment interface in the status page by clicking the shoulder adjustment button. Use the “up” and “down” buttons to position the mechanical hands; click “left” to go back to the previous page.

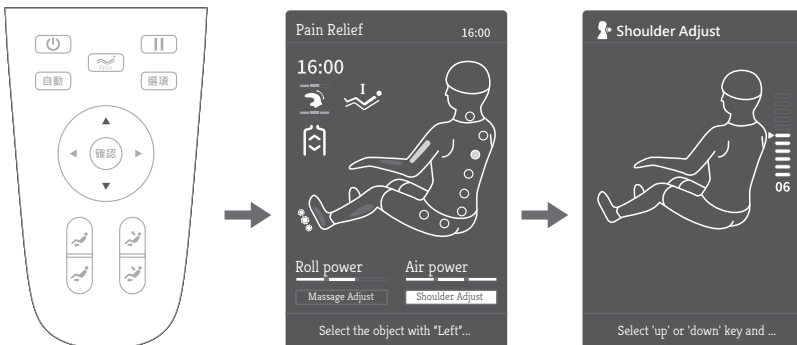

## How to Use

### 2. Massage Position Adjustment


- After entering the massage program, if it is necessary to adjust the massage position, press the "Menu" key on the manual controller to select the manual program to enter, select the manual program, and then press the "Left" key to return to the information interface (the information interface is changed to the manual program interface).
- From the manual program interface, if the massage position displays fixed points or local, you can directly press the "Up" or "Down" key to move the manipulator up and down.
- From the manual program interface, if the massage position is not fixed point or local, you need to select massage adjustment to enter, then select the massage part, select fixed point or local, then press the "Left" key to return to the information interface, and then press the "Up" or "Down" key to move and adjust the manipulator up and down.

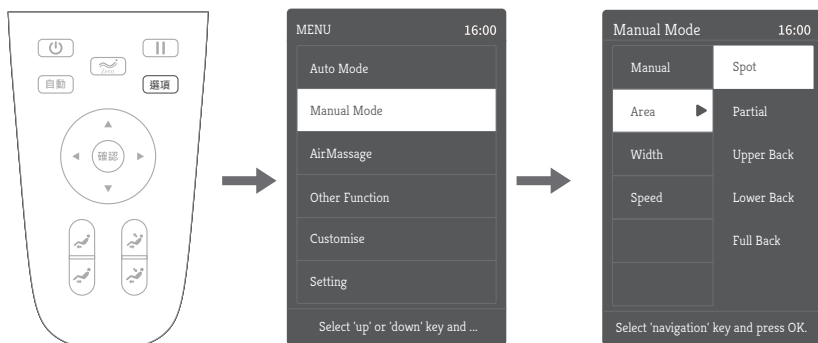

Note: In auto mode, the manipulator cannot be moved up and down through the "up key" or "down key".

### 3. Pause Button

When the massage is in progress, pressing the "pause" button will stop all massage functions; and pressing it again will resume them. If the pause time exceeds 20 minutes, all massage functions will automatically shut down.

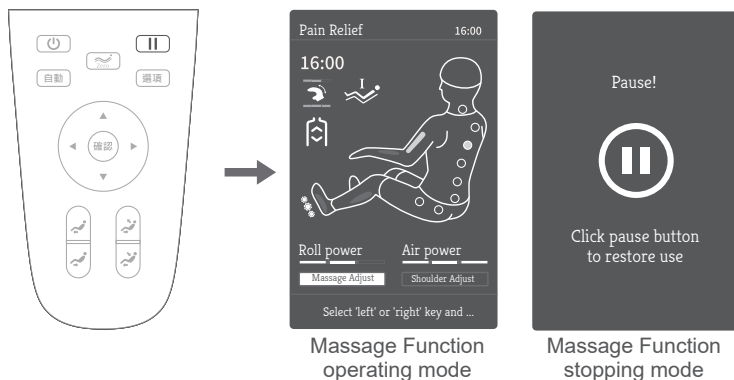


Massage Function  
operating mode

Massage Function  
stopping mode

## How to Use

### 4. Menu Button

Press “menu” to enter masaaage function interface, where there are auto massage mode, manual mode, air pressure massage, auxiliary function, storage and setting, etc. Use the “up” or “down” button to choose the desired function and then press “OK” to confirm. Click “left” to go back to the previous page.

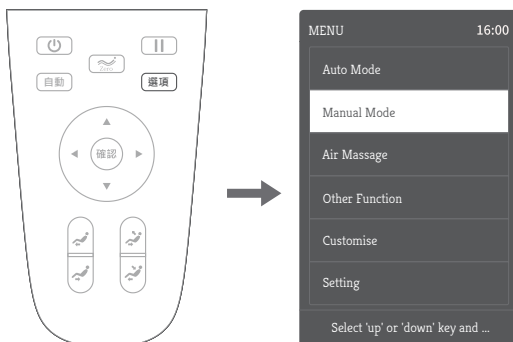

#### A) Auto Massage: (Menu - Auto massage; press “auto” button to enter directly into the auto-massage interface)

- Press “menu” to enter masaaage function interface, Use the “up” or “down” button to choose the auto massage function and then press “OK” to confirm.
- Auto massage function includes: Pain Relief, Neck And Shoulder, Bottom Toning, Full-body Stretch, Fresh Morning, Humpback Fix, Office Regimen, Spine Massage, Noon Recharge, Relax, Quick Refresh, Comfort, Thai Massage, Sport Recovery and Circulation Boost.
- Use the “up” or “down” button to choose the desired function and then press “OK” to confirm. The system will automatically return to the status page and also show that auto massage is on.






## How to Use

Function	Description
Pain Relief	Soothe fatigue and pain on the human body, relax the limbs and improve micro-circulation, relieve joint pain and restore vitality.
Neck And Shoulder	Massage neck and shoulder area with the special designed program, to effectively relieve shoulder pain and promote blood circulation (to brain).
Bottom Toning	Focus on hip and thigh area for a shaping effect, help burning excessive body fats.
Full-body Stretch	Increase body flexibility by stretching body muscle.
Fresh Morning	Wake body muscle and other mechanism to start the day fresh after a night of sleep.
Humpback Fix	Massage manipulator combine with shoulder airbags, kneading massage focus on neck and shoulder, kneading and tapping massage on back and waist, kneading massage on buttocks, kneading and tapping massage on thigh, foot rollers, legs airbags and arm airbags massage started at same time. With back and leg heat therapy function.
Office Regimen	Especially designed for office workers who tend to sit for a long time. Keep the muscle active by massaging and enhance body health.
Spine Massage	Massage every section of the spinal cord to ensure a health spine. Help with disc herniation and humpback.
Noon Recharge	Boost quality of nap and hence raise work efficiency.
Relax	Dredge meridian to encourage body's microcirculation. Enhance immune system.
Quick Refresh	The most comfortable massage experience.
Comfort	With kneading, shiatsu massage techniques, combined with low-intensity air pressure massage and function, through the 12 health acupoints massage, help to regulate and balance on human body; At the same time, the massage chair automatically swings back and forth to form a zero-gravity cradle massage state, so that people can fall asleep more quickly, improve sleep quality.
Thai Massage	Traditional Thai massage consisting of whole-body stretching. Comprehensively erase body tiredness.
Sport Recovery	By pressing and rubbing with the air bag and rolling with the wheel, it can relieve the swelling of the shoulders, arms, knees, calves and other key parts after exercise, and quickly recover the fatigue after exercise.
Circulation Boost	Improve blood circulation, reenergize body and boost the immune system.

## How to Use


### B) Manual Massage: (Menu - Manual)

- Press “menu” to enter masaage function interface, Use the “up” or “down” button to choose the manual massage function and then press “OK” to confirm.
- Manual massage includes: massage modes, areas, width and speed, etc.
- Use the “up” or “down” button to choose the desired function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

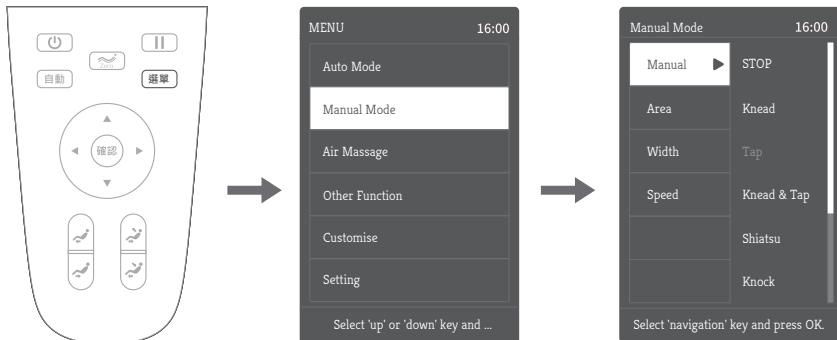

### (i) Manual - Mode

- Method: including 7 different massage methods such as knead, tap, knead and tap,shiatsu, knock,and 3D (3DI and 3DII).The width of the manipulator massage ball can be adjusted under the massage methods including tapping, shiatsu,knocking and 3DII. Except for stopping, the massage speed can be adjusted under other massage methods.
- In the manual function, use the “up” or “down” button to choose the desired function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

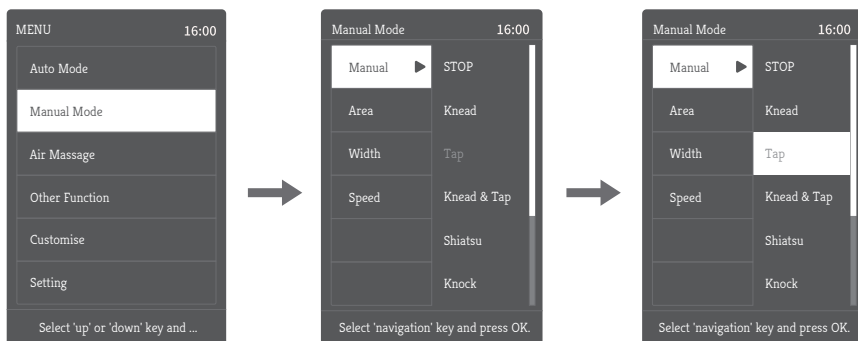


## How to Use

Function	Mechanical Hand Adjustment	
STOP	Width	3 speed levels available 3 width levels available
Knead	Speed	
Tap	Speed & Width	
Knead & Tap	Speed	
Shiatsu	Width	
Knock	Speed & Width	
3D I	Speed	
3D II	Speed & Width	

### (ii) Manual - Position

- Position : Spot, Partial, Upper Back, Lower Back and Full Back.
- In the manual function, use the “up” or “down” button to choose the desired position and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

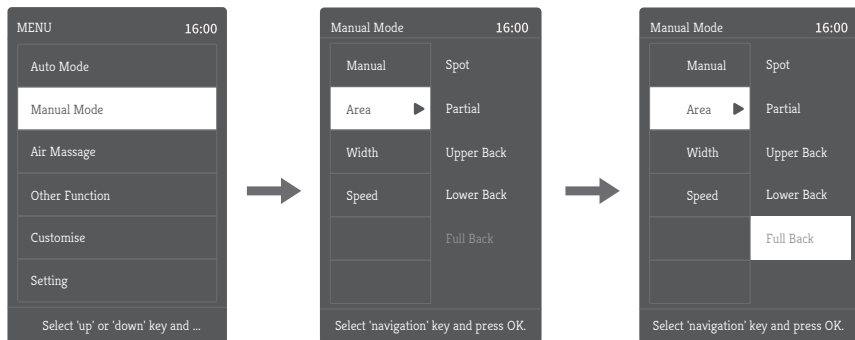


Function	Description	
Spot	Massging a particular point.	When choosing “spot” and “partial”, adjust the targeted area by using “up” and “down” buttons in the status page.
Partial	Massaing a particular small area.	
Upper Back	Circling around the upper back.	
Lower Back	Circling around the lower back.	
Full Back	Circling around the whole back.	

## How to Use

### (iii) Manual - Width


- Width: 3 width levels of adjustment of mechanical hands.
- In the manual function, use the “up” or “down” button to choose width function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

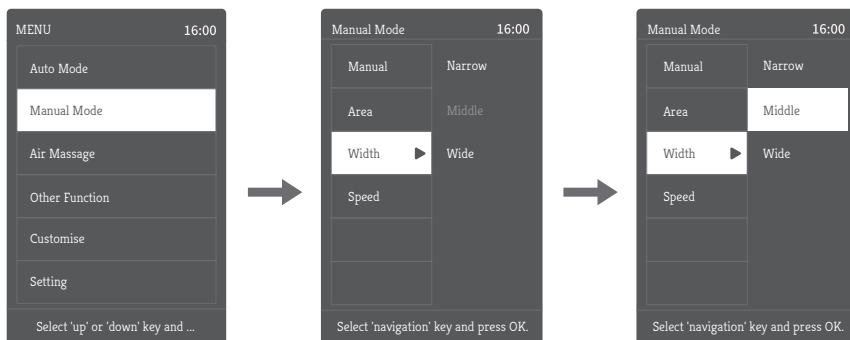

Note: Width: the width can be adjusted under the massage methods including tap,shiatsu,knock and 3D II, 3 levels of adjustment available.

### (iv) Manual - Speed


- Speed: 3 speed levels of adjustment of mechanical hands.
- In the manual function, use the “up” or “down” button to choose speed function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

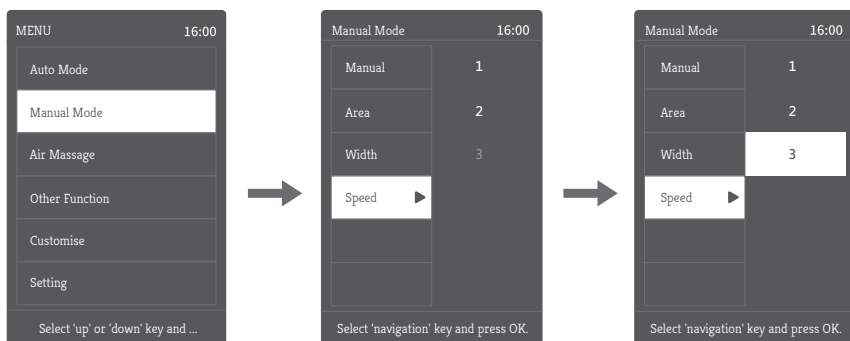

## How to Use


### C) Air Pressure Massage: (Menu - Air Pressure)

- Press “menu” to enter message function interface, Use the “up” or “down” button to choose the air pressure function and then press “OK” to confirm.
- Air pressure massage: upper body, lower body and air pressure.
- In the air pressure massage function, use the “up” or “down” button to choose a desired air pressure message function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

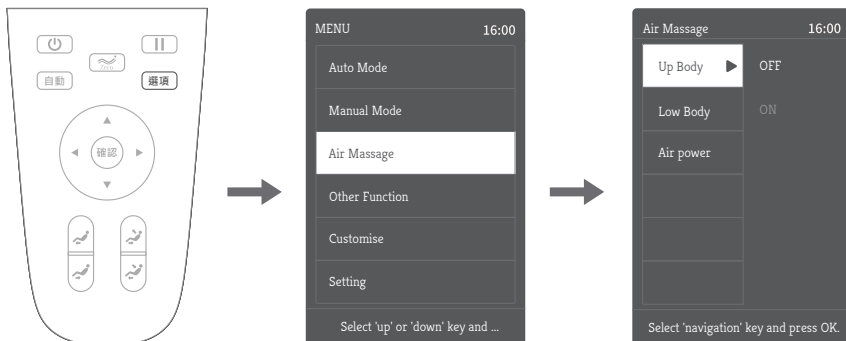

### (i) Air Pressure Massgae - Upper Body


- Upper body air compression: including plam air pressure, arm air pressure, shoulder air pressure, and other air compression massage functions.
- Enter the air compression message interface, press the "Up" or "Down" key to select the upper body air compression message function, enter the upper body air compression sub-menu using the "Right" key to select, then press the "Up" or "Down" key to select to turn on or off the upper body air compression function, and press the "OK" key to confirm. Press the "Left" key to return to the previous menu interface.

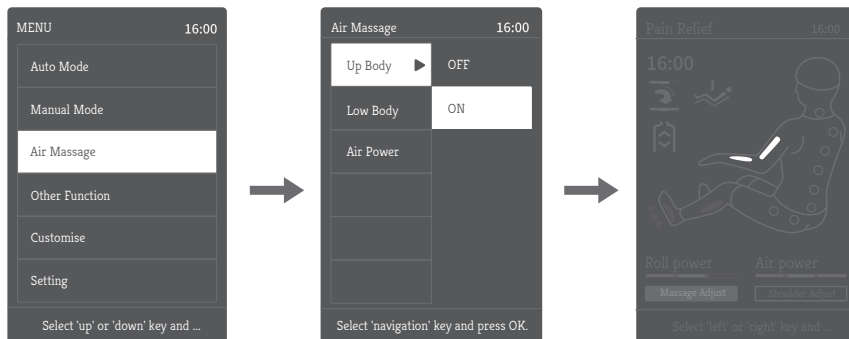

## How to Use


### (ii) Air Pressure Massage - Lower Body

- In the air pressure massage function, use the “up” or “down” button to choose lower body air pressure function and then press “right” to enter the sub-menu of the said function. Again, use the “up” or “down” button to switch on /off the lower body air pressure function. Press “OK” to confirm. Click “left” to go back to the previous page.

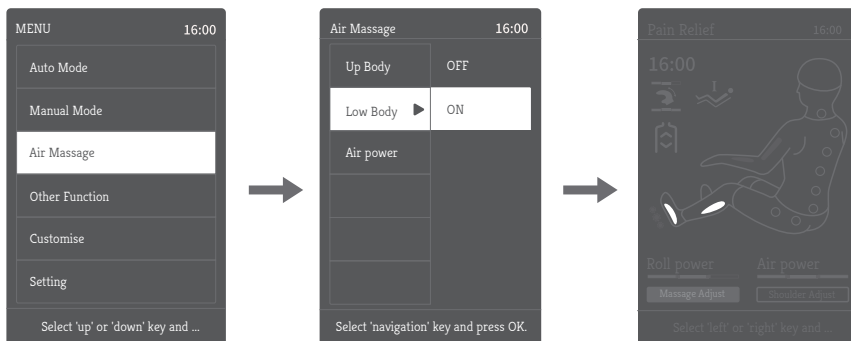

### (iii) Air Pressure Massage - Air Pressure


- Pressure level : 3 levels available.
- In the air pressure massage function, use the “up” or “down” button to choose air pressure function and then press “right” to enter the sub-menu of the said function. Again, use the “up” or “down” button to choose a desired level of pressure. Press “OK” to confirm. Click “left” to go back to the previous page.

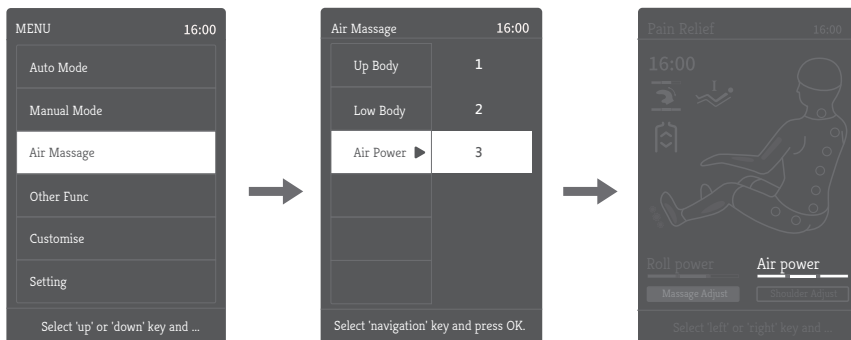

## How to Use


### D) Other Function: (Menu - Auxiliary Function)

- Press “menu” to enter massage function interface, Use the “up” or “down” button to choose the auxiliary function and then press “OK” to confirm.
- Auxiliary Function includes foot roller, back thermotherapy and calf thermotherapy, etc.
- In the auxiliary function, use the “up” or “down” button to choose a desired function and then press “right” to enter the sub-menu of the said function. Press “OK” to confirm. Click “left” to go back to the previous page.

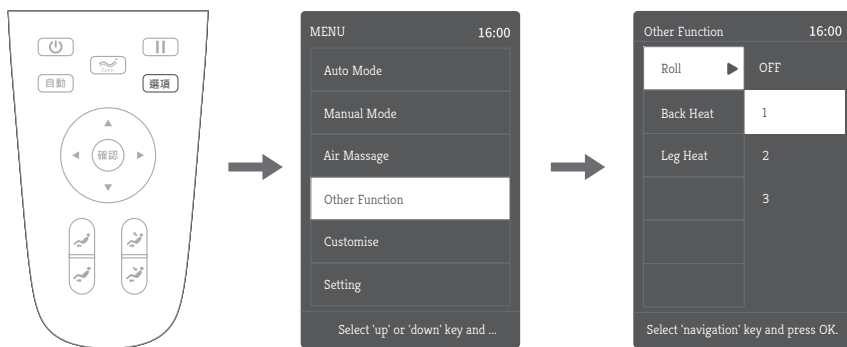

### (i) Other Function - Foot Roller


- Foot roller: 5 levels available. Default level set at 2.
- In the auxiliary function, use the “up” or “down” button to choose foot roller function and then press “right” to enter the sub-menu of the said function. Again, use the “up” or “down” button to choose a desired level of pressure. Press “OK” to confirm. Click “left” to go back to the previous page.

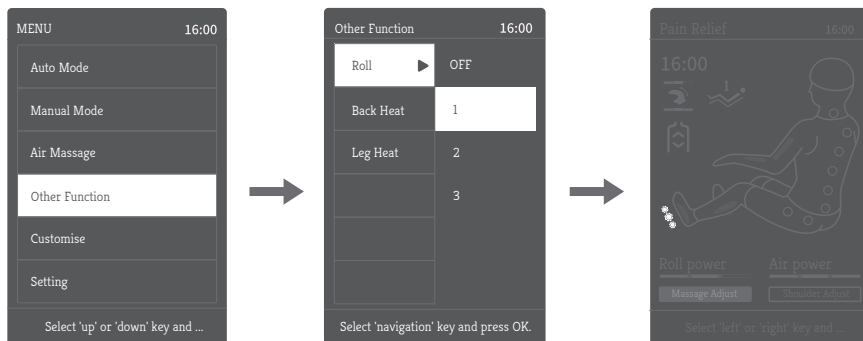

## How to Use


### (ii) Other Function- Back heating function

- Back heating function: thermotherapy in effect after switching on for 3 minutes.
- In the auxiliary function, use the “up” or “down” button to choose back thermotherapy function and then press “right” to enter the sub-menu of the said function. Again, use the “up” or “down” button to switch on /off the thermotherapy function. Press “OK” to confirm. Click “left” to go back to the previous page.

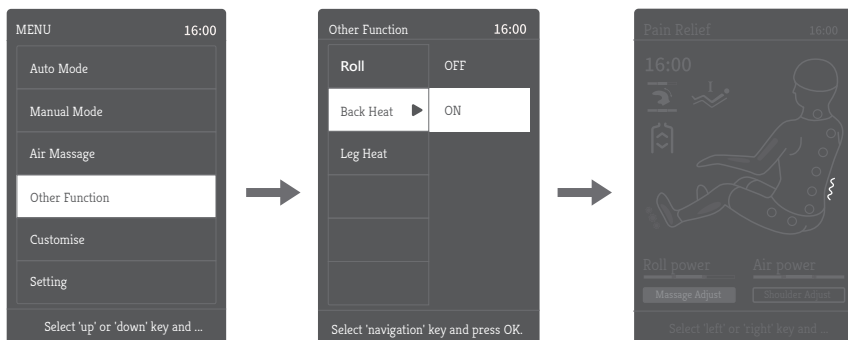

### (iii) Other Function- Calf heating function


- Calf heating function: thermotherapy in effect after switching on for 3 minutes.
- In the auxiliary function, use the “up” or “down” button to choose calf thermotherapy function and then press “right” to enter the sub-menu of the said function. Again, use the “up” or “down” button to switch on /off the thermotherapy function. Press “OK” to confirm. Click “left” to go back to the previous page.

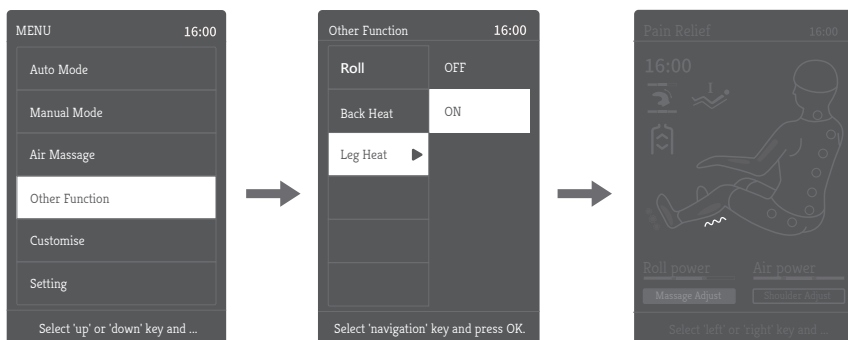

## How to Use

### E) Storage: (Menu - Storage)



- Press "menu" to enter message function interface, Use the "up" or "down" button to choose the storage function and then press "OK" to confirm.
- Storage: Memory 1, Memory 2, Memory 3.
- In the storage function, use the "up" or "down" button to choose Memory I-III and then press "right" to enter the sub-menu to select whether to employ or store by using the "up" or "down" button. Press "OK" to confirm. Click "left" to go back to the previous page.
- Save: to store the current message program, including message modes, air pressure setting and its level, the chair position and other message functions.
- Use: to employ a previously stored message program, including message modes, air pressure setting and its level, the chair position and other message functions.

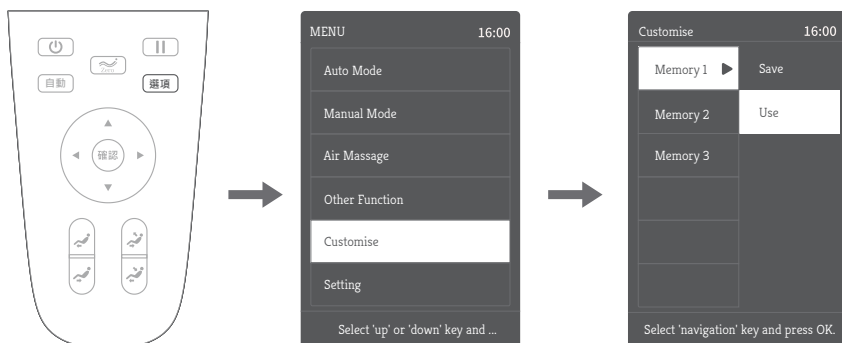

### F) Setting: (Menu - Setting)




- Press "menu" to enter message function interface, Use the "up" or "down" button to choose the setting function and then press "OK" to confirm.
- Setting: Bluetooth, language and serial numbers, etc.
- In the setting function, use the "up" or "down" button to choose setting and then press "right" to enter the sub-menu to select whether to employ or store by using the "up" or "down" button. Press "OK" to confirm. Click "left" to go back to the previous page.

## How to Use

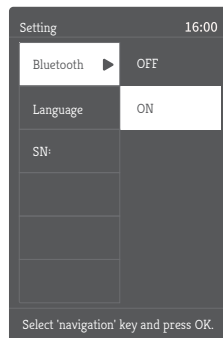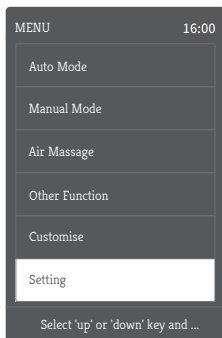

### (i) Setting - Bluetooth


- Bluetooth: Turn on the Bluetooth function to connect to your smart devices (e.g. mobile phone and computer). After pairing, the music played by your device can be played through the sound system of the chair (Bluetooth No. such as IMCM-XXXX).
- In the setting function, use the "up" or "down" button to choose Bluetooth function and then press "right" to enter the sub-menu to switch on/off Bluetooth by using the "up" or "down" button. Press "OK" to confirm. Click "left" to go back to the previous page.

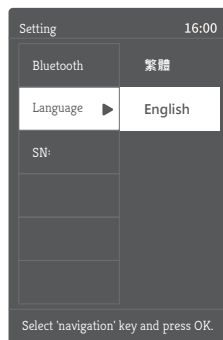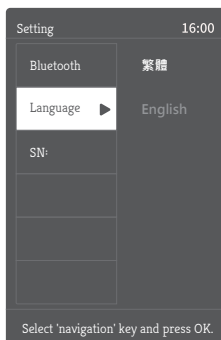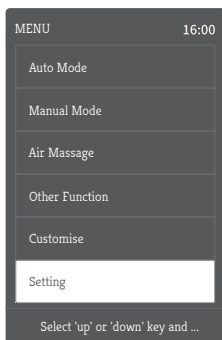

### (ii) Setting - Language

- Language: Traditional Chinese and English.
- In the setting function, use the "up" or "down" button to choose language and then press "right" to enter the sub-menu to choose the desired language by using the "up" or "down" button. Press "OK" to confirm. Click "left" to go back to the previous page.

## How to Use

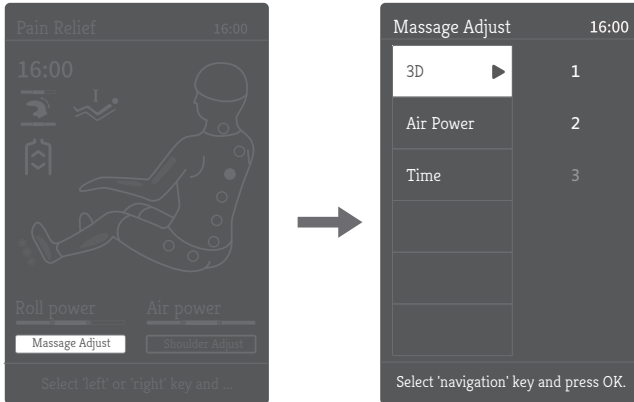


### Customization

- In the status page, press “left” or “right” button to choose customization function.
- Customization includes 3D, air pressure level and time adjustment, etc.
- In the customization page, use the “up” or “down” button to select an adjustment function. Press “right” to enter its sub-menu to choose the desired level by using the “up” or “down” button. Press “OK” to confirm. Click “left” to go back to the previous page.

Function	Description
3D	Neck, back, waist and hip pressure adjustment. 3 levels available.
Air Pressure Level	3 levels available.
Time Adjustment	+10 minutes, +5 minutes, -5 minutes, -10 minutes. 4 options available.

- Depending on the remaining massage time, when the displayed time is less than a certain time adjustment option, that option will not be shown (max. 40 minutes). E.g. The displayed time on the remote control is 39 minutes, the corresponding time adjustment range will be -5 minutes and -10 minutes only (+10 minutes and +5 minutes will not be available).
- Air pressure level adjustment only available when air pressure massage function/ auto massage function is turned on (refer to P.18 of this manual)

## How to Use

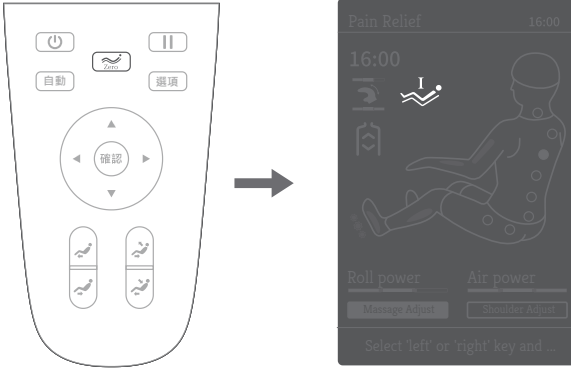

### Zero-gravity Button

- Press the zero-gravity button to adjust the zero-gravity position. Choose Zero-gravity I, Zero-gravity II and Zero-gravity III according to your needs.

Button	Description
	Zero-gravity Button: leg rest and back will be automatically tuned to a zero-gravity position each time this button is pressed. 3 options available.


### Angle Adjustment Button

Button	Description
	Leg rest ascend: Press this button to raise the leg raise; release the button to stop motion.
	Leg rest descend: Press this button to lower the leg raise; release the button to stop motion.
	Sit-up: Press and hold this button, the back cover will slowly ascend while the leg rest will slowly descend. Release the button to stop the motion. If the massage chair is completely sit-up, it will stop the motion automatically.
	Recline: Press and hold this button, the back cover will slowly descend while the leg rest will slowly ascend. Release the button to stop the motion. If the massage chair is completely laid down, it will stop the motion automatically.

## How to Use


### H) End of Massage

- Press the power button when the massage is ongoing will immediately turn off all massage functions. The chair will be set to its starting position.
- When the rated time is up, all massage functions will stop. The chair will however remain in the then position.

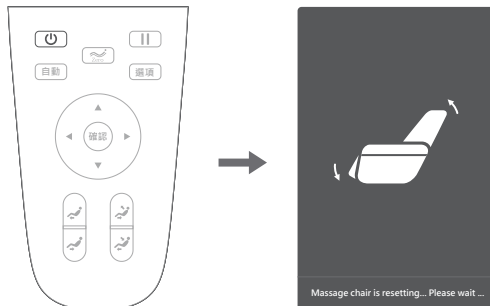


- Cut off all power.


#### ① Switch off the power.

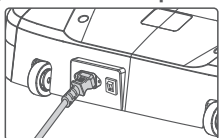

#### ② Unplug the connection wires.

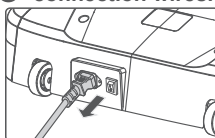

#### ③ Unplug the socket.

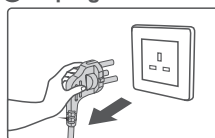

### I) Bluetooth Music

1. In your smart device, go to setting and choose Bluetooth. Turn on Bluetooth, and it will automatically search for new Bluetooth devices.
2. The smart device will show the corresponding Bluetooth device, choose "pair" and get connected (Bluetooth No.such as IMCM-XXXX.).
3. Go to your music player and enjoy!






## How to Use




### 3. Headrest & back cushion installation and use

Using the headrest can lessen the massage intensity borne by the neck and shoulders. You may choose to use the headrest according to your own needs (recommended). Back cushion is attached to the back through zipper (1), headrest is attached to the back cushion through zipper(2).

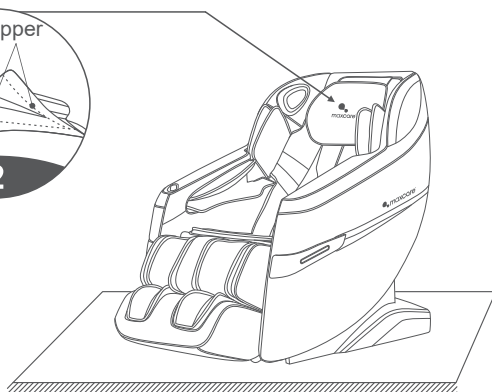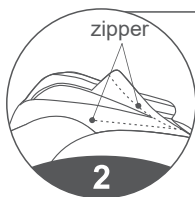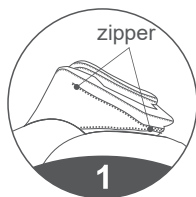

### 4. Floor protection

The floor may be damaged due to the heavy pressure from the massage chair. It is advised to place a floor mat or other materials as a buffer to prevent the damage.

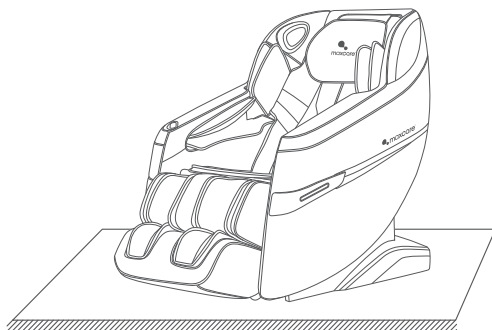

**CAUTION:** Do not move the chair with its wheels on a bumpy ground or in a narrow space. Two people are required to lift the chair. Hold the leg rest cover and the dent in the back cover while lifting the chair (please do not lift from the armrest).




## How to Use



### 5. Method of movement

Please make sure unplugged the power. Hold a groove in the back plastic cover to tilt the backrest backward to a certain degree (with the gravity center resting on the wheels), push the chair forward or backward with your hands and finally resume the chair to the normal position in a slow and gentle manner. (Note: During movement you must turn off the power switch first and then unplug the power line and hand controller connecting line.)

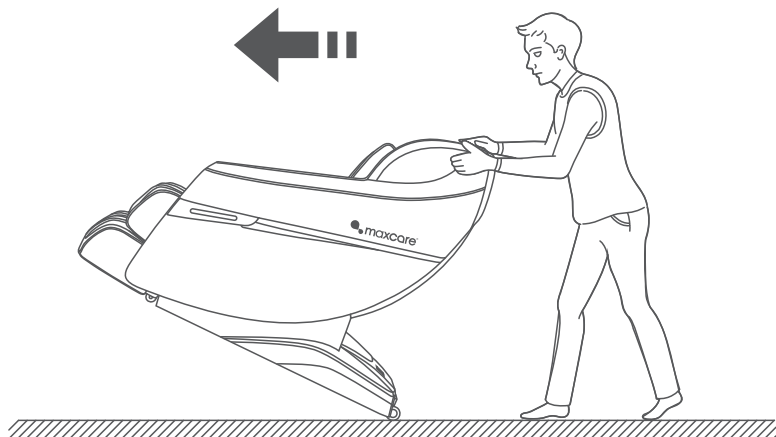

#### CAUTION

The massage chair with its wheel are suitable for moving in a short distance. Please use other tools or method to move the massage chair when you need to move in long distance.






## How to Use






### 3. Instructions for installation of lower leg

- After connecting the lower leg and the air tube on the massage chair with the wire harness, untie the fixed pipeline belt and insert the connecting part into the seat frame (Fig. 1).
- Sleeve the lower leg fixing block on the round tubes on the left and right sides of the lower leg (Fig. 2).

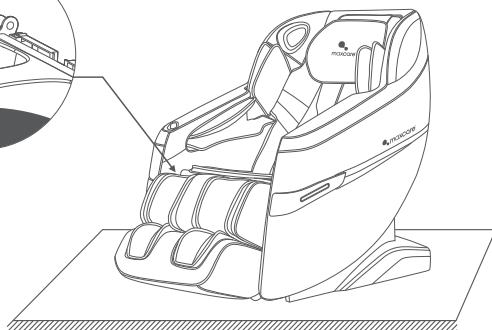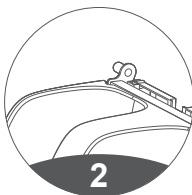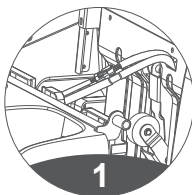

- Hang the lower leg sleeved with the fixing block on the hook of the massage chair seat frame (Fig. 3).
- Fasten the lower leg on the massage chair with 4 M4 screws (2 on the left and 2 on the right). Check whether there is any abnormality in the up-and-down turning and whether the pipeline is folded. Then the lower leg installation is completed (Fig. 4).

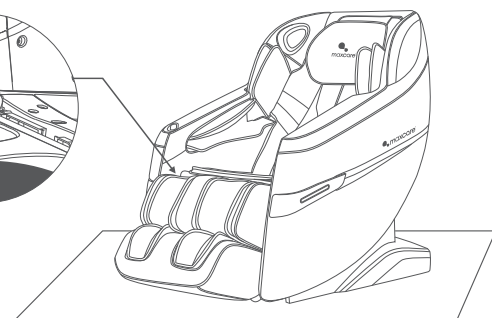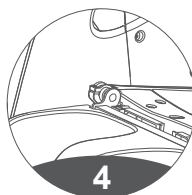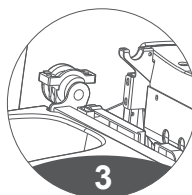

 **Cleaning and Maintenance****Cleaning**

- Before cleaning, please make sure the switch has been turned off and the power plug has been disconnected from the socket.
- Use soft cloths to clean the surface of product and control panel.

**CAUTION**

- Do not pull the plug from the socket with wet hands.
- Please do not rinse the product. Avoid water/ liquids from getting into the product.
- Do not use corrosive cleaning agents like benzene or thinner to clean the product, otherwise, the color of product will fade, and the internal parts may be corrupted or broken.
- Do not disassemble the product when it is humid. Simply let it dry.

**Maintenance**

- Make sure that the plug has been disconnected from the socket before maintenance.
- Keep the product clean and away from high-temperature/ humid areas, or any environment with volatile gases.
- If this product is to be left idle for long, properly cover it with a dust cover (self-purchase).

**CAUTION**

- Please keep the product away from direct sunlight, or high temperature, to avoid fading or product damage.

**Kindly reminding**

- If the product is used through the year, please check conditions as below:
  - \* Burnt smell.
  - \* Unstable power supply after contact with the power cord.
  - \* Power cord heating up.
  - \* Power cords damage.
  - \* Other abnormalities.

**CAUTION**

- If above mentioned happens, please stop using the product.
- Switch off the power supply, pull out the plug, and contact us.
- Do not disassemble the product by yourself.



## Maintenance and Cleaning of PU Leather

PU leather is made by the coating and drying of polyurethane. Compared with genuine leather, PU leather is more resistant to tear, heat and sunlight. Depending on its nature and chemical substances, however, it should be kept away from many dusty, high/low-temperature, humid environment, under direct sunlight or places with corrosive substance. Otherwise, the leather could tear, quickening aging and hydrolysis.



### Cleaning

1. Please do not use organic reagent or grease solution, especially strong solvent, e.g. alcohol, turpentine and thinner, to clean the surface of the PU leather. Otherwise it will corrode the surface.
2. Please do not use water or washing powder to clean the surface of the PU leather. Otherwise there will be cracks.
3. When cleaning the surface of the PU leather, choose a cleaning agent that has a pH value of between 5~7, no dentifrice, and is harmless to the environment.
4. For dirty PU surface, apply egg whites to the surface using a piece of soft cloth. Avoid using a brush for cleaning.



### Maintenance

1. Keep a good ventilation indoor, overly dry or humid environment can accelerate the aging of leather
2. Please keep the product away from direct sunlight or air vent, otherwise the leather will be hardened. Direct sunlight will also cause fading.
3. People are generally sweeter in summer. The holes on the surface of the leather will absorb sweat. While heat and high humidity prompt the chemical reaction between certain substances in sweat and the leather, and hence, causes smell. Please clean the surface more often with a piece of lightly wet cloth.
4. Please do not use alkaline cleaning agents on the leather. It will adversely affect the elasticity of the leather and lead to cracks as a result.



## FAQs and Solutions

No.	Symptoms	Possible Causes	Solution
1	"Tik tik" or "tak tak" sound during motion	Normal mechanical sound during motion or the frictional sound between the massage balls and cover	No action needed
2	Cannot be turned on	The adapter has not been well connected	Connect the adapter properly
		The power switch is off	Switch on the power button
		The power plug has not been well connected	Connect the plug to the socket properly
3	Abrupt stop during operation	Power disconnected	Check if the power cord is connected
		Rated working time (20 minutes) is up and sleep mode is on	Press the power button to restart
		The machine has been in use for too long, causing overheating protection	Press the power button to restart the machine after cooling down
		The massage hands are stuck	Reduce the pressure on the product

If the abnormal situations persist after taking the above solutions, please contact us.



## Technical Specifications

Product Name	: Prime Chair
Product Model	: MAX-78A202S (Silver) MAX-78A202B (Black)
Rated Voltage	: 220-240V ~ 50/60Hz
Rated Power	: 90W
Rated Time	: 20minutes
Noise	: ≤ 55dB
Product Gross Weight	: 88KGS
Product Net Weight	: 70KGS
Air Pressure	: 0.025-0.03MPa
Product Size (Erected)	: 137CM*74CM*107CM
Product Size (Stretched out)	: 158CM*74CM*93CM
Back Max. Loading Capacity	: 90KGS
Leg rest Max. Loading Capacity	: 30KGS
Safety structure	: Class I
Material	: PU, leather, cotton, wood
Material Description	: PVC, PA, steel, electronic parts
Compliance	: GB4706.1-2005 GB4706.10-2008



商標持有：美斯凱實業有限公司  
Trademark holder: Maxcare Industrial Limited

客戶服務熱線 Customer Service Hotline :  
香港 Hong Kong - (852) 2950-9383  
中國 China - (86) 400 882-1883

[www.maxcareonlineshop.com](http://www.maxcareonlineshop.com)  
[www.maxcare.net](http://www.maxcare.net)