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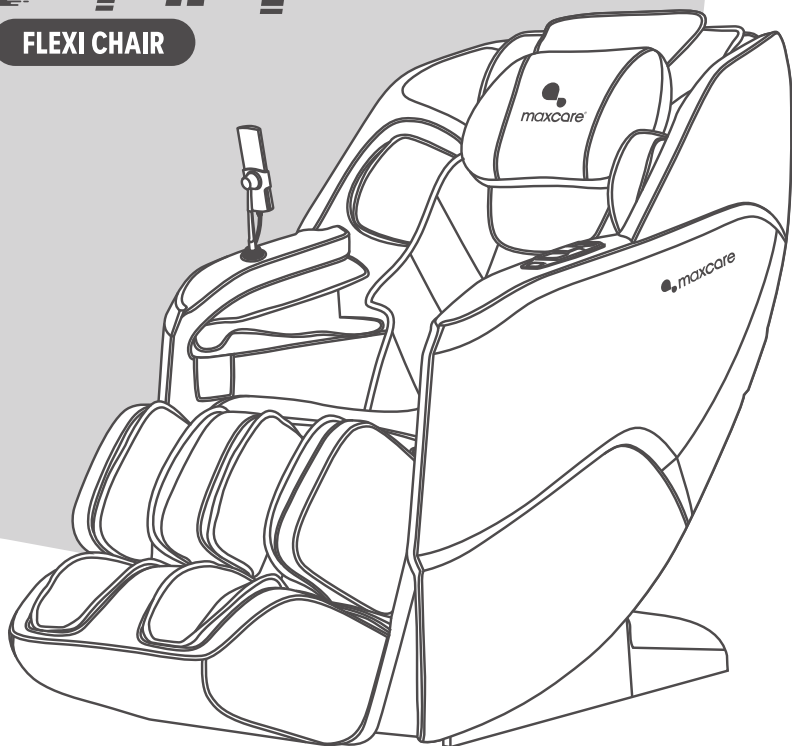
FLEXI CHAIR

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MAX-Z05A2801

  
maxcare®

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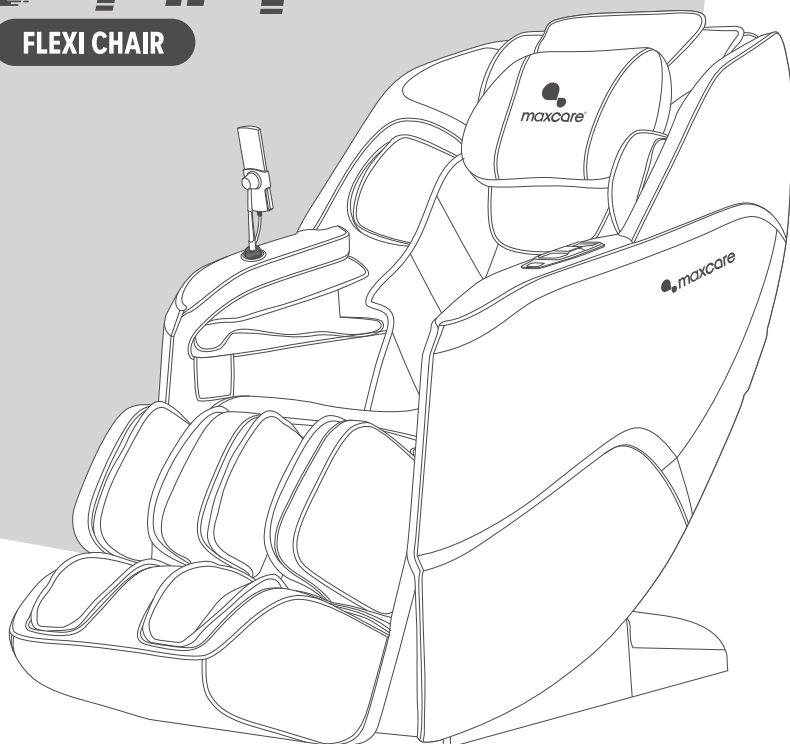
MAX-Z05A2801

## COMPANY PROFILE

After two decades of experience in the health industry, **maxcare** founder utilized his knowledge and expertise in massage therapy from an Asian-Pacific perspective to establish our brand - **maxcare**. Our massage therapy business was expanded to Hong Kong in 2012 and is conferred 2017 Hong Kong Top Brand Award. Based in Hong Kong, we are specializing in design & innovation, manufacturing, and retailing of massage and healthcare products.

Believing in the value of 'Technology improves our quality of life' and 'Caring for your family', **maxcare** has been innovating highly qualified products with international patents. Besides, **maxcare** products fulfill various international safety standards, namely EU CE Mark, US UL/ETL Mark, Germany GS Mark, TUV certification, and China CCC Mark.

Motivated by our value, **maxcare** founder has been leading us to be one of the most competitive brands in Hong Kong health industry. **maxcare** applies the contemporary fashion design – neat and modern – on the mechanical engineering of massage chairs. To deliver our value and offer incredible massage experience and healthcare solution to the public, **maxcare** devotes to the innovation, design, manufacturing the top-notch family massage products. By integrating the company philosophy –Spirit of Healthy Life - into our products, **maxcare** brings health to everyone.

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## FLEXI CHAIR

Thank you for purchasing **maxcare**'s products. Before using the product, please read this manual thoroughly, particularly the Safety Precautions section, for the correct use of the product.

This manual should be well kept for easy reference. It enables you to perform a safe and efficient operation.

### CONTENT

---

|                                 |             |
|---------------------------------|-------------|
| Safety Precautions              | P.01 - P.06 |
| Name and Function of Components | P.07 - P.11 |
| How to Use                      | P.12 - P.52 |
| Cleaning and Maintenance        | P.53 - P.54 |
| FAQs and Solutions              | P.55        |
| Technical Specifications        | P.56        |

## Safety Precautions

### 1. Important safety warning

- Please read this manual instruction carefully before operation.
- Any other usage that is not listed in this manual instruction is forbidden.
- Keep children and pets away from the movable parts of this product to avoid accidents caused by the up-and down-ward movement of the calf rest and back cover.
- When in use, sliding or transporting, please ensure that there are no people or pets nearby (the back, bottom, front and sides of the product), and that no foreign objects or pets inside the product, to avoid accidents.
- Please use well-grounding power supply suited to this product.
- Please pull out the plug after use or before cleaning to avoid injury or damage to the product.
- Do not use unauthorized accessories.
- Do not use this product outdoors.
- People whose bodies are wet are forbidden to use this product.
- Recommended usage time of this product is 20 minutes/ session.
- Please check if any part of this products is damaged before use. If so, please do not use it and contact us for repair.
- Please do not use it if the cover is broken or damaged.
- Please do not use it if the shell or surface is damaged.
- Please do not drop anything into this product.
- Please do not fall asleep while using this product.
- Please do not use it if drunk or feeling unwell.
- Please do not use it within one hour after having meal.
- Please do not adjust to the maximum intensity at the beginning of the massage to avoid possible injury.
- Please do not keep sharp objects in pockets when use.
- Keep the plug clean from needles, rubbish or water.
- If the power cord is damaged, do not attempt to change on your own. Please contact us for repair for safety reasons.

### 2. Environment for usage

- Please do not use it in a moist environment such as bathrooms.
- Please do not use it when there is a drastic change in temperature.
- Please do not use it under heavy-dusty or caustic environment.
- Please do not use it where there is not enough space or not good ventilation.
- Please switch off the power and unplug the machine after use.
- Please do not use it when surrounding temperature is up to 40°C.
- Please do not use it when surrounding temperature is below 5°C. The lubricant can thicken quickly and increase the friction between parts.
- Turn off and pull out the plug during thunderstorms.
- Place the product on a flat surface.
- Do not use outdoors.
- Do not place it near fire or directly under the sun.
- When moved from a low-temperature storage to a warm environment, the moisture inside the machine may affect, or damage, its functionality. Please rest the product within room temperature for 1 hour before use.

### 3. People who have the below symptoms, please consult a doctor before use

## Safety Precautions

- Pregnant women or women that are in menses period.
- People who got injured or skin diseases.
- Children under 14-year old or people mentally unbalanced without being supervised are not allowed to use this machine.
- People who are told by doctor to have rest or people who feel unwell.
- The appliance has heating function. People who are insensitive to heat must be careful when using the appliance.
- People who are receiving medical treatment, especially those who are physically unfit.
- People who have open wounds.
- People who aim for recovery.

## 4. Safety

- Please do not place heavy objects on this product.
- Do not use other therapy devices while using this product.
- This product is not to be used for medical treatment purpose.
- Only one person at a time.
- Recommended usage time of this product is 20 minutes/ session.
- It is possible to place a mat underneath the machine to avoid damage to the floor.
- For safety reason, maintain a minimum of 0.5m between the product and others when use.
- Check if the voltage is suitable to the specification of this product.
- Do not use wet hands to pull the plug.
- Do not allow water to get into the product, to avoid electric shock or damage to this product.
- Do not pull or drag the power cord when unplugging. Please be gentle.
- Do not damage the wires or change the circuit of this product.
- Do not use wet cloths to clean the electric parts such as the switch and plug.
- Be away from this product during power cut to prevent injury in case power resumes suddenly.
- Stop using this product if there appears any irregularity, and please contact us immediately.
- Stop using this product if it causes discomfort, and please consult a doctor.
- This appliance is not intended for people (including children) with reduced physical, sensory or mental capabilities, or a lack of experience and knowledge, unless they are under supervision or instruction concerning the use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- If the power cord is damaged, it must be replaced by us in order to avoid a hazard.
- Do not ride on the handle to avoid injury or malfunction.
- Keep the power cord away from hot or moving objects.
- Please cut off power and pull out plug before moving.
- If the product is found damaged, broken, suffering from power leakage, or with exposed parts, please stop using it.

## 5. Maintenance

- This product should be maintained by us, users are prohibited to disassemble or maintain by themselves.
- Please always turn off the main power after use.
- Do not use this product if the socket is loose.
- If this product is to be left idle for long, please tie up the power cords and keep it in dry and dustless environment.
- Do not keep this product at a high temperature or near fire. Do not keep under direct sunlight.
- Please clean this product with dry cloths. Do not use thinner, benzene or alcohol.
- The mechanical components of this product are specially designed and made, no special maintenance required.
- Do not point sharp objects at this product.
- Do not roll or pull this product above uneven ground, it should be lifted before moving.
- Please use it intermittently; do not keep it continuously running for long.

## 6. Solution of ordinary malfunctions

- It is normal if the motor makes noises during operation.
- If the remote control can't work normally, please check the plug and socket if they are connected, and make sure the switch is turned on.
- If the rated working time is over, the power will automatically go off. If this product works continuously for too long, the temperature control will automatically turn off the machine, if so, please allow a 30-minute break before resuming.

**CAUTION**

Please make sure all the power is cut off before moving.  
The power cord and all other cables should be disconnected.

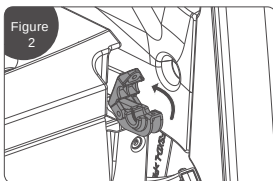
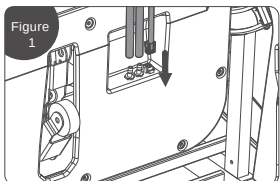
## Safety Precautions

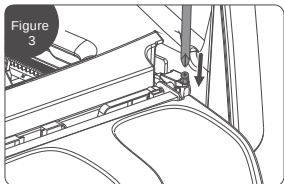
### Installation parts

#### Calf mechanism installation instructions

1. Insert the leg wire and air pipes into the respective front ports of the steel frame assembly.  
The air pipe with the white-marked clip connects to the white pipe connector. The other air pipe without a marked clip connects to the black pipe connector (Figure 1).
2. After completing the connection, open the calf securing caps on both sides of the steel frame assembly (Figure 2).

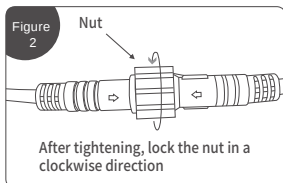
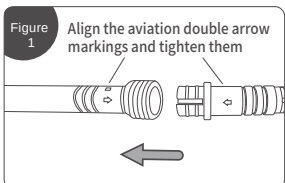
3. Raise the calf section, place it into the slot, cover it with the securing caps, and finally fasten it in place with two M4 screws (Figure 3).

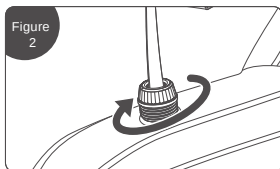
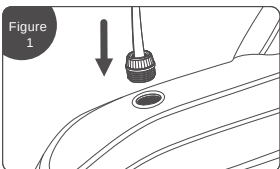
#### Installation instructions for the aviation plug of the manual controller

Non fixed aviation plug installation method: Align the arrow marks of the aviation plug (Figure 1) and the aviation plug (Figure 2), then tighten them, and then lock the nut clockwise.

#### Remote control holder installation instruction

Align the threads on the bracket assembly support knob with the corresponding thread position of the bracket mount on the armrest (Figure 1), turn clockwise and tighten to secure (Figure 2)

## Safety Precautions

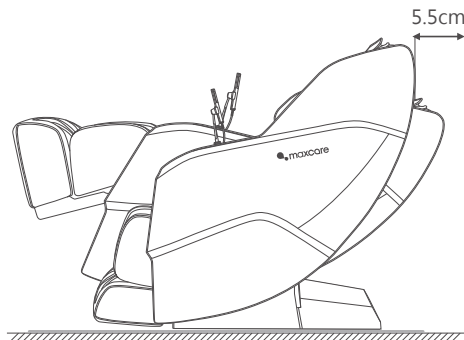
### Place massage chair

#### Installation placement

**Make sure there is enough space for the massage chair can be tilted at least 5.5 cm.**

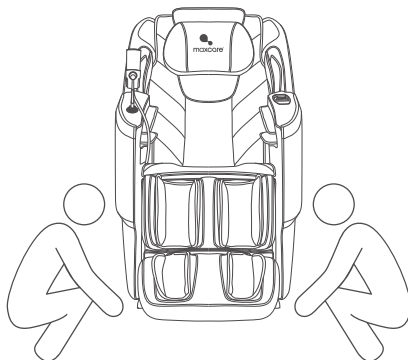
Do not expose the massage chair to direct sunlight or high-temperature environments (e.g., in front of heating devices). Place it in an area where it can avoid heat and direct sunlight. For long-term use in a fixed location, it is recommended to place a soft cushion underneath. The cushion should be large enough to cover the area where the chair and footrest touch the floor.

#### Product lift instruction

**Two-person lifting required due to product weight.**

1. Please lift the massage chair when moving it on delicate surfaces, such as wooden floors.
2. When moving on similar surfaces, lifting is recommended to avoid damaging the casters. Be cautious when lowering the chair into the designated position to avoid and or foot injuries. Do not release the chair until it is completely flat. To prevent floor damage, it is recommended to place a mat or protective covering on the floor.

## Safety Precautions

### Place massage chair

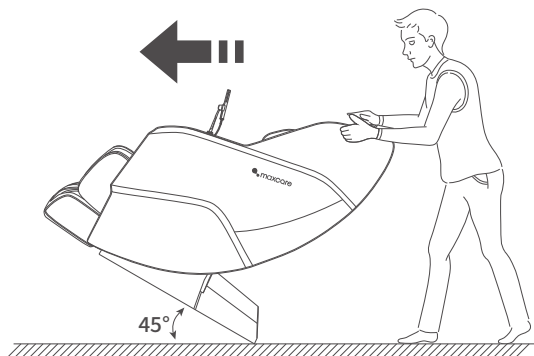
#### Move method

Use the rear casters to move the massage chair. Tilt the chair backward approximately  $45^{\circ}$  as shown in the diagram to move it.

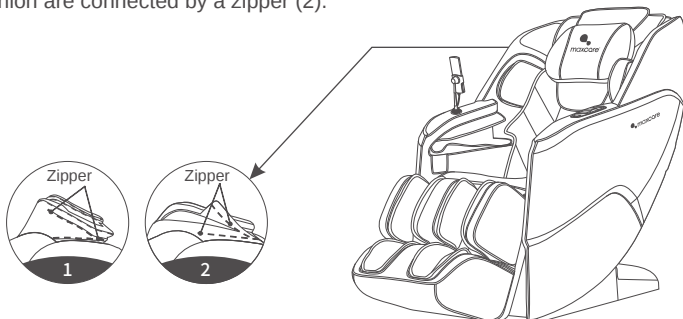
**NOTE:** Excessive force to tilt will cause the chair to tip over completely and product damage.

1. Before moving, ensure the massage chair is properly powered off, with both the backrest and calf sections fully reset, and the power supply disconnected.
2. The chair should remain empty during movement; no people, pets, or objects should be on it.
3. Do not move the chair over obstacles higher than 2mm or gaps wider than 5mm.
4. When moving the chair using the casters, avoid continuous movement over distances greater than 50 meters. If necessary, move the chair slowly and at a steady pace.

#### How to disassemble and use headrest cushion and backrest cushion

Using a headrest cushion can reduce the intensity of kneading massage on the neck and shoulders, and you can decide whether to use a headrest cushion according to your needs (recommended to use a headrest cushion). The backrest cushion and the backrest are connected by a zipper (1), and the headrest cushion and the backrest cushion are connected by a zipper (2).

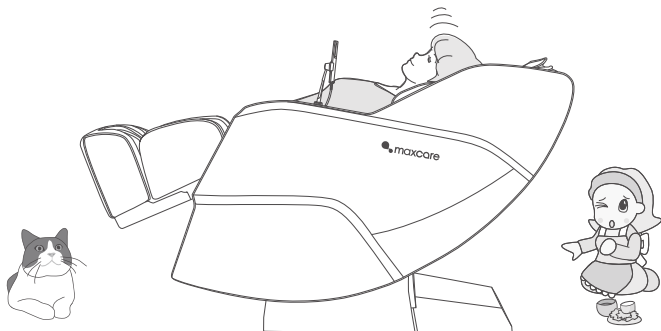
## Safety Precautions

### Before use

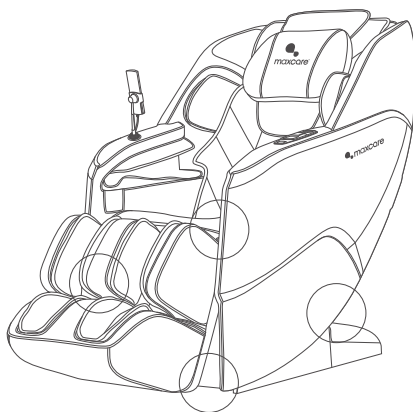
#### Check around

Make sure that there are no people, pets and other objects around the unit.

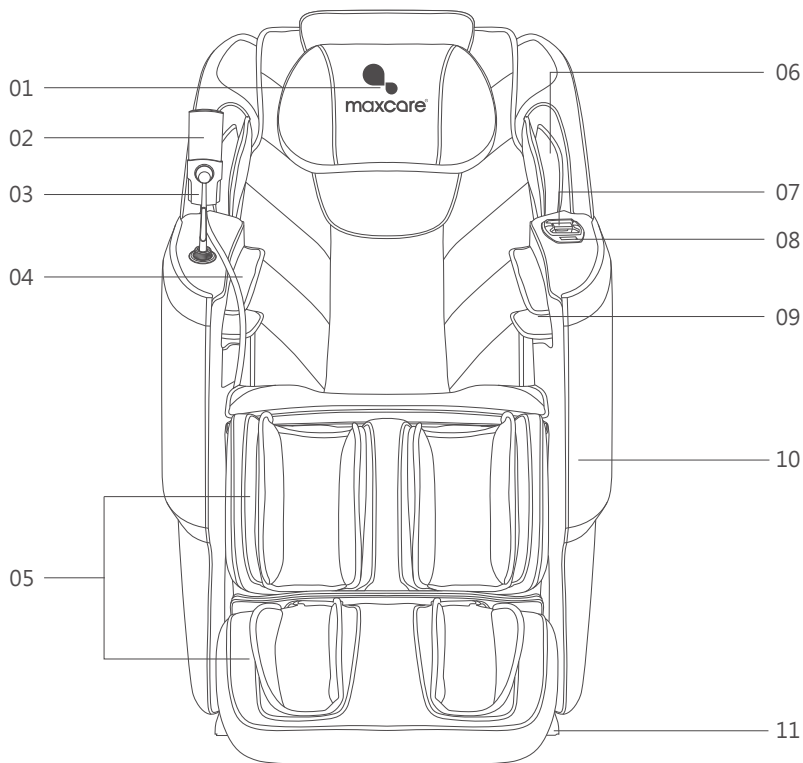
#### Make sure there are no foreign objects in the gaps of the machine

Before massaging, make sure that no foreign objects are caught in the legs, feet, etc. Under no circumstances allow hands, feet, and head to penetrate into the area between the lower leg mechanism and main body, or the area between the armrest and the capsule. Under no circumstances should hands, feet, or head sink under the fabric or leather covering of the massage area.

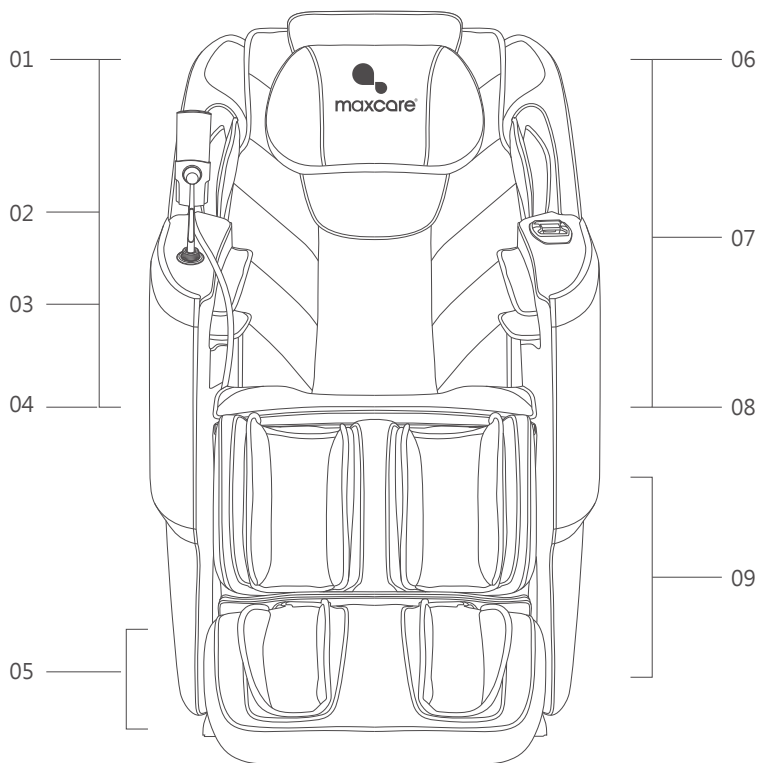
## Name and Function of Components

- 01.Head Cushion
- 02.Remote Control
- 03.Remote Control Holder
- 04.Armrest: Airbags Assembly
- 05.Leg: Massage Mechanism
- 06.Shoulder: Airbags Assembly

- 07.Armrest Shortcut Button
- 08.USB / Type-C Charging Port
- 09.Arm: Airbag Assembly
- 10.Armrest
- 11.Side Cover

## Name and Function of Components

### 01.Head Sides

- 3D Digital Audio
- Music Playback

### 02.Shoulder Sides

- Air Massage

### 03.Head&Neck to Seat

- Manipulator Massage

### 04.Right Armrest

- Air Massage

### 05.Foot

- Air Massage
- Rollers + front and back scraping massage
- Ankle Airbag Lump Massage

### 06.Left Head Side

- Voice Control
- Built-in voice-activated microphone

### 07.Back

- Heating Function

### 08.Left Armrest

- Air Massage

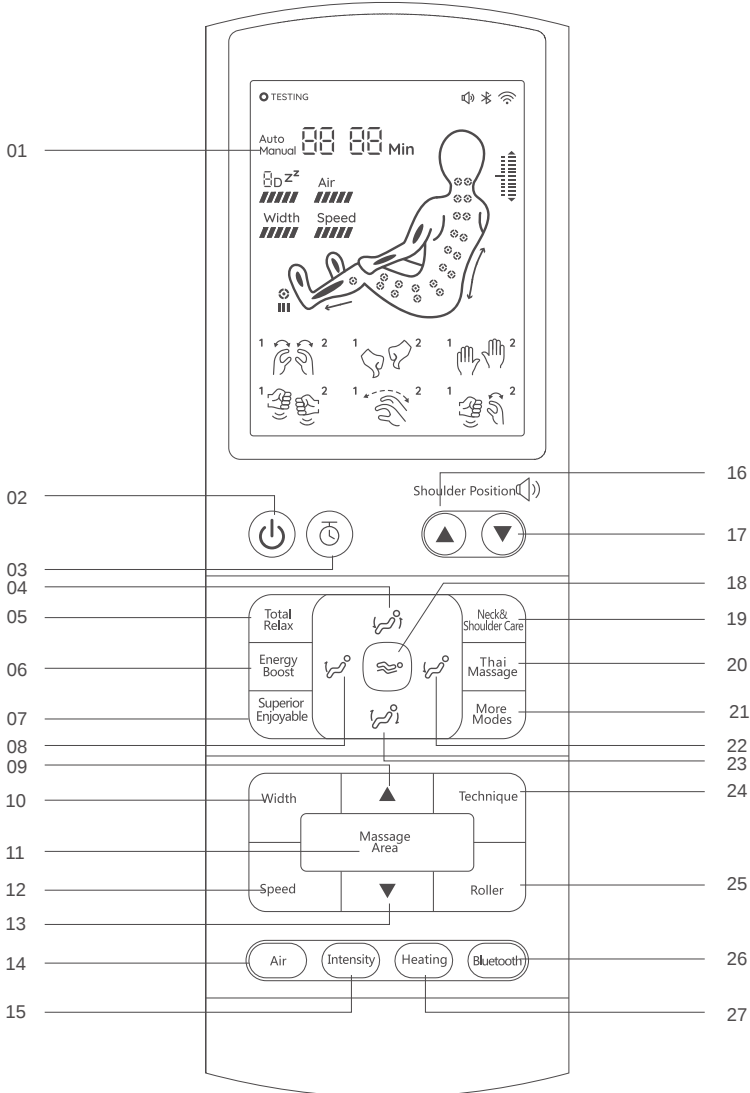
### 09.Leg

- Air Massage

## Name and Function of Components

### Remote controller keys instruction

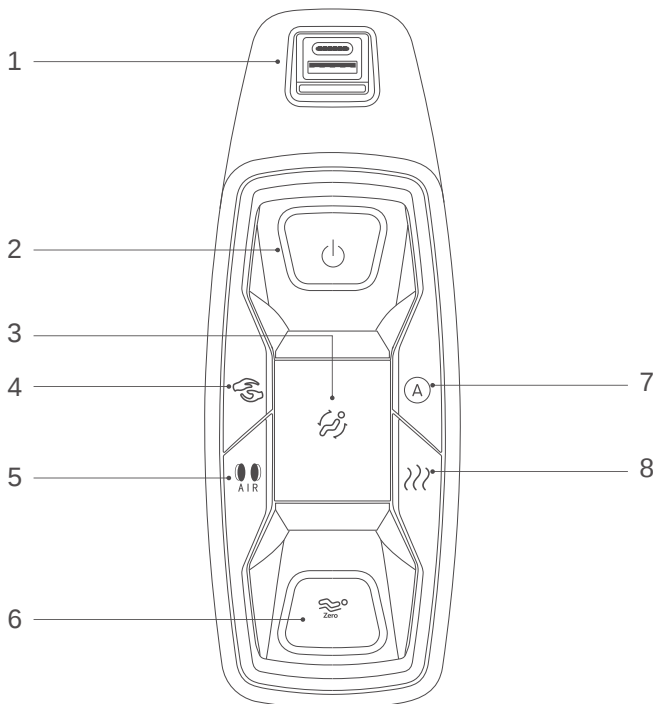
## Name and Function of Components

### Remote controller keys instruction

- 01.Display: liquid crystal display which can display the massage function in running state.
- 02.Power key: Turn the massage chair on or off.
- 03.Timer key: Massage Timer Adjustment
- 04.Linkd Sit-Up Key: Press and hold this button to link the adjustment of the backrest angle upwards, and the calf angle downwards, release to stop the adjustment.
- 05.Total Relax key: Active Total Relax massage program.
- 06.Energy Boost key: Active Energy Boost massage program.
- 07.Superior Enjoyable key: Active Superior Enjoyable massage program.
- 08.Calf Upward key: Press and hold this button to adjust the calf section upwards, release to stop the adjustment.
- 09.Massage Up Key: Adjust the massage area upwards
- 10.Width key: Massage width is adjustable in 5 levels.
- 11.Area key: Cycle switching massage area selection.
- 12.Speed key: Massage speed is adjustable in 5 levels.
- 13.Massage Down Key: Adjust the massage area downwards.
- 14.Air key: Turn air massage on or off. Cycle through Upper / Lower / Full / Off.
- 15.Ait Intensity key: Air massage intensity is adjustable in 5 levels.
- 16.Shoulder / Volume Up Key: Adjust shoulder position upwards. Long press for volume up adjustment.
- 17.Shoulder / Volume Down Key: Adjust shoulder position downwards. Long press for volume down adjustment.
- 18.Zero-Gravity key: Zero-Gravity angle adjustment. Cycle through Zero I / Zero II / Off.
- 19.Neck & Shoulder Care key: Active Neck & Shoulder Care massage program.
- 20.Thai Massage key: Active Thai Massage massage program.
- 21.More Auto key: Cycle through more auto massage program.
- 22.Calf Downward key: Press and hold this button to adjust the calf section downwards, release to stop the adjustment.
- 23.Linkd Reclining Key: Press and hold this button to link the adjustment of the backrest angle downwards, and the calf angle upwards, release to stop the adjustment.
- 24.Technique key: Turn on or off the massage technique.12 massage techniques available
- 25.Roller key: Turn on or off the massage rollers. Massage intensity is adjustable in 3 levels.
- 26.Bluetooth key: Long press to turn on or off the Bluetooth.
- 27.Heat key: Short press

## Name and Function of Components

### Armrest shortcut key description

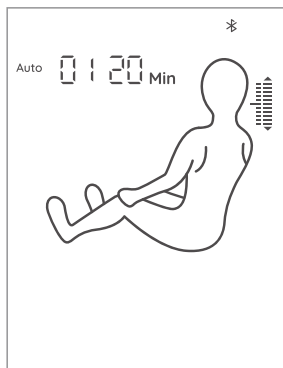
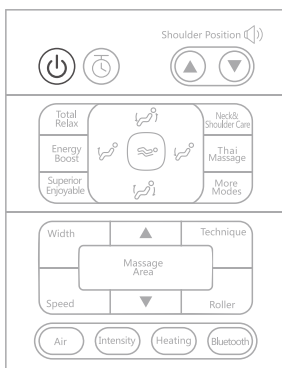
- 01.USB / Type-C Charging Port: Provides charging for mobile phones or other devices.
- 02.Power Button: Long press this button for 3 seconds to turn on or off the massage chair. After turning on the massage, short press for pause/run.
- 03.Linkage Lift Key: Pull forward once and hold to raise the seat position and lift the massage chair upright. Pull backward once and hold to lower the seat position and recline the massage chair. Pull forward twice quickly and hold to lower the calf section. Pull backward twice quickly and hold to raise the calf section. Release to stop adjustment.
- 04.Massage Technique Key: Press this button to cycle through the massage techniques, including no technique, kneading (Kneading I-Kneading IV), tapping (Tapping I-Tapping II), knead&tap (Knead&Tap I-Knead&Tap IV) and Shiatsu, total in 12 massage techniques to choose from.
- 05.Air Pressure Key: Press to turn the air pressure function on or off.
- 06.Zero-gravity Key: Press this button to cycle through the massage chair's zero-gravity angle adjustment (2 setting angles are available).
- 07.Auto Massage Key: Press this button to cycle through the auto massage functions (18 kinds of auto-massage functions are available).
- 08.Heating Key: Press this button to turn on or off the back heating function.

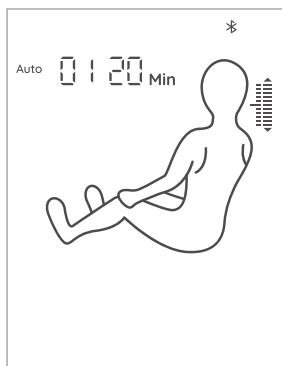
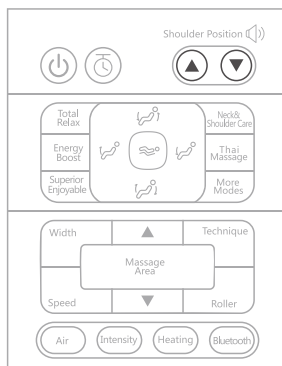
## How to Use

### I. Start massage

1. Massage Techniques: Press the "Power" button on the remote control to cycle through the massage techniques.

2. After switching on the massage chair will automatically adjust the angle, after adjustment will start human shoulder position detection, at this time, please keep the body and sitting posture and massage chair fit.
3. Shoulder position fine-tuning, after body type detection is completed, the hand controller or voice prompts will prompt the user to fine-tune the shoulder position, if the massage wheel stops at your shoulder position, there is no need to adjust and directly start massage, if not, please use the shoulder position of the "up key / volume up" key or "down key / volume down" key to adjust the massage wheel to your shoulder position and then start the massage. If it is not operated for more than 10 seconds then it will directly default the current massage wheel position as shoulder position and start massaging. Note: Pressing the corresponding key during power on or detection can switch the default automatic programme mode, and pressing it again can skip the detection and enter the massage directly.

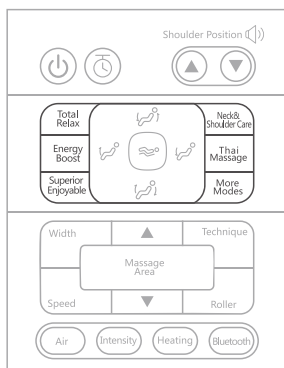
## How to Use

### II. Automatic program (please use this function after the chair is turned on)

- (1) Automatic programs selection
  - (1.1) Select click on the remote controller auto program button area to select the auto program to start the massage.
  - (1.2) There are a total of 18 automatic programs; 5 featured automatic programs have independent buttons and can be selected directly, and the other automatic programs are switched by the "More Modes" button.

Featured programs: Total Relax, Energy Boost, Superior Enjoyable,  
Neck & Shoulder Care, Thai Massage

More Programs: Chinese Massage, Fatigue Recovery, Gentle Waves Rub,  
Meridian Treatment, Waist Care, Leg & foot Relieve, President Cosy,  
Keep Fit, Office Regimen, Health Recharge, Lunch Break,  
Warming and detox, Sweet Dream.

| Function           | Description                                                                                                                                                                                                                                                                                                                                | Display |
|--------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------|
| Total Relax        | With the aid of flexible guide rails, let the body posture angle closer to lying flat; with the effective anti-arch stretching of movement to the back, achieve zero pressure stretching, effectively relieve lumbar muscle pain, and eliminate fatigue, thus to get a better massage experience, relieve the stress on the body and mind. | 01      |
| Energy Boost       | Deeply massage the whole body by means of kneading, tapping and Shiatsu techniques, to wake up the vitality of the human body and stimulate the energy.                                                                                                                                                                                    | 02      |
| Superior Enjoyable | Apply air massage on the whole body, with the foot roller and calf kneading massage. Calf stretching function is canceled, suitable for the elderly and people suffering stress on back. The massage intensity is gentle.                                                                                                                  | 03      |

## How to Use

|                      |                                                                                                                                                                                                                                                                                                                                                                                                         |    |
|----------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----|
| Neck & Shoulder Care | Massage the neck and shoulders by means of kneading and stretching techniques mainly, and also the upper back by medical massage, and then press the Jianjing acupoint to relieve cervical spondylosis and achieve the effect of relaxing the muscles of the neck and shoulders. Massage intensity is gentle, suitable for people working in office and suffering from cervical fatigue.                | 04 |
| Thai Massage         | Mainly use calf and backrest electric push rod to clamp legs, hands and shoulders with the aid of air massage, and then apply stretching massage, while the movement holds against the back and waist for medical massage, so that the body is in the Thai stretching state. It is suitable for people under high stress to improve their head-down on chest and humpback. Massage intensity is strong. | 05 |

### More modes

|                    |                                                                                                                                                                                                                                                                                                                                                                      |    |
|--------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----|
| Chinese Massage    | By means of Traditional Chinese medicine massage technique mainly, apply massage on the back and waist so as to promote the blood circulation of the back and waist, soothe the meridians and relieve the blockage of the back and waist through massage. It is suitable for people with back and waist pain, and lumbar muscle strain. Massage intensity is gentle. | 06 |
| Fatigue Recovery   | Apply deep pressure on the neck, back and waist, buttocks, and legs by means of kneading, patting, finger pressing techniques mainly to relieve the fatigue on body, allowing the body to better relax.                                                                                                                                                              | 07 |
| Gentle Waves Rub   | Apply full air massage mainly, with the use of airbags for cross circulation of kneading and pressure, so as to promote blood circulation, alleviate pain in various body parts, and deeply relieve muscle fatigue. (Remarks: full air massage mainly, with frequent air massage action)                                                                             | 08 |
| Meridian Treatment | By means of kneading, patting, finger pressing and medical massage techniques mainly, clamp the legs, hands and shoulders with the aid of air massage, and then apply stretching massage to soothe the meridians, promote blood circulation, relieve blockages in the body.                                                                                          | 09 |
| Waist Care         | Massage the waist by kneading, kneading&tapping, tapping, and then perform 4D massage of the waist                                                                                                                                                                                                                                                                   | 10 |

## How to Use

|                       |                                                                                                                                                                                                                                                                                                                                                                                                                                 |    |
|-----------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----|
| Leg & foot<br>Relieve | Massage the shoulders and waist by means of kneading, tapping, kneading and tapping mainly, apply air massage on the calves and feet for squeezing massage, allowing more soothing calf and foot massage.                                                                                                                                                                                                                       | 11 |
| President<br>Cosy     | Massage the shoulders, neck, back and waist by means of kneading, tapping, kneading and tapping mainly, to relieve fatigue, promote blood circulation, and release stress by such deep massage. It is suitable for men, and the massage intensity is moderate.                                                                                                                                                                  | 12 |
| Keep Fit              | Massage the shoulders, neck, waist and hips by means of kneading and Shiatsu techniques, through the massage on the whole body, to relax the body and mind, and enjoy the queen-level treatment. It is suitable for women. Massage intensity is gentle.                                                                                                                                                                         | 13 |
| Office<br>Regimen     | Massage the shoulders, neck and waist by means of kneading, tapping, kneading and tapping techniques mainly, to relieve shoulder and neck pain, improve lumbar muscle soreness and maintain lumbar spine through such deep massage. It is suitable for office white-collar and sedentary people.                                                                                                                                | 14 |
| Health<br>Recharge    | Deeply massage the whole body by means of kneading, tapping and Shiatsu techniques, to promote blood circulation throughout the body and prevent blood clots. It is suitable for people who live at home for a long time and do not like to go out, and the massage intensity is moderately gentle.                                                                                                                             | 15 |
| Lunch<br>Break        | Massage the back and waist by means of kneading and tapping techniques, and press the Shenyu acupoint, to improve the quality of lunch break, so that the user is more energetic in afternoon work.                                                                                                                                                                                                                             | 16 |
| Warming<br>and detox  | Deeply massage the whole body by pressing related acupoints by means of kneading, patting, finger pressing and other massage techniques, to let the stimulation fully reach the deep layer of muscle tissue, feel warm locally combined with the function of hot compress, and regulate the Qi and blood in a bid to achieve the effect of warmth and damp expelling. (Note: the whole body massage together with hot compress) | 17 |
| Sweet<br>Dream        | Massage the shoulders, neck, back and waist by means of kneading and medical massage techniques, and press the acupoints of Fengchi and Shenyu, to improve the sleep quality and help sleep. It is suitable for people suffering from poor sleep, and the massage intensity is gentle.                                                                                                                                          | 18 |

## How to Use

### III. Automatic program (Please use this function after power on)

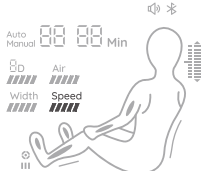
- (1) Massage Techniques: Press the “Technique” button on the remote control to cycle through the massage techniques.
- (1.1) Techniques include none, Knead (4 techniques), Tap (2 techniques), Knead&Tap (4 techniques), Shiatsu(1 techniques) a total of 12 techniques are available. Press the “Technique” button to cycle through all 12 techniques. And display the technique on the interface.

| Function                                                                                                   | Massage roller adjustment         |
|------------------------------------------------------------------------------------------------------------|-----------------------------------|
| Stop                                                                                                       | The width and be adjusted.        |
| Knead I                   | The speed and be adjusted.        |
| Knead II                  | The speed and be adjusted.        |
| Knead III (flashing)      | The speed and be adjusted.        |
| Knead IV (flashing)       | The speed and be adjusted.        |
| Tap I                     | The speed, width and be adjusted. |
| Tap II                    | The speed, width and be adjusted. |
| Knead&Tap I               | The speed and be adjusted.        |
| Knead&Tap II              | The speed and be adjusted.        |
| Knead&Tap III (flashing)  | The speed and be adjusted.        |
| Knead&Tap IV (flashing)   | The speed and be adjusted.        |
| Shiatsu                   | The width and be adjusted.        |

5 speed levels adjustable  
5 width levels adjustable

- (2) Massage Speed: Press the “Speed” button on the remote control to adjust the massage speed (in a total 5 levels adjustable, level 1 is the slowest, level 5 is the fastest). Speed adjustment should be applied to the following massage techniques: Knead (Knead I - Knead IV), Tap (Tap I - Tap II), Knead&Tap (Knead&Tap I - Knead&Tap IV).

- (2.1). Speed displayed on the interface.

| Speed Display                                                                       | Speed Adjustment                                                                                                                                                                                |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  |  <div> <div>level 1</div> <div>level 2</div> <div>level 3</div> <div>level 4</div> <div>level 5</div> </div> |

## How to Use

- (3) **Massage Width:** Press the “Width” button on the remote control to adjust the massage width (in a total of 5 levels adjustable, level 1 is the narrowest, level 5 is the widest). Width adjustment should be applied to the following massage techniques: Stop, Tap(Tap I-Tap II), Shiatsu
- (3.1) Width displayed on the interface.

| Width Display | Width Adjustment                                                       |
|---------------|------------------------------------------------------------------------|
|               | <div>level 1      level 2      level 3      level 4      level 5</div> |

- (4) **Massage Area:** Press the “Up” or “Down” button for massage area on the remote control to adjust it.
- (4.1) There are 5 massage areas to choose from: Spot / Part / Full Back./ Upper Back / Lower Back.

| Function                                                                                  | Description                                  |                                                                                                                                                                                                                                     |
|-------------------------------------------------------------------------------------------|----------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Spot     | Fixed-point massage at fixed-point position. | When selecting fixed-point and local massage techniques, the massage area can be adjusted through the "up key" and "down key" next to the massage mechanism key of the remote controller under the status of information interface. |
| Partial  | Small-scope local massage back and forth.    |                                                                                                                                                                                                                                     |
| Full     | Full back massage back and forth.            |                                                                                                                                                                                                                                     |
| Upper    | Upper back massage back and forth.           |                                                                                                                                                                                                                                     |
| Lower    | Lower back massage back and forth.           |                                                                                                                                                                                                                                     |

- (5) **Air Massage:** Press the “Air” button on the remote control to select the air massage function. There are 3 options for air massage: Upper, Lower, Full Body. “Off” to turn off the air massage function.

| Air pressure displayed       |                              |                                  |
|------------------------------|------------------------------|----------------------------------|
| <b>Upper Air Massage</b><br> | <b>Lower Air Massage</b><br> | <b>Full Body Air Massage</b><br> |

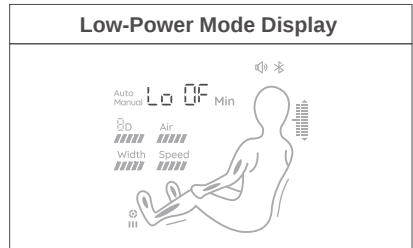
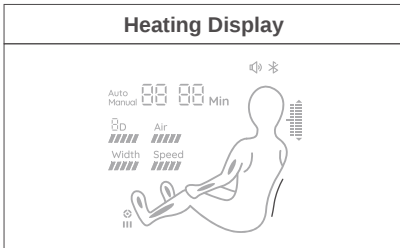
## How to Use

- (6) **Air Massage Intensity:** Press the “Intensity” button on the remote control to adjust the air massage intensity (in a total 5 levels adjustable, level 1 is the weakest, level 5 is the strongest). The air massage intensity can only be adjustable after activating the air massage function.

| Air Intensity Display | Air massage Intensity Adjustment                                       |
|-----------------------|------------------------------------------------------------------------|
|                       | <div>level 1      level 2      level 3      level 4      level 5</div> |

## IV. Other Function (please use this function after the chair is turned on)

- (1) **Heating:** Short press the “Heat” button on the remote control to turn on or off the back heating function. There is a noticeable warmth after activating the heating function for 3 minutes.
- (1.1) Heating displayed on the interface.
- (2) **Low-Power mode:** Long press the “Heat” button on the remote control to turn on or off the Low-Power mode. After activating the low-power mode, the massage chair enters a low-power state when powered off, turning off all functions. It can only be turned on using the power button on the remote control
- (2.1) 2.1.Low-Power displayed on the interface.

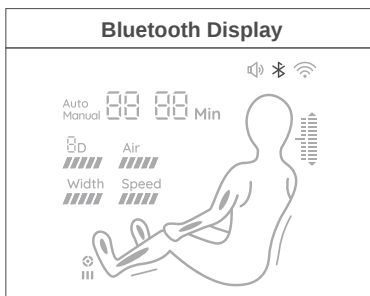
- (3) **Sole Roller:** Press the “Roller” button on the remote control to turn on or off the sole roller function. The rollers speed is adjustable in 1 to 3 level. Each press of the roller button increases the speed by one level. Pressing it a fourth time turns off the sole roller function, cycling through the options in sequence.

| Speed Display | Speed Adjustment                             |
|---------------|----------------------------------------------|
|               | <div>level 1      level 2      level 3</div> |

## How to Use

- (4) Volume Adjustment: Long press the "Shoulder / Volume Up" or "Shoulder / Volume Down Key" button for shoulder position adjustment on the remote control to adjust the background music volume.
- (5) Bluetooth: Press the "BT" button on the remote control to turn on or off the Bluetooth function. After activating the Bluetooth function, you can pair and connect Bluetooth-enabled audio devices (such as smartphones, MID tablets, etc.) with the massage chair's Bluetooth module. This allows music played on the audio device to be wirelessly transmitted by Bluetooth to the massage chair's sound system for playback.
- (5.1) Enter the main menu of your smart device, open "Settings" → select the "Bluetooth" option → turn on the "Bluetooth" → Bluetooth will automatically "search for new Bluetooth devices".
- (5.2) The smart device will detect the corresponding Bluetooth model (e.g., MAX-Z05A2801-\*\*\*\*). Tap the pair button; If prompted whether to pair with this device, select "Pair" to complete the pairing process.
- (5.3) Open the music player on your smart device and select a song to play.

## V. Angle adjustment (please use this function after the chair is turned on)

### (1) Zero-gravity display

| Function | Description                                                                                                                                                                    |
|----------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <br>Zero | Press the "Zero" button on the remote control to cycle through the zero gravity angle options (2 angle options available). Each press switches to the next angle in the cycle. |

### (3) Whole body angle

| Function | Description                                                                                                                                                                                                                                                                                                               |
|----------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|          | Linkage descending key: press and hold this key, the backrest mechanism slowly descends while the leg mechanism slowly rises, release it to stop the backrest mechanism from descending and the leg mechanism from rising; when adjusting to the limit position, it will stop adjusting and emit a "drip drip" sound.     |
|          | Linkage Rise Key: Press and hold this key, the backrest mechanism rises slowly and the leg mechanism falls slowly at the same time, release it to stop the backrest mechanism from rising and the leg mechanism from falling, and it will stop adjusting when it reaches the limit position and emit a "drip drip" sound. |

## How to Use

### (3) Leg mechanism angle

| Function                                                                          | Description                                                                                                                                                                                                          |
|-----------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Leg mechanism descending key: press and hold this key the leg mechanism slowly descending, release it to stop descending; when adjusting to the limit position, it will stop adjusting and emit a "drip drip" sound. |
|  | Leg mechanism rising key: press and hold this key the leg mechanism rises slowly, release it to stop rising; when adjusting to the limit position, it will stop adjusting and emit a "drip drip" sound.              |

## VI. End the massage

During the massage session:

- (1) Press the "power" button on the remote control to turn off all massage functions and
- (2) reset the massage chair to its default angle.
- (3) press the armrest power key, all massage functions are closed, massage chair angle reset.

When the timer ends, all massage functions will turn off, but the massage chair will not reset to its default angle.

### **Warm Reminder**

The touchscreen hand controller operation for the massage chair has ended.

## How to Use

### VII. Instructions for connecting the massage Chair via Bluetooth APP

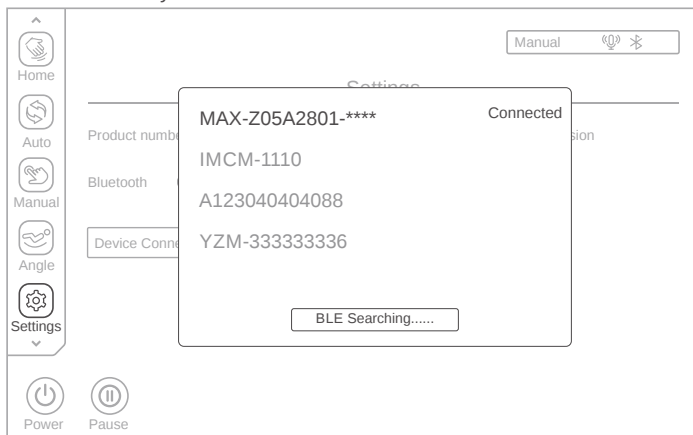
- (1) 1. Bluetooth APP download and install:
  - (1.1) 1.1. Android (Android system) download method: Search "Maxcare massage chair" in Google Play (Google Store) to download and install the Android version, or scan the QR code below to download and install. After successful installation, you can use
  - (1.2) the Bluetooth APP to operate the massage chair.
  - 1.2. Apple (iOS system) download method: Search "Maxcare massage chair" in the App Store (Apple App Store) to download and install the Android version, or scan the QR code below to download and install. After successful installation, you can use the Bluetooth APP to operate the massage chair.

Maxcare massage chair

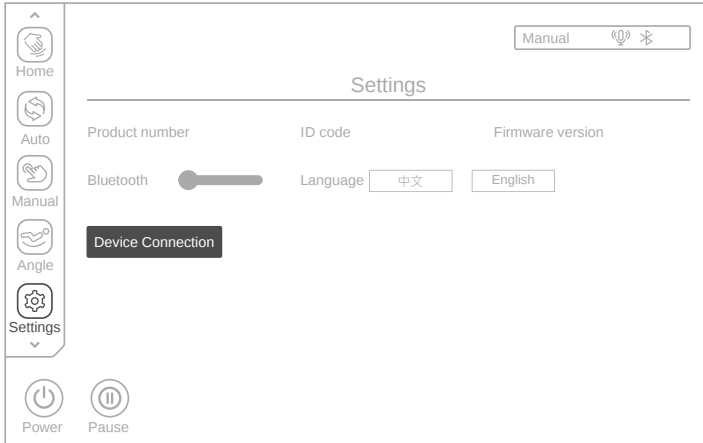
- (2) APP Bluetooth search connection: After the APP is successfully installed, click the "Maxcare massage chair" icon in the smart device interface. After opening the "Maxcare massage chair" APP, it will automatically jump to the Bluetooth connection interface. At the same time, the APP will automatically search for the corresponding device Bluetooth number (such as MAX-Z05A2801-\*\*\*\*), then click on the corresponding device Bluetooth number (and displays connected), then the connection is successful. After the connection is successful, you can operate the massage chair through the APP. Note: The Bluetooth function and location information of the smart device system must be turned on before the device is connected.

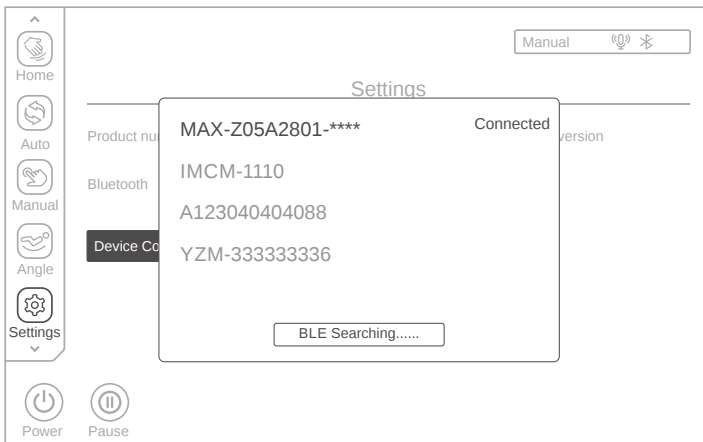
## How to Use

- (3) APP Bluetooth name change: If the location of the massage chair is changed or the Bluetooth name is changed, you need to reconnect the device according to the actual situation; after entering the APP interface as shown below, select and click the "Setting" key in the left menu bar, and then select "Device connection" key to connect the device. Note: The Bluetooth function and location information of the smart device system must be turned on before the device is connected.

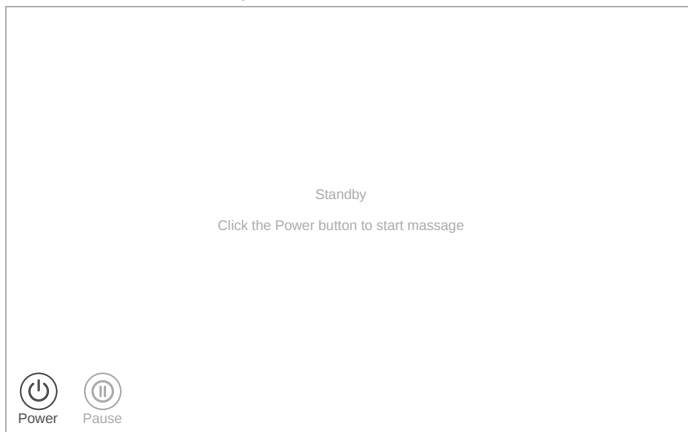
- (3.1) After entering the setting interface as shown below, select and click the "Device connection" key. In the pop-up window interface, the APP will automatically search for the corresponding device Bluetooth number (such as MAX-Z05A2801-\*\*\*\*), and then click to select the corresponding device Bluetooth number ( and displays connected), then the connection is successful. After the device is successfully connected, the massage chair can be operated.

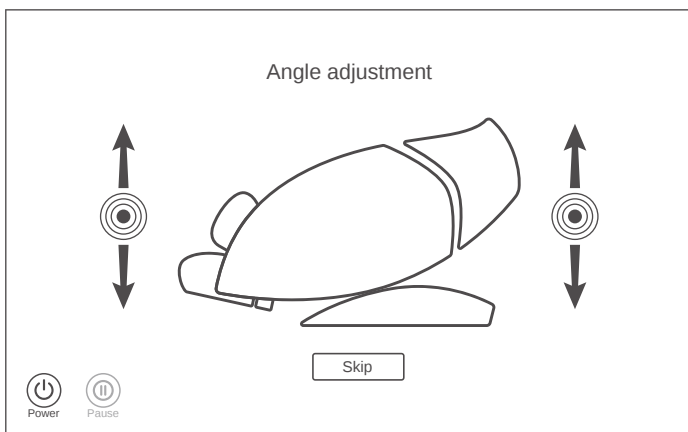
## How to Use

### VIII. Instructions for operating the massage chair via the Bluetooth APP

- (1) After the device is successfully connected, the Bluetooth APP will automatically redirect to the standby interface. As shown in the figure below, select and click the "Power" key in the lower-left corner of the menu bar to enable the massage function.
- (2) After enabling, the system will automatically enter the interface to display the startup screen and wait for the next operation.

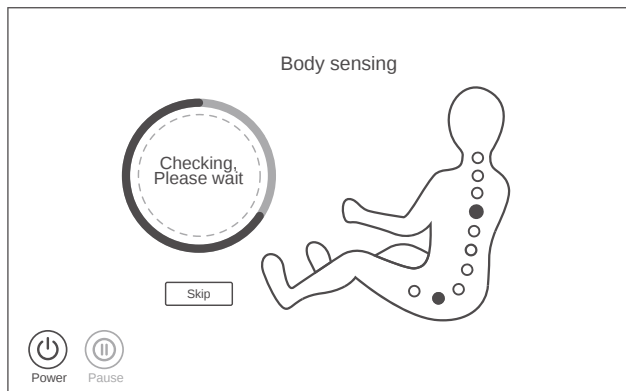
- (3) Auto Lying Down: After the massage intensity is selected, the system will automatically enter the "Angle adjustment" interface. The massage chair will automatically adjust the lying Angle. Just wait for the adjustment to complete. If no adjustment is needed, you can directly click the "Skip" key. The massage chair will maintain the current sitting position and proceed to the next step of intensity selection.

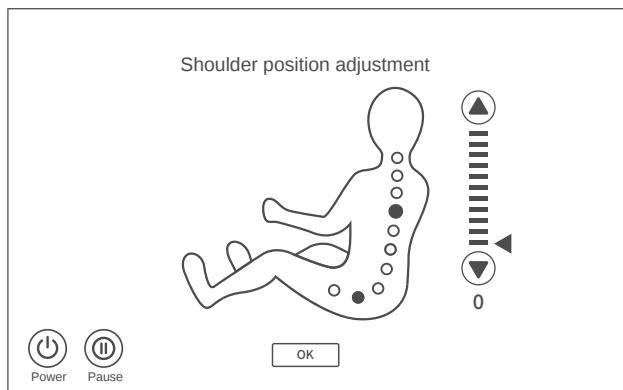
 How to Use

- (4) **Body Sensing Progress:** After the massage chair automatically reclining and adjusting, the system will automatically enter the "Body sensing" interface. The massage chair will automatically detect the user's body shape. After the detection is completed, the mechanism will be positioned at the shoulder. If no detection is needed, you can directly click the "Skip" key. The system will default the body shape and start the massage directly, and then enter the shoulder position adjustment interface.

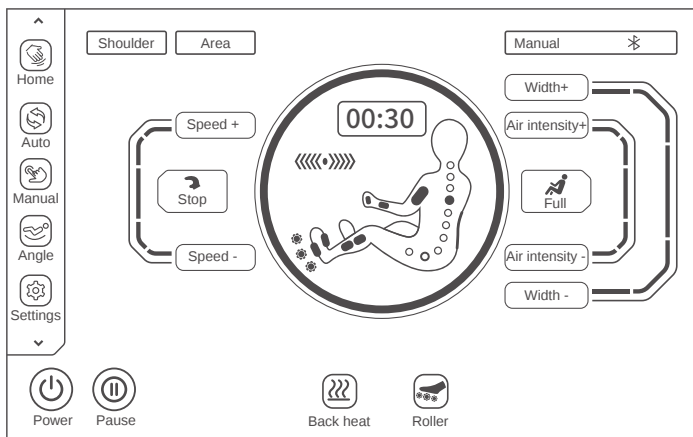
- (5) **Shoulder Position Adjustment:** After the system enters the "Shoulder Position Adjustment" interface, a voice broadcast will prompt you to manually make fine adjustments to the shoulder position. If the mechanism stops at your shoulder position, no adjustment is needed. You can directly click "OK" to start the massage. If the massage mechanism is not at your shoulder position, click the up "▲" key or the down "▼" key to adjust the mechanism to your shoulder position (there are 1 to 11 adjustable shoulder positions). After the adjustment is completed, click "OK" to start the massage.

Note: If no operation is performed within 10 seconds, the system will default the current mechanism position to your shoulder position and start the massage directly.

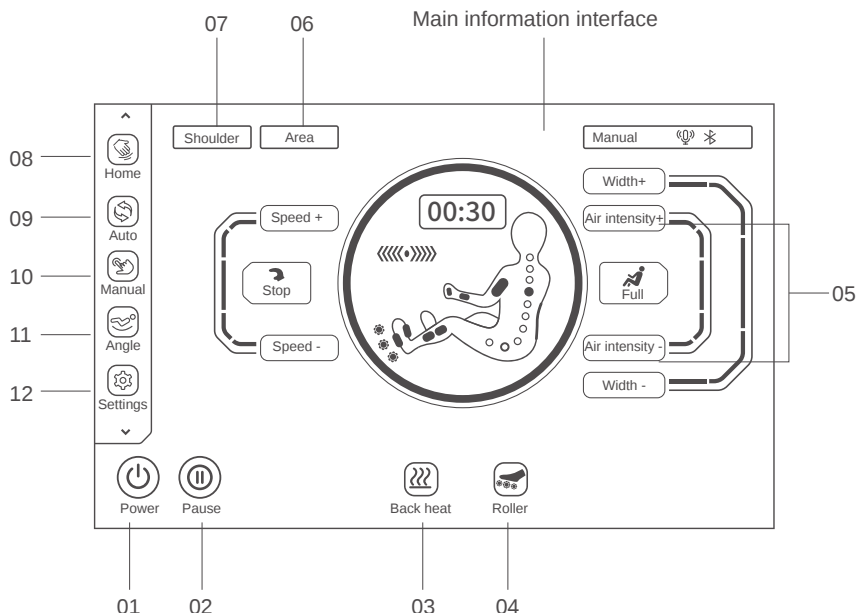
## How to Use

- (6) After the body shape detection or shoulder position detection is completed, you can enter the main interface and select the corresponding massage function according to your own needs.

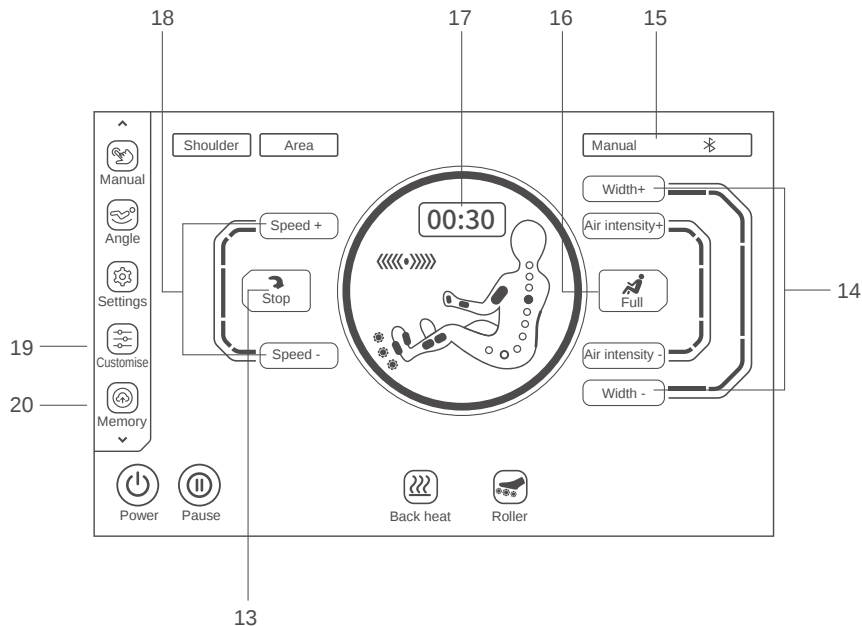
## How to Use

### (7.1) Remote controller keys instruction

01. On/off key: Enable or disable the massage chair function
02. Pause key: Pause The massage or exit from pause massage
03. Back heat key: Turn the Back heat function on/of
04. Roller key: Enable or disable the sole roller function
05. Air pressure strength key: Adjust air pressure intensity massage function.
06. Massage areal key: Select the appropriate area for massage
07. Shoulder position adjustment key: Regulate shoulder position and confirm massage criterion.
08. Current status key: Enter to view current massage chair status
09. Auto function menu key: Enter the Auto massage function interface
10. Manual function menu key: Enter the Manual massage function interface
11. Angle function menu key: Enter the Angle adjustment interface to select sitting posture angle.
12. Setting menu key: Enter the setting menu interface

## How to Use

13. Massage technique selection key: Select the massage technique
14. Massage width key: Adjust the massage width.
15. Status display bar key: Display the massage program status information.
16. Air massage selection key: Select air position.
17. Time adjustment menu key: Adjust the massage time.
18. Massage speed key: Adjust the massage speed.
19. Customise keys: Enter the customise interface.
20. Memory key: Enter the memory interface

## How to Use

### IX. Automatic program (please use this function after the chair is turned on)

Click the "Auto" key on the left menu bar (as shown in the figure below) to enter the automatic function interface. Then, select the automatic massage function according to the massage needs in different scenarios.

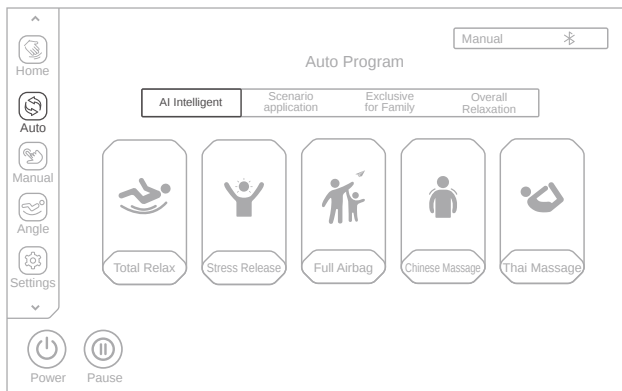
1. There are 18 automatic functions to choose from, classified into four major massage scenarios.

1.1. AI Intelligent: Total Relax, Energy Boost, Superior Enjoyable, Neck & Shoulder Care, Thai Massage

1.2. Scenario application: Chinese Massage, Fatigue Recovery, Gentle Waves Rub, Meridian Treatment, Waist Care

1.3. Exclusive for Family: Leg & foot Relieve, President Cosy, Keep Fit, Office Regimen, Health Recharge,

1.4. Overall Relaxation: Lunch Break, Warming and detox, Sweet Dream.

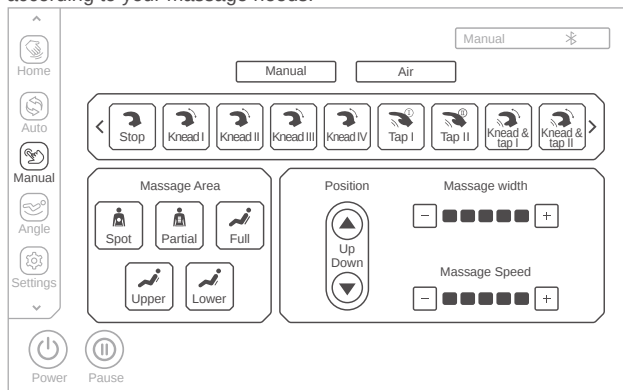
**Note:** For detailed description of each automatic function name, please refer to pages 18-20 of this manual.

Schematic diagram of automatic massage program selection

### X. Remote program (please use this function after the equipment is turned on)

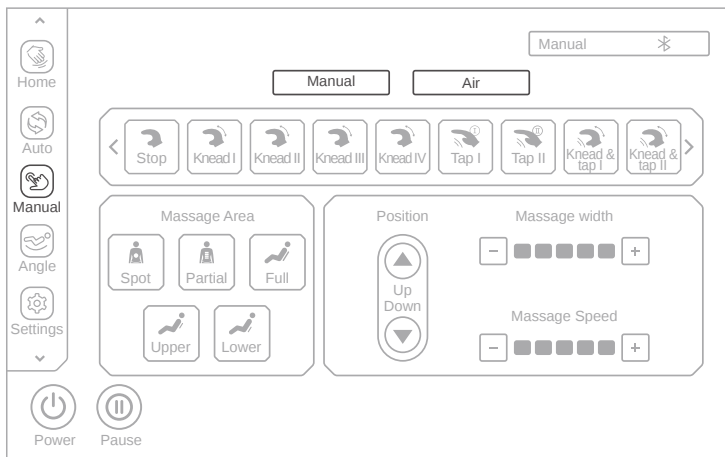
Click the "Manual" button on the left menu bar (as shown below). After the system enters the manual mode interface, you can select the corresponding manual massage function according to your massage needs.

Manual mode - Manual massage schematic diagram

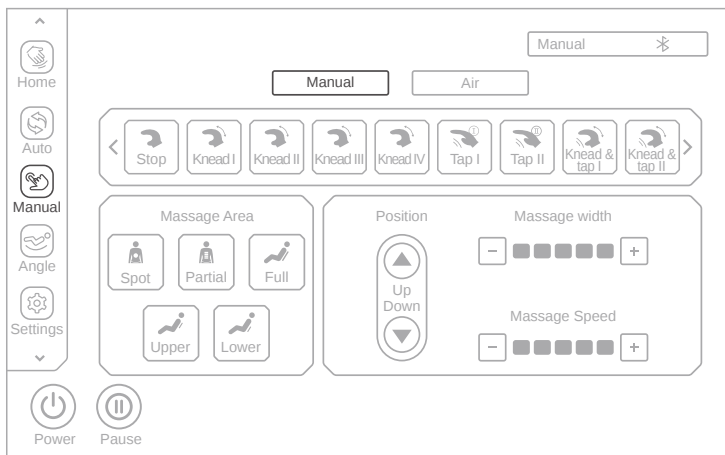
## How to Use

- (1) Manual mode: After entering the manual mode interface, as shown in the following figure, the manual massage function is mainly divided into 2 categories to choose from.
  - (1.1) Manual: Massage technique, Massage area, Mechanism up/down adjustment, Massage width, Massage speed
  - (1.2) Air massage: Full, Upper, Lower modes and air pressure intensity adjustment

Manual mode - Schematic diagram of manual function selection

- (2) Manual: After entering the manual mode interface, As shown in the figure below, click the "Manual" button, and you can select the Manual massage function according to your needs, including Massage techniques, Massage area, Massage hand up and down adjustment, Massage width, Massage speed and other massage functions.

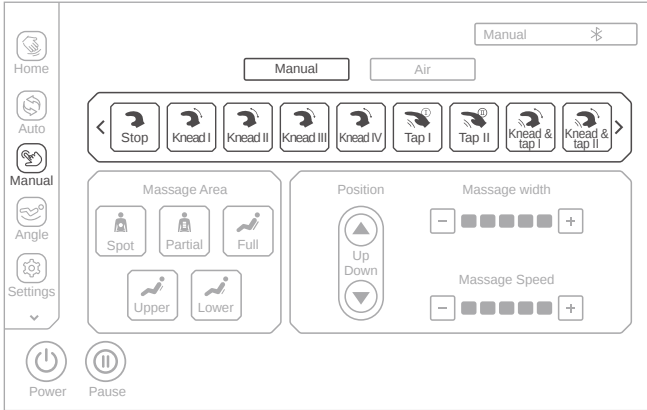
Manual mode - Manual selection diagram

## How to Use

- (2.1) Technique: After entering the Manual massage function menu, as shown in the figure below, click the Technique "icon" key to perform technique massage, including Stop, Knead (4 techniques, Knead I - Knead IV), Tap (2 techniques, Tap I - Tap II), Knead&Tap (4 techniques, Knead&Tap I - Knead&Tap IV), Shiatsu, etc., with a total of 18 techniques available. (Note: Click the "Left Arrow" and "Right Arrow" keys in the technique adjustment menu to scroll left and right to display more massage techniques).

Manual mode - Manual Schematic diagram of technique selection

| Function      | Message roller adjustment |                                   |
|---------------|---------------------------|-----------------------------------|
| Stop          |                           | The width can be adjusted.        |
| Knead I       |                           | The speed can be adjusted.        |
| Knead II      |                           | The speed can be adjusted.        |
| Knead III     |                           | The speed can be adjusted.        |
| Knead IV      |                           | The speed can be adjusted.        |
| Tap I         |                           | The speed, width can be adjusted. |
| Tap II        |                           | The speed, width can be adjusted. |
| Knead&Tap I   |                           | The speed can be adjusted.        |
| Knead&Tap II  |                           | The speed can be adjusted.        |
| Knead&Tap III |                           | The speed can be adjusted.        |
| Knead&Tap IV  |                           | The speed can be adjusted.        |
| Shiatsu       |                           | The speed can be adjusted.        |

5 speed levels  
adjustable

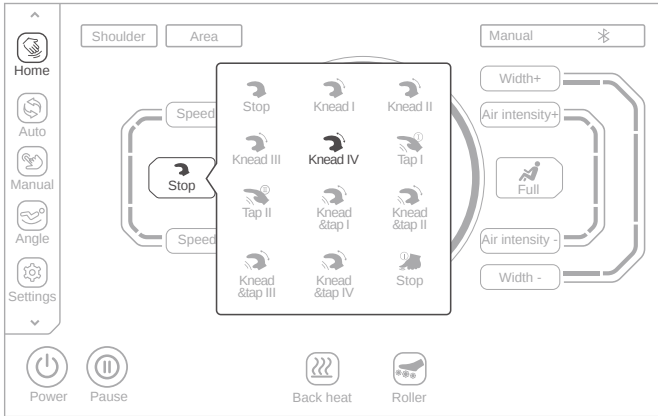
5 width levels  
adjustable

## How to Use

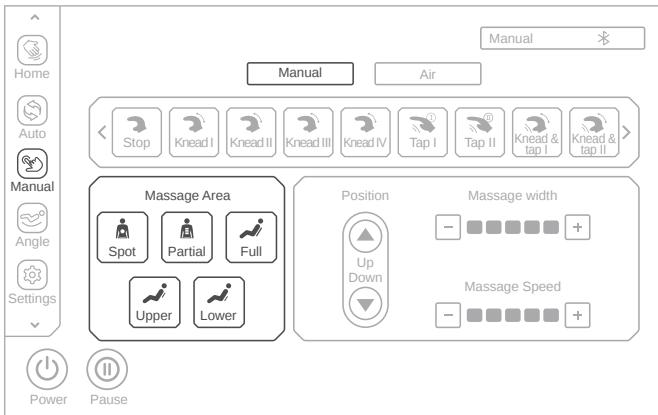
Note: During the massage program running, as shown in the figure below, click the "Home" key on the left side of the screen, enter the interface and click the Technique "icon" key, then select the corresponding technique icon (such as kneading, tapping, shiatsu, etc.) in the pop-up window to switch massage techniques.

- (1.1.1) Message shortcut method: This feature allows for quick switching between massage techniques in manual mode within the current state interface; please refer to the illustration provided.

Home interface - Schematic diagram of Manual selection

- (2.2) Message Area: After entering the Manual massage function menu, as shown in the figure below, select the "Message Area" key to perform area massage. It includes 5 massage areas: Spot, Partial, Full, Upper, Lower. Select the corresponding massage area in the massage range menu for massage.

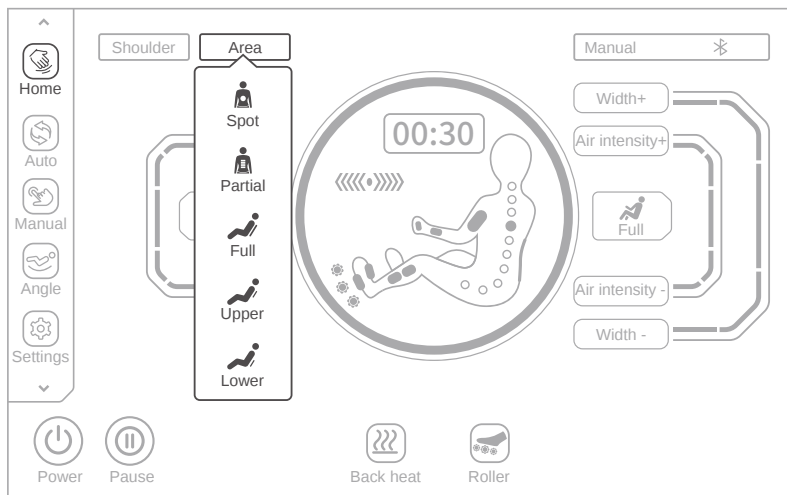
Manual mode - Schematic diagram of Manual massage area selection

## How to Use

| Key                                                                               | Description                                  |
|-----------------------------------------------------------------------------------|----------------------------------------------|
|  | Fixed-point massage at fixed-point position. |
|  | Small-scope local massage back and forth.    |
|  | Full back massage back and forth.            |
|  | Upper back massage back and forth.           |
|  | Lower back massage back and forth.           |

Note: During the massage program running, as shown in the figure below, click the "Home" key on the left side of the screen, enter the interface and click the "Area" key, then select the corresponding massage area icon (such as fixed point, partial, full back, etc.) in the popup window to switch massage areas.

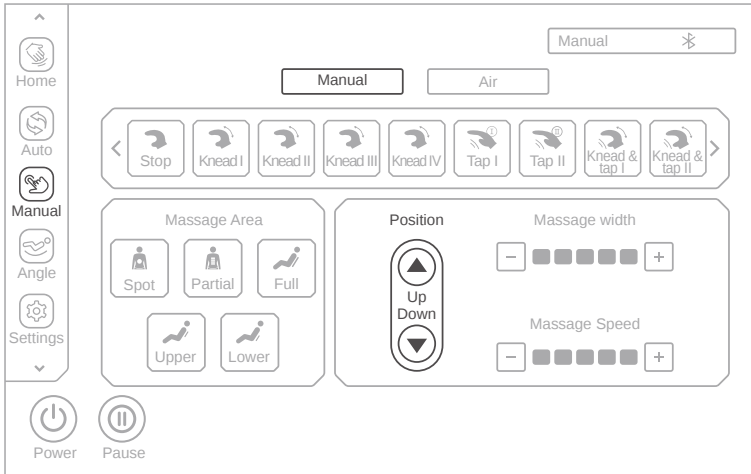
(2.2.1) Massage Area Shortcut: This function is a shortcut to switch massage areas in the current status interface in manual mode.

Home interface - Schematic diagram of massage area selection

## How to Use

(2.3) Message Hand Adjustment: After entering the Manual message function menu, as shown in the figure below, select the message hand "▲" or "▼" key to adjust the up and down position of the message hand. The message hand adjustment needs to be performed under the fixed point and partial message area in manual mode.

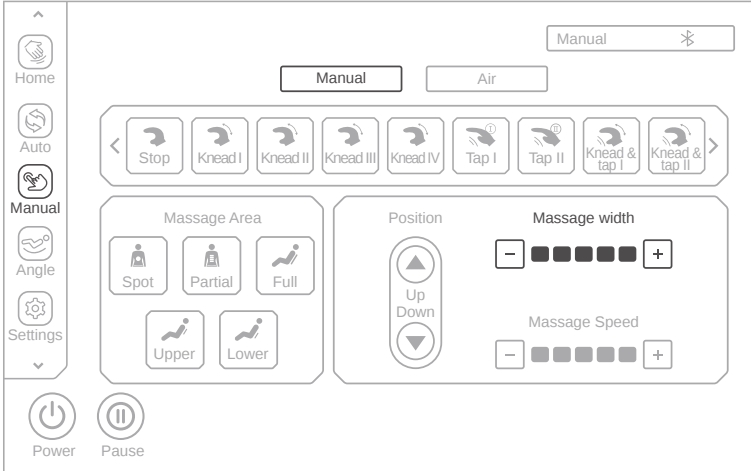
Manual mode - Schematic diagram of Manual position adjustment

| Key | Description                                                                                                   |
|-----|---------------------------------------------------------------------------------------------------------------|
| ▲   | Hold the up key, the massage hand will move up, release it, the massage hand will be stop to moving up.       |
| ▼   | Hold the down key, the massage hand will move down, release it, the massage hand will be stop to moving down. |

## How to Use

- 2.4. Massage Width: After entering the Manual massage function menu, as shown in the figure below, select the massage width "+" or "-" key to adjust the massage width (Width is adjustable from levels 1 to 5, with level 1 being the narrowest, and level 5 being the widest). width adjustment can be used in the following massage techniques: Stop, Tap I, Tap II, shiatsu. Users must select a massage technique or massage area to enter manual mode before they can adjust the width.

Manual mode - Schematic diagram of Manual massage width

| Manual mode - Schematic diagram of Manual massage width adjustment |         |         |         |         |
|--------------------------------------------------------------------|---------|---------|---------|---------|
|                                                                    |         |         |         |         |
| Level 1                                                            | Level 2 | Level 3 | Level 4 | Level 5 |

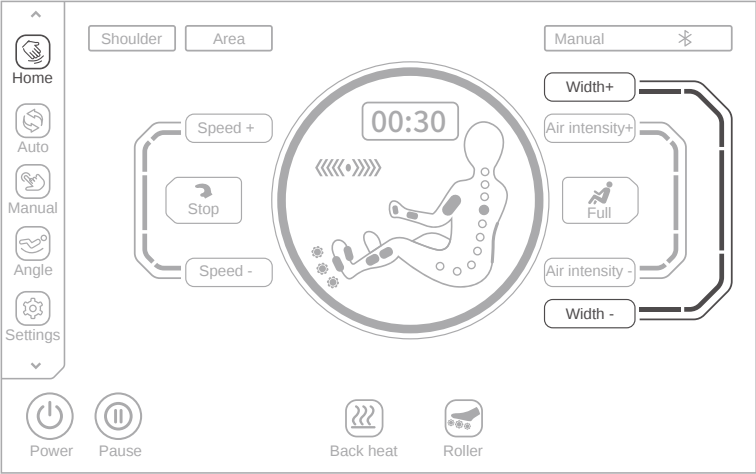
| Key | Description                                                               |
|-----|---------------------------------------------------------------------------|
|     | Press this key to increase massage width which is adjustable from 1 to 5. |
|     | Press this key to decrease massage width which is adjustable from 1 to 5. |

## How to Use

Note: During the operation of the massage program, as shown in the figure below, click the "Home" key on the left side of the screen, enter the interface, and then click the "Width+" or "Width-" key to adjust the massage width(Width is adjustable from levels 1 to 5, with level 1 being the narrowest, and level 5 being the widest)..

(2.4.1)Massage Width Shortcut: This function serves as a shortcut to switch Massage Width in the current status Width under manual mode.

Home interface - Schematic diagram of Massage Width

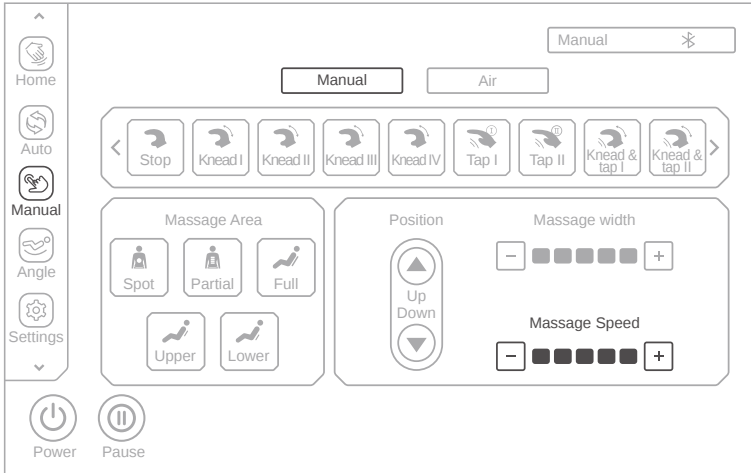
| Home interface - Massage Width adjustment                                                     |                                                                                               |                                                                                               |                                                                                               |                                                                                               |
|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
| <br>Level 1 | <br>Level 2 | <br>Level 3 | <br>Level 4 | <br>Level 5 |

| Key                                                                                 | Description                                                                                   |
|-------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
|  | Press this key to widen the massage width, the width can be adjusted from 1 to 5 levels.      |
|  | Press this key to narrow the massage width, and the width can be adjusted from 1 to 5 levels. |

## How to Use

- (2.5) Message Speed: After entering the Manual message function menu, as shown in the figure below, select the message speed "+" or "-" key to adjust the message speed (Speed is adjustable from levels 1 to 5, with level 1 being the slowest, and level 5 being the fastest). the speed cannot be adjusted when the massage technique is stopped. Users must select a massage technique or massage area to enter manual mode before they can adjust the speed.

Manual mode - Schematic diagram of Manual message speed

| Manual mode - Schematic diagram of Manual message speed adjustment |         |         |         |         |
|--------------------------------------------------------------------|---------|---------|---------|---------|
|                                                                    |         |         |         |         |
| Level 1                                                            | Level 2 | Level 3 | Level 4 | Level 5 |

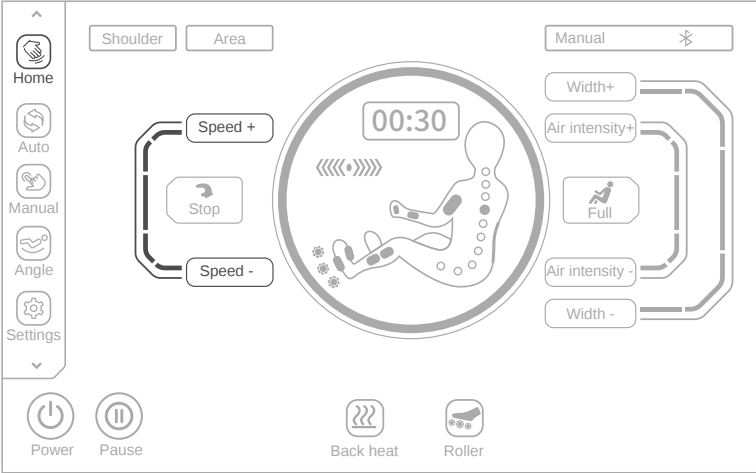
| Key | Description                                                               |
|-----|---------------------------------------------------------------------------|
|     | Press this key to increase message speed which is adjustable from 1 to 5. |
|     | Press this key to decrease message speed which is adjustable from 1 to 5. |

## How to Use

Note: During the operation of the massage program, as shown in the figure below, click the "Home" key on the left side of the screen, enter the interface, and then click the "Speed+" or "Speed-" key to adjust the massage speed (Speed is adjustable from levels 1 to 5, with level 1 being the slowest, and level 5 being the fastest).

(2.5.1) Massage Intensity Shortcut: This function serves as a shortcut to switch massage intensity in the current status interface under manual mode.

Home interface - Schematic diagram of Massage Speed

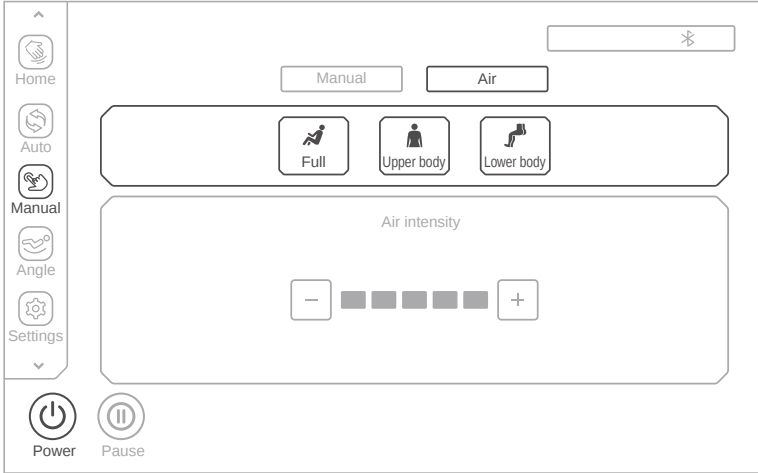
| Home interface - Massage Speed adjustment                                                     |                                                                                               |                                                                                               |                                                                                               |                                                                                               |
|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
| <br>Level 1 | <br>Level 2 | <br>Level 3 | <br>Level 4 | <br>Level 5 |

| Key                                                                                 | Description                                                               |
|-------------------------------------------------------------------------------------|---------------------------------------------------------------------------|
|  | Press this key to increase massage speed which is adjustable from 1 to 5. |
|  | Press this key to decrease massage speed which is adjustable from 1 to 5. |

## How to Use

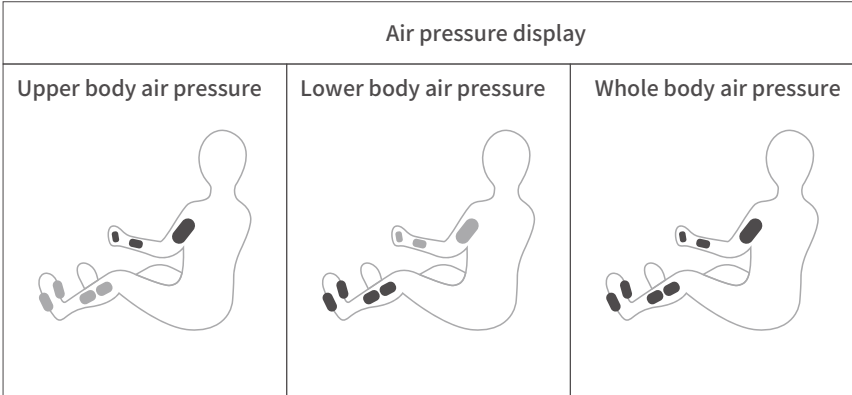
- (3) Air massage: After entering the manual mode interface, As shown in the figure below, click the "Air" button, to select the corresponding air pressure massage function according to your massage needs. Including full body, upper body, lower body air pressure mode, and air pressure intensity adjustment.
- (3.1) Air pressure area selection: After entering the Air menu, you can select the "Full, Upper body, Lower body" buttons, according to your massage needs to turn on or off the air pressure massage function of the corresponding area. If you turn off all, all air pressure functions will be turned off.

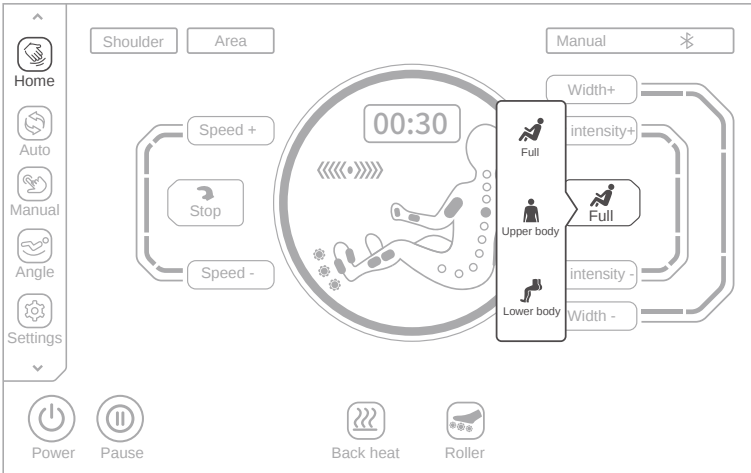
Manual program - Schematic diagram of air massage

| Key | Description                                                              |
|-----|--------------------------------------------------------------------------|
|     | Press this key to enable or disable the whole body air massage function. |
|     | Press this key to enable or disable the upper body air massage function. |
|     | Press this key to enable or disable the lower body air massage function. |

## How to Use

Note: When the massage program is running, click the "Home" button on the left side of the screen as shown in the figure below, click the air pressure "icon" button after entering the interface, and then select the corresponding air pressure massage icon (such as Full, Upper, Lower) in the pop-up window to switch to air pressure massage area.  
(2.1.2) Air pressure massage shortcut: The function here is a shortcut to switch to air pressure massage in the Home interface in manual mode.

Home interface - Schematic diagram of air massage

## How to Use

(3.2)Air pressure intensity: After entering the air pressure massage function menu, turn on the air pressure massage function of the corresponding area as shown in the figure below. to massage needs, increasing or decreasing the air pressure intensity step by step.(Intensity is adjustable from levels 1 to 5, with level 1 being the weakest, and level 5 being the strongest.)

Manual mode - Schematic diagram of air massage

| Manual mode - Air intensity adjustment |         |         |         |         |
|----------------------------------------|---------|---------|---------|---------|
|                                        |         |         |         |         |
| Level 1                                | Level 2 | Level 3 | Level 4 | Level 5 |

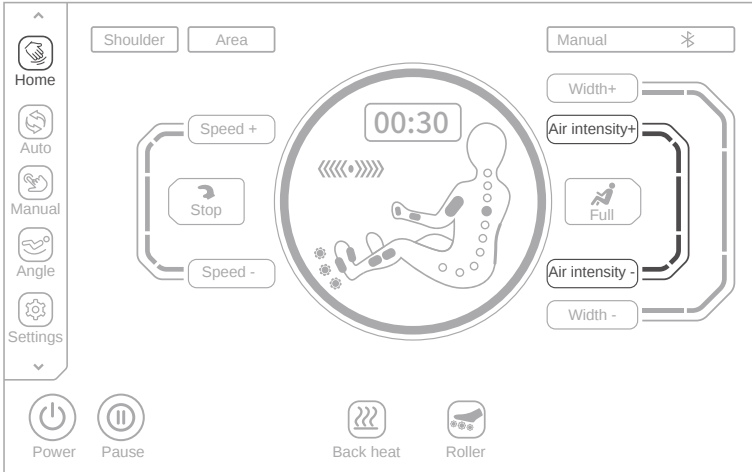
| Key | Description                                                                    |
|-----|--------------------------------------------------------------------------------|
|     | Press this key to increase the air intensity which is adjustable from 1 to -5. |
|     | Press this key to increase the air intensity which is adjustable from 1 to -5. |

## How to Use

Note: When the massage program is running, click the "Home" button on the left side of the screen as shown in the figure below, and then click the "Air intensity+" or "Air intensity-" button to adjust the air pressure intensity, to increase or decrease the air pressure intensity step by step. The intensity can be adjusted from 1 to 5 levels, level 1 represents the weakest intensity, and level 5 represents the strongest intensity.

(3.2.1) Air pressure intensity shortcut: You must first turn on the air pressure massage function for the corresponding massage area (such as full body, upper body, lower body) in the manual interface before you can use the above shortcut adjustment method.

Home interface - Schematic diagram of Air intensity

| Home interface - Air intensity adjustment |         |         |         |         |
|-------------------------------------------|---------|---------|---------|---------|
| Level 1                                   | Level 2 | Level 3 | Level 4 | Level 5 |

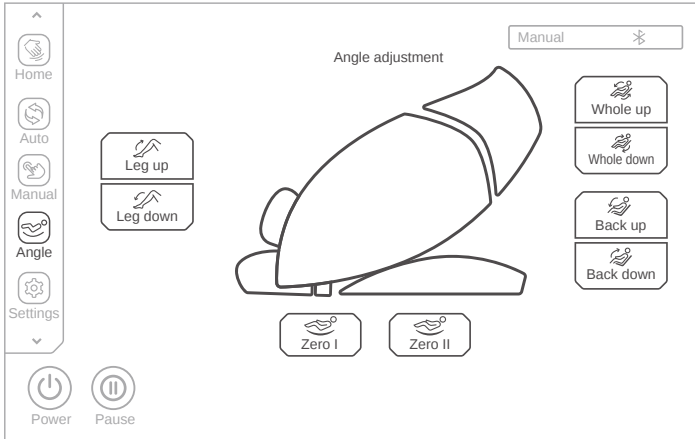
| Key            | Description                                                                    |
|----------------|--------------------------------------------------------------------------------|
| Air intensity+ | Press this key to increase the air intensity which is adjustable from 1 to -5. |
| Air intensity- | Press this key to decrease the air intensity which is adjustable from 1 to -5. |

## How to Use

### XI. Seat adjustment

Click the "Angle" key on the left menu bar (as shown in the figure below) to enter the Angle menu. According to the massage needs, select the sitting angle adjustment, including Foot shorten, Foot extend, Leg up, Leg down, Whole up, Whole down, Back up, Back down, Zero I, Zero II.

Schematic diagram of sitting posture adjustment

#### 1. Zero-gravity adjustment

| Key                                                                                           | Description                                                                                                       |
|-----------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------|
| <br>Zero 1   | Zero-gravity I key: automatically adjust the position of leg unit and backrest unit to zero-gravity I position.   |
| <br>Zero 2 | Zero-gravity II key: automatically adjust the position of leg unit and backrest unit to zero-gravity II position. |

#### 2. Leg unit extension/shortening adjustment

| Key                                                                                             | Description                                                                                                                                                                                                                                                          |
|-------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <br>Leg up   | Leg unit Up key: press and hold this key to slowly raise the leg unit, and release it to stop rising; after the leg unit rising action is completed, the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position.       |
| <br>Leg down | Leg unit Down key: press and hold this key to slowly lower the leg unit, and release it to stop lowering; after the leg unit lowering action is completed, the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position. |

## How to Use

### 3.Backrest unit angular adjustment

| Key                                                                                            | Description                                                                                                                                                                                                                  |
|------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <br>Back up   | Backrest unit Up key: press and hold this key to slowly raise the backrest unit and release it to stop rising. the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position.     |
| <br>Back down | Backrest unit Down key: press and hold this key to slowly lower the backrest unit and release it to stop lowering. the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position. |

### 4. Linkage lifting Angle adjustment

| Key                                                                                             | Description                                                                                                                                                                                                                                                                                                                                                                                                                                     |
|-------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <br>Whole up   | Linkage Up key: press and hold this key to slowly raise the backrest unit while slowly lowering the leg unit, and release it to stop raising the backrest unit or lowering the leg unit; after the linkage rising action is completed, the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position. Linkage Rise action is completed, the leg unit will automatically detect the Calf length       |
| <br>Whole down | Linkage Down key: press and hold this key to slowly lower the backrest unit while slowly raising the leg unit, and release it to stop lowering the backrest unit or raising the leg unit; after the linkage lowering action is completed, the adjustment will stop and a "drip" sound will be heard when the adjustment reaches the limit position. Linkage Decline action is completed, the leg unit will automatically detect the Calf length |

## XII. Setting (please use this function after the equipment is turned on)

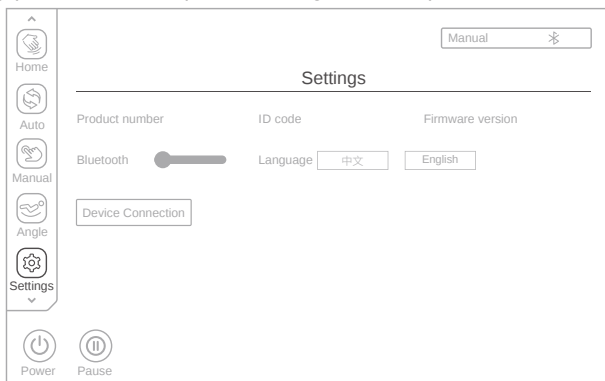
Click the "Settings" button in the menu bar on the left side of the remote control screen (as shown below), and the system will enter the settings interface. You can then select system settings functions according to your massage needs.

(1)The settings function is mainly divided into 3 categories

(1.1)Product information: Product model, ID code, Firmware version.

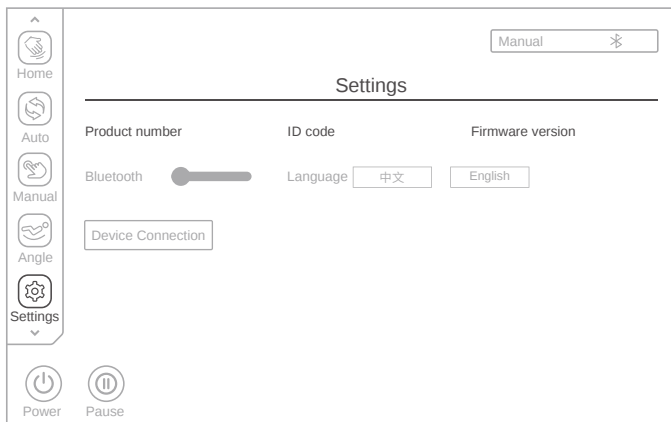
(1.2)System settings: Bluetooth, language selection and other system settings functions.

(1.3)Device connection: After the massage chair is connected to the Bluetooth APP and successfully paired, it can be operated through a mobile phone.

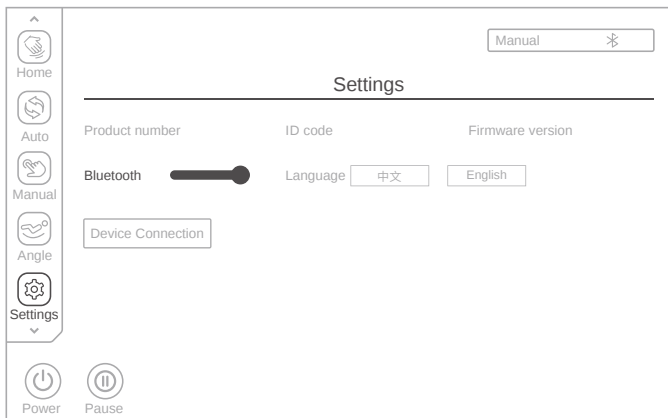
## How to Use

- (2) Device Information: After entering the settings interface, as shown in the image, you can view Product model, ID code, Firmware version show the information of current device.

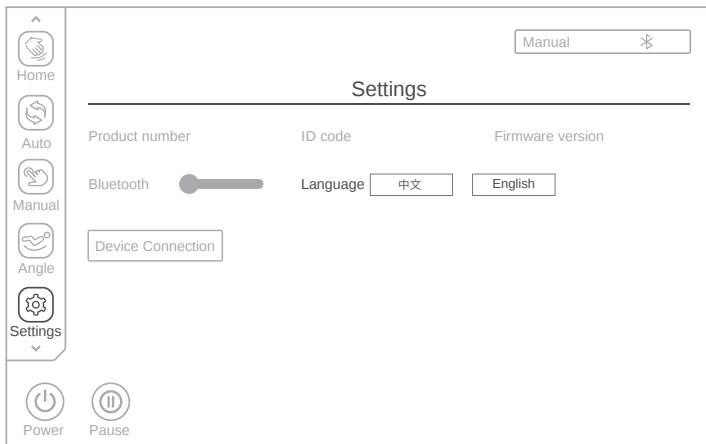
- (3) Bluetooth: After entering the settings interface, as shown in the figure below, Click "Bluetooth" switch button to enable or disable the Bluetooth feature. After you turn on the Bluetooth function, you can connect the audio device with Bluetooth to the massage chair. (such as mobile phone, MID tablet, etc.) with the Bluetooth module of the massage chair, the music played by the sound source device can be wirelessly transmitted to the sound system of the massage chair through Bluetooth for music playing.

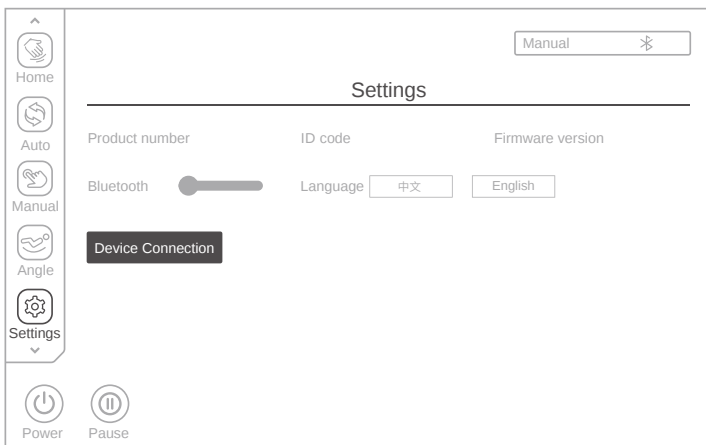
- (3.1) Enter the smart device main menu. Open "Settings" and select the "Bluetooth" option. Open "Bluetooth", and Bluetooth will automatically "Search for a New Bluetooth Device".
- (3.2) The smart device will search for the corresponding Bluetooth model (e.g. MAX-Z05A2801-\*\*\*\*). Click the "Match" button. If you are asked whether to match with the device, you just select "Match" to complete matching.
- (3.3) Turn on "Music Player" in the smart device and select the song to play.

## How to Use

- (4) Language selection: After entering the settings interface, as shown in the figure below, choose different languages according to the user's needs. The default language is English, and Chinese can be selected.

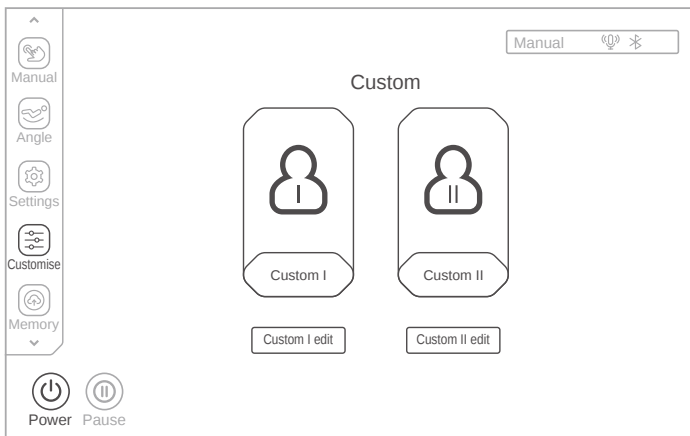
- (5) Device Connection: If you need to change the Bluetooth number or reconnect the device, enter the settings interface and click the "Device Connection" key as shown in the figure below. In the pop-up window, the system will automatically search for nearby Bluetooth device numbers, select the Bluetooth number corresponding to the massage chair from the list, and complete the APP matching. After pairing, you can control the massage chair through your phone.

## How to Use

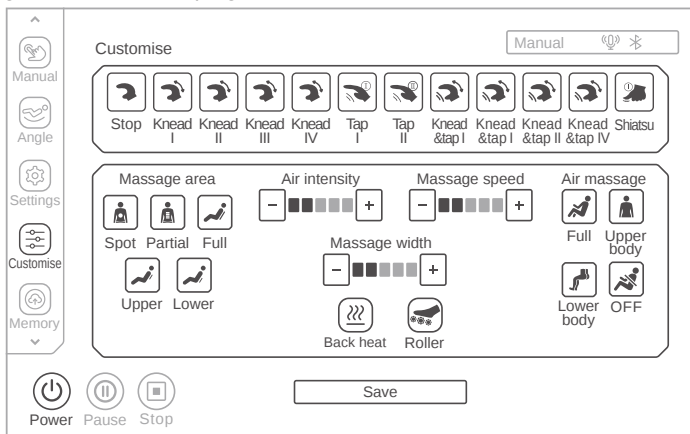
### XIII. Custom (please use this function after the equipment is turned on)

Swipe up on the left menu bar of the manual controller and select the "Custom" key (as shown in the figure below). After entering the Custom menu, interface and select either Custom I or Custom II (only if the corresponding custom has been previously set and saved) for massage.

Schematic diagram of Custom

Click Custom I Setting or Custom II Setting to enter the Custom Setting interface, then select the preferred massage technique whose icon will be highlighted after selection, and select the massage interval, 4D intensity, air massage, massage width, massage speed, air intensity, back heating, calf kneading and other functions, then click Save to save the current settings as Custom I and Custom II. After the settings are saved successfully, the custom program will be immediately available and can be used for the next massage by clicking the saved custom program.

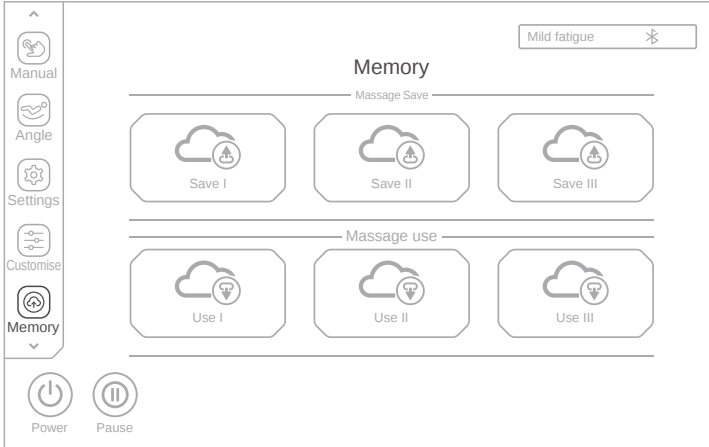
## How to Use

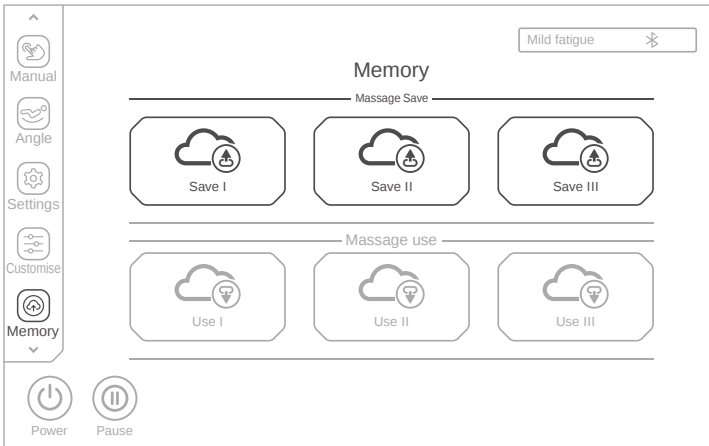
### XIV. Custom (please use this function after the equipment is turned on)

Swipe up the menu bar on the left side of the display screen, click the "Memory" button (as shown below). to enter the memory selection interface, where you can select the memory function according to your massage needs.

(1) The memory function is mainly divided into two categories: "Save" and "Use". Each category has three groups of mode selections.

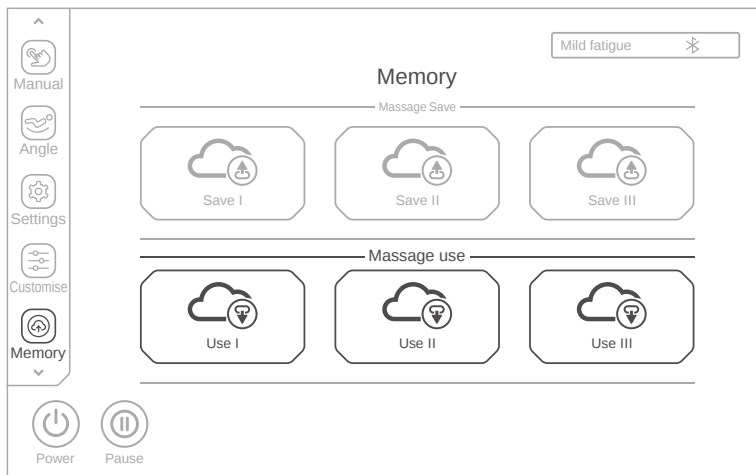
(2) Message Save: After entering the memory interface, click any of the "Save I, Save II, Save III" buttons as shown in the figure below to save all the massage programs in operation (up to three sets of massage programs can be stored); including automatic function, manual function, air pressure mode and intensity massage functions, as well as massage chair angle adjustment and other massage functions.

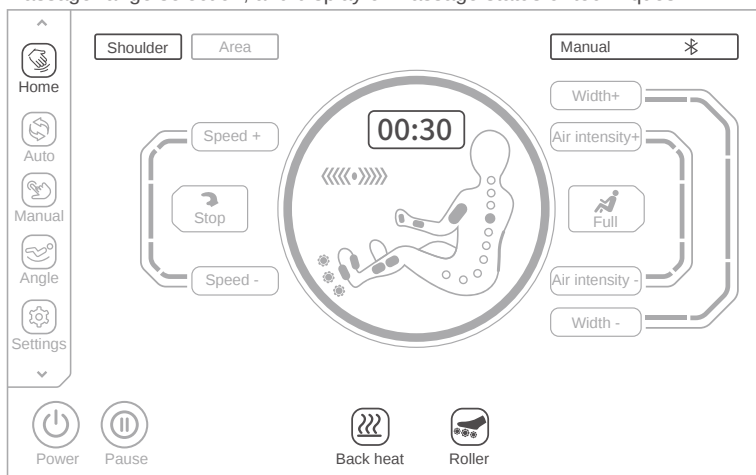
## How to Use

- (3) Message Use: After entering the storage interface, click any of the “Use I, Use II, Use III” buttons as shown in the figure below to use the stored program for message; including automatic function, manual function, air pressure mode and intensity massage functions, as well as massage chair angle adjustment and other massage functions.

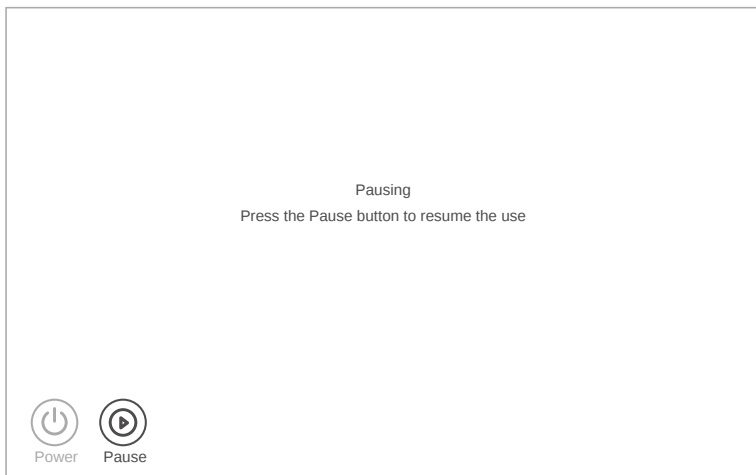
## XV. Other functions (please use this function after the chair is turned on)

On the "Home" interface of the display screen, as shown in the figure below, you can select the corresponding massage functions based on your needs. These functions include Pause, Back heat, Roller, Time adjustment, Shoulder position adjustment, Massage range selection, and display of massage status or techniques.

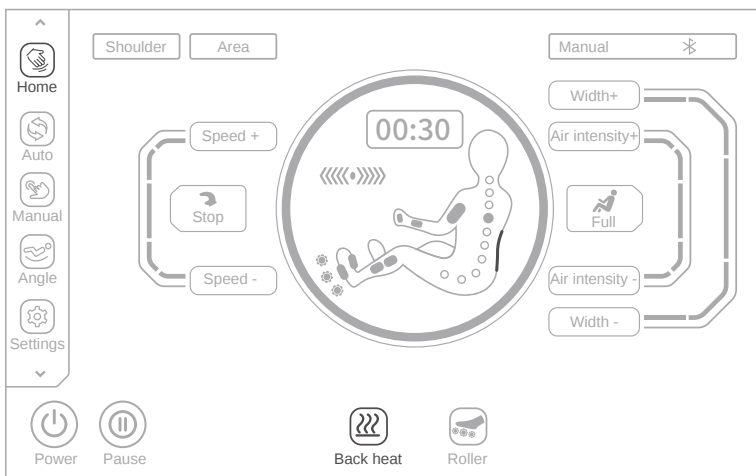
## How to Use

- (1) Press: After the massage chair is powered on and operating, press the “pause” key to enter the pause state, then all massage functions will be suspended, and press the “pause” key again to resume the previous massage state. Note: In the pause state, all functions do not work except shutdown.

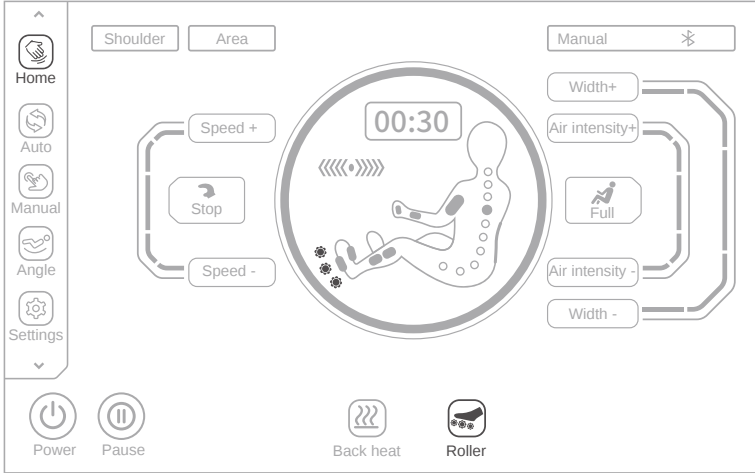
- (2) Back heat: On the "Home" interface, as shown in the figure below, select the "Back heat" key to turn the shawl heating function on or off. The heating function will generate warmth 3 minutes after being activated.

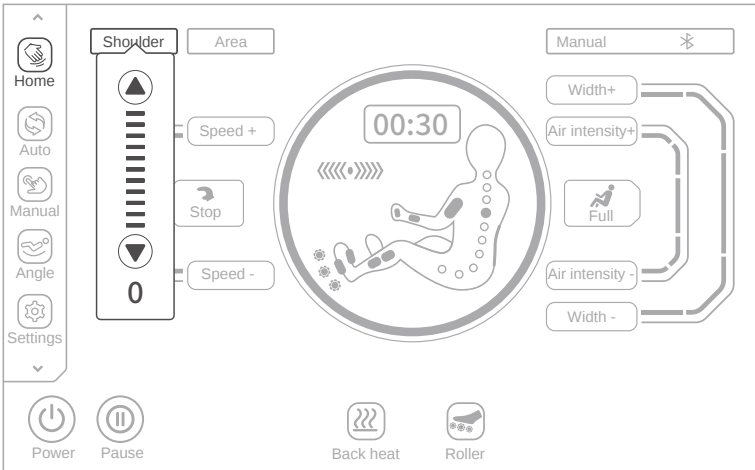
## How to Use

- (3) **Foot Rollers:** On the "Home" interface, as shown in the figure below, select the "Foot Rollers" key to turn the Foot Rollers function on or off. The roller speed can be adjusted from (level 1 to level 3). Each press of the foot roller key increases the speed by one level; pressing it a fourth time will turn off the foot roller function, and the options will cycle through accordingly.

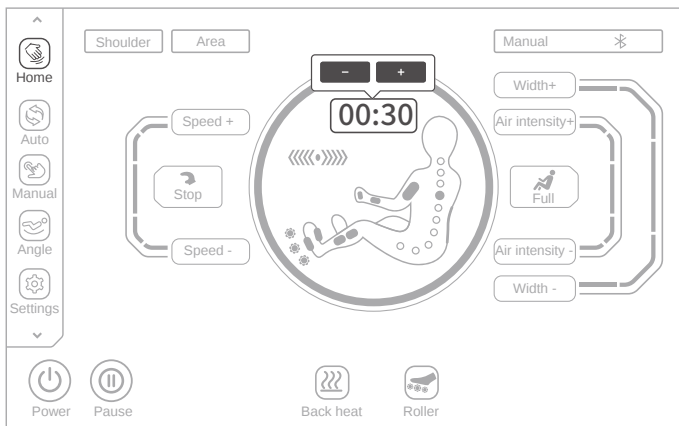
- (4) **Shoulder Position Adjustment:** On the "Home" interface, as shown in the figure below, select the "Shoulder" key, and press the up and down arrow buttons in the pop-up menu to adjust the shoulder position, with a total of 11 adjustable levels.

## How to Use

- (5) Message time adjustment: On the "Home" interface, as shown in the figure below, select the message time adjustment key "00:30", press the plus and minus arrow head button in the pop-up menu, you can increase or decrease the massage time. Click the key "+", the time increased by 5 minutes, the maximum increase to 40 minutes. Click the key "-", the time decreased by 5 minutes.

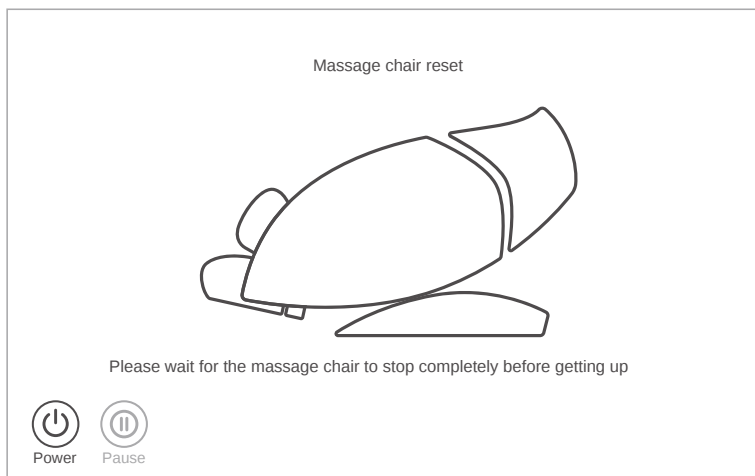
## How to Use

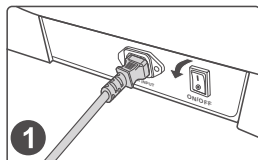
### XVI. End massage

During the massage:

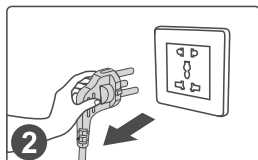
- (1) Long press the power button on the armrest to turn off all massage functions and reset the massage chair angle.
- (2) Press the ON/OFF key on the Bluetooth APP interface, all massage functions stop, and the angle of the massage chair will be return to original position.
- (3) Press the PAUSE key on the Bluetooth APP interface then press the ON/OFF button, all massage functions are stop, and the angle of the massage chair will be return to original position.
- (4) When the timer ends, all massage functions will turn off, but the massage chair will not reset to its default angle.

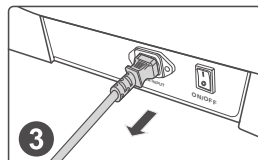
Cut off the whole machine power supply, as shown in the figure (Schematic diagram of power switch position of the whole machine).

1 Turn off the power switch ("0" position)

2 Schematic diagram of pulling out the power plug

3 Remove the product connection power cord

 **Cleaning and Maintenance****Cleaning**

- Before cleaning, please make sure the switch has been turned off and the power plug has been disconnected from the socket.
- Use soft cloths to clean the surface of product and control panel.

**CAUTION**

- Do not pull the plug from the socket with wet hands.
- Please do not rinse the product. Avoid water/ liquids from getting into the product.
- Do not use corrosive cleaning agents like benzene or thinner to clean the product, otherwise, the color of product will fade, and the internal parts may be corrupted or broken.
- Do not disassemble the product when it is humid. Simply let it dry.

**Maintenance**

- Make sure that the plug has been disconnected from the socket before maintenance.
- Keep the product clean and away from high-temperature/ humid areas, or any environment with volatile gases.
- If this product is to be left idle for long, properly cover it with a dust cover (self-purchase).

**CAUTION**

- Please keep the product away from direct sunlight, or high temperature, to avoid fading or product damage.

**Kindly reminding**

- If the product is used through the year, please check conditions as below:
  - \* Burnt smell.
  - \* Unstable power supply after contact with the power cord.
  - \* Power cord heating up.
  - \* Power cords damage.
  - \* Other abnormalities.

**CAUTION**

- If above mentioned happens, please stop using the product.
- Switch off the power supply, pull out the plug, and contact us.
- Do not disassemble the product by yourself.

## Maintenance and Cleaning of PU Leather

PU leather is made by the coating and drying of polyurethane. Compared with genuine leather, PU leather is more resistant to tear, heat and sunlight. Depending on its nature and chemical substances, however, it should be kept away from many dusty, high/low-temperature, humid environment, under direct sunlight or places with corrosive substance. Otherwise, the leather could tear, quickening aging and hydrolysis.

### Cleaning

1. Please do not use organic reagent or grease solution, especially strong solvent, e.g. alcohol, turpentine and thinner, to clean the surface of the PU leather. Otherwise it will corrode the surface.
2. Please do not use water or washing powder to clean the surface of the PU leather. Otherwise there will be cracks.
3. When cleaning the surface of the PU leather, choose a cleaning agent that has a pH value of between 5~7, no dentifrice, and is harmless to the environment.
4. For dirty PU surface, apply egg whites to the surface using a piece of soft cloth. Avoid using a brush for cleaning.

### Maintenance

1. Keep a good ventilation indoor, overly dry or humid environment can accelerate the aging of leather
2. Please keep the product away from direct sunlight or air vent, otherwise the leather will be hardened. Direct sunlight will also cause fading.
3. People are generally sweatier in summer. The holes on the surface of the leather will absorb sweat. While heat and high humidity prompt the chemical reaction between certain substances in sweat and the leather, and hence, causes smell. Please clean the surface more often with a piece of lightly wet cloth.
4. Please do not use alkaline cleaning agents on the leather. It will adversely affect the elasticity of the leather and lead to cracks as a result.

## ? FAQs and Solutions

The following symptoms do not indicate malfunction. Please check the below phenomenon to find out the solution before contact customer services department. If the problem still cannot be solved, please do not dismantle or repair the product. You should stop using it and contact us immediately.

|                 |                                                                                                                                                                                                                                                                                                                                                                                                                               |
|-----------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Fault</b>    | When the calf mechanism or the backrest reaches a certain position, press the corresponding adjustment button and it will no longer change, and a continuous "beep, beep" sound will be emitted.                                                                                                                                                                                                                              |
| <b>Solution</b> | Due to product structure design and ergonomic requirements, this product has a limited protection design for the limit state of attitude adjustment. At the same time, a reminder sound will sound when the limit is reached.                                                                                                                                                                                                 |
| <b>Fault</b>    | The function does not work or some functions do not work after the machine is turned on.                                                                                                                                                                                                                                                                                                                                      |
| <b>Solution</b> | After the machine is turned on by pressing the switch key, it is necessary to manually select an automatic massage program or other working state. If it is not operated, the machine will automatically shut down after 20 minutes. If the manual operation mode is selected, the functions that are not selected may be disabled, and the working mode needs to be manually set in the manual mode.                         |
| <b>Fault</b>    | The machine makes a leather rubbing sound when adjusting its posture.                                                                                                                                                                                                                                                                                                                                                         |
| <b>Solution</b> | This product adopts high-grade anti-corrosion leather that meets environmental protection requirements. Due to the inherent characteristics of leather materials, there will be a normal slight noise when rubbing at the relative movement, which is a normal phenomenon. Note: If you hear fabric or leather tearing sound when the machine is running, please stop using it immediately and seek professional maintenance. |
| <b>Fault</b>    | The massage wheel did not reach the shoulders or neck.                                                                                                                                                                                                                                                                                                                                                                        |
| <b>Solution</b> | If the head does not touch the head cushion or the back does not touch the backrest, the shoulder position may be lower than the actual position during the body shape detection process. Sit in the deepest position of the seat, rest your head on the headrest, and start over from the beginning.                                                                                                                         |
| <b>Fault</b>    | The unit is damaged.<br>The power cord or power plug is abnormally hot.                                                                                                                                                                                                                                                                                                                                                       |
| <b>Solution</b> | To prevent accidents, be sure to contact an authorized service center.                                                                                                                                                                                                                                                                                                                                                        |

## Product Specifications

|                                |                                                           |
|--------------------------------|-----------------------------------------------------------|
| Product Name                   | : Flexi Chair                                             |
| Product Model                  | : MAX-Z05A2801                                            |
| Rated Voltage                  | : 220V~ 50Hz                                              |
| Rated Power                    | : 100W                                                    |
| Rated Time                     | : 20minutes                                               |
| Noise                          | : ≤60dB                                                   |
| Product Gross Weight           | : 86.5kg                                                  |
| Product Net Weight             | : 73.5kg                                                  |
| Product Size (Erected)         | : L1405 x W755 x H1195mm                                  |
| Product Size (Stretched out)   | : L1665 x W755 x H945mm                                   |
| Back Max. Loading Capacity     | : 95KG                                                    |
| Leg rest Max. Loading Capacity | : 25KG                                                    |
| Safety structure               | : Class I                                                 |
| Compliance                     | : GB 44246-2024 ; GB/T 4706.1-2024 ;<br>GB/T 4706.10-2024 |

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Trademark holder: Maxcare Industrial Limited